

Ropox Shower & Changing Bed

User Manual Mounting instruction

Keep this folder with the product at all times



Content

Content	2
General information	<i>3</i>
Product information	3
Technical data	4
Mounting Instructions	7
Important info before mounting	7
Electrical components	19
Wire diagram	19
Connecting cables	20
Disconnecting cables	20
Placement of the hand control	20
Labelling	21
Support	22
Spareparts	23
Accessories	24
Maintenance	26
Complaints	27

Product information



It is important to read this manual before mounting and daily use.

Product : Ropox Shower & Changing Bed, height adjustable

Producer : Ropox, DK-4700 Naestved, Phone +45 55 75 05 00

Usability – The height adjustable shower bed can be used to nurse children and adults (up to 220Kg/485lbs). The shower bed can be used at home, in day care center or institutions to obtain good and correct working conditions for the carer.



Ropox shower bed is NOT suitable for installation in especially aggressive environments (for example environments with chlorine like swimming pools or similar)

When not in use, the shower tray can be folded up against the wall to save space. By using the up and down button on the handheld control switch, the height adjustment of the shower bed can be performed.

Part Numbers

Length of shower bed	Part No. (excl. back support)	Part No. (incl. back support)
146cm/57,5in	40-25023	40-25073
178cm/70in	40-25026	40-25076
210cm/82,7in	40-25029	40-25079



General information

Technical data

Electrical connection: 110V AC, 60Hz, max 2.0A

Standby consumption: 7W

Duty cycle: Max 10%, 2 min/18 min

Height: Work surface 30 – 100 cm / 11,8-39,4in

Unit, B=60 cm/ 23,6", H=116cm/ 45,7", D=13cm/5,1"

Material – Frame: Welded steel tubes St.37, Stainless and aluminum

Shower tray: ABS

Surface treatment: Powder coating, white RAL 9010 matt, Chromate

Maximum load: 220 kg / 485lbs



This product has CE-marking according to Medical equipment, Low Voltage and EMC directives. Has been tested according to the European standards DS/ISO 17966 and DS/EN ISO 60601-1. This product is in risk group 1.

Daily use

Before start using this product, be sure that "main power" has been connected as described in electrical connections.

By using the up ▲ down ▼ bottom on the handheld control switch, the height adjustment of the shower bed can be performed. The control switch has a spiral cord, which can be pulled out to a length of 200cm/78,7in.




Maximum load onto the Shower & Changing Bed 220 kg/485lbs.

The slat

The slats have two sides with different hardness and can be turned around in the Shower tray.

The soft side is used, when the person on the Shower bed needs to be washed. The hard side is used, when the person on the Shower bed needs to be changed.

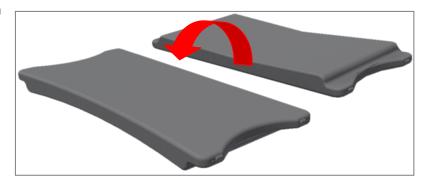

General information

After use

When not using the shower bed, the shower tray can be folded up against the Lifting unit. This way it will only take up the space of approx. 30cm/11,8in from the wall.

Inside the lifting unit, a gas damper is mounted, to reduce the power needed, when folding the tray up and down.

Intended Use

The product is intended for changing and bathing disabled persons by a carer that have appropriate training in health care.






DO NOT place hands or other limbs between the hoisting unit and wall. There is a risk of squeezing when the shower bed is operated.

Safety in use

- Ropox Shower Bed should only be used by people, who have read and understood these instructions.
- The shower bed must be operated only by adults.
- Always use the Shower Bed so there is no risk of damage to people or property. The person, who
 operates Ropox Shower Bed, is responsible for avoiding damage or injury.
- Do not operate Ropox Shower Bed in case of defects or damage.
- Do not overload Ropox Shower Bed. The maximum user weight is 220kg/485lbs
- Modifications, which may or may not influence the operation or construction of the shower bed, are not allowed.
- Installation, service and repairs must be carried out by competent personnel.
- If the washbasin has not been assembled according to mounting instructions the right to complain may become void.
- Only use Ropox original spare parts as replacement parts. If other spare parts are used, the right to complain may become void.
- To avoid risk of pinching be aware not to place your hand between the shelves and the bed when tilting it from vertical to horizontal position.
- Do not place you fingers in the gap between the wall mount and the housing of the lifting mechanism.
- The Shower bed may only be used for personal hygiene purposes
- Do not lean on the accessories attached to the shower bed
- NEVER leave a person on the shower bed unattended not even for a very short period.
- Pay particular attention to other people in the room, when using the shower bed. Make sure that no children are playing under or with the shower bed.
- Keep the hand control out of the reach of children. Make sure that they cannot operate the shower bed unattended. A bracket for wall mounting is included with delivery.
- Make sure that there is free space above and below the shower bed to allow height adjustment.
 The shower bed is up foldable, but please be aware of obstacles above, below and around the shower bed to prevent risk of trapping.
- If the control switch has been damaged or stopped working, it has to be replaced immediately. The up and down buttons are "push and hold". The motor stops when the button is being released. This is a very important safety element.
- If two shower beds are mounted next to each other, we recommend to mount a side cover between, to avoid the risk of trapping.
- Constantly be aware of objects under the shower bed.

Important info before mounting

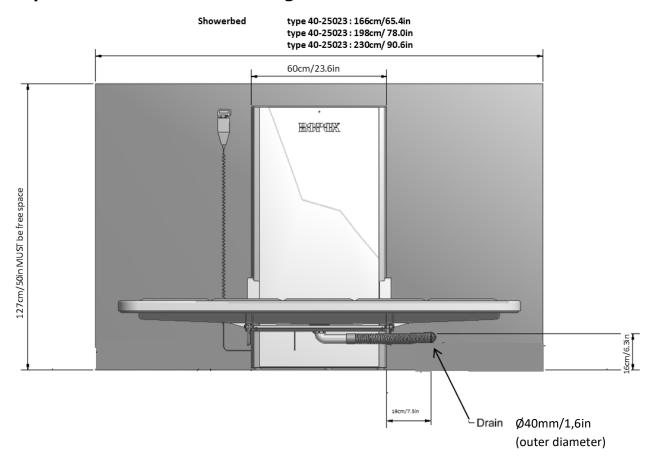

Make sure that you allow free space on all sides of the unit as shown below.

Placement of the Main 110V socket, the hot and cold Water and the Drain

The drain must be placed in the free area beside the lifting unit. The placement on the picture above is a suggestion of optimal position.


Make sure that the Main 110V socket is placed according to the national requirements for such installations.

The main power cable (110V) has a total length of approx. 320 cm of which 250 cm is outside the lifting unit. The cable can be pulled out in either left or right side. The outlet for main power cannot be placed behind the lifting unit.


Hot and cold water shall be placed so as not to not come in conflict with the area of height adjustment.

Receiving the Shower & Changing Bed:

The Ropox Shower & Changing Bed will come in two packages. One for the "Lifting unit" and one for the "Shower tray unit".

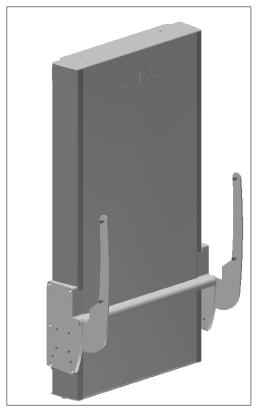

Lifting unit

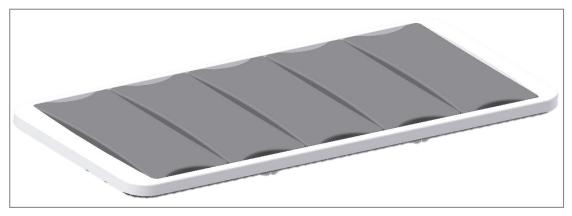


Shower tray

It is important to follow the mounting instructions to assemble the unit in the right way and to obtain a safe product.

1. Prepare the mounting / dismount the top cover plate


Pull up the cover plate on the top of the lifting unit to get access to the nut, which holds the front plate. Loosen this nut.

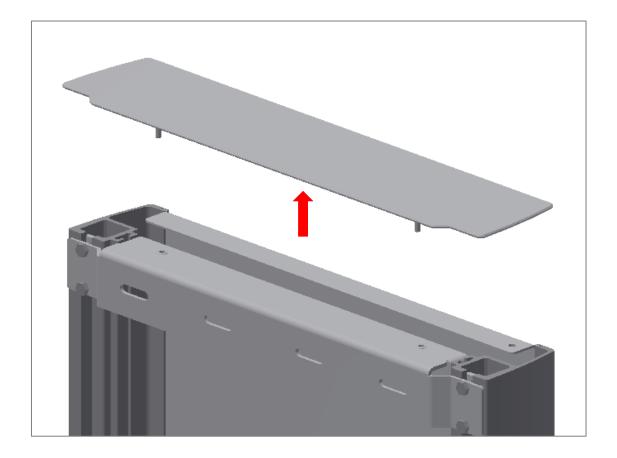

2. Prepare the mounting / dismount the front plate

Slide the front plate out of the Lifting unit to get access to the wall mounting holes in the unit.

!! Be aware not to bend or scratch the front plate in the process !!

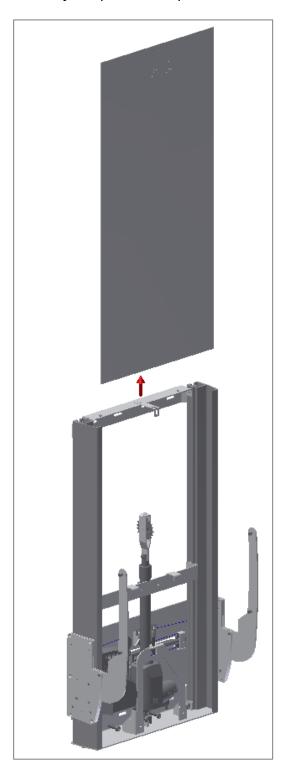

3. Mounting to the wall

Attach the frame securely to the wall by using adequate bolts and rawlplugs suitable for the type of wall construction / material on site. The Shower bed is being supplied with a set (7 pcs) of screws and rawlplugs, which is ONLY used to mount the unit in a concrete wall. If the unit must be mounted on another kind of wall, other kinds of plugs and screws must be used. Shower bed is tested according to a load of up to 220 kg/485lbs. applied to the shower bed.



If you have other objects mounted right next to this shower bed, you have to consider the risk of trapping, and therefore consider a safe distance to the other objects.

The attachment holes can be marked up on the wall, four possibilities in the top, and four in the bottom of the frame. Place the frame against the wall and make sure it is horizontal. The frame is standing on two adjustable feet. The holes in the frame are all \emptyset 10 mm/0.4in. Remove the frame and drill the holes.




Make sure that no dust or other obstacles are interfering with the rollers, chain, sprocket or gas springs. This can cause malfunction and damage the system


Force for each of the four screws in top of the unit:

 $F_{\text{screw}} = (220 \text{ kg} * 923)/(1122 * 4 \text{ screws})$

 $F_{\text{screw}} = 45 \text{ kg} / 99 \text{lbs for each screw}$

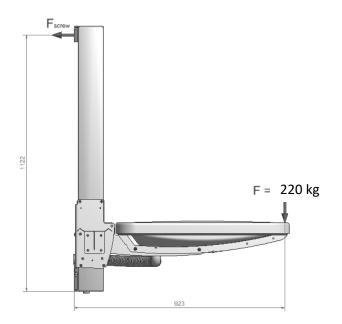




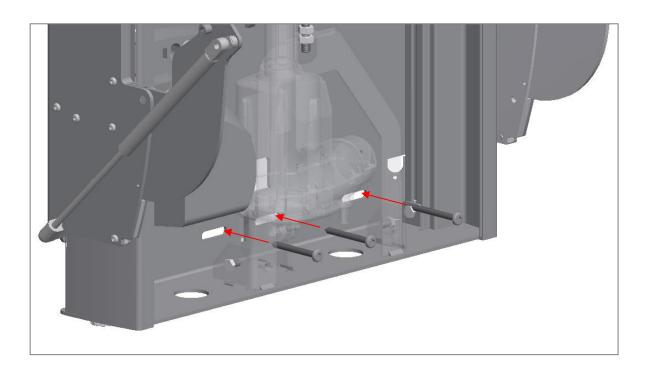


4. Remount the front plate

Slide the front plate back into the Lifting unit and fasten it with the nut.

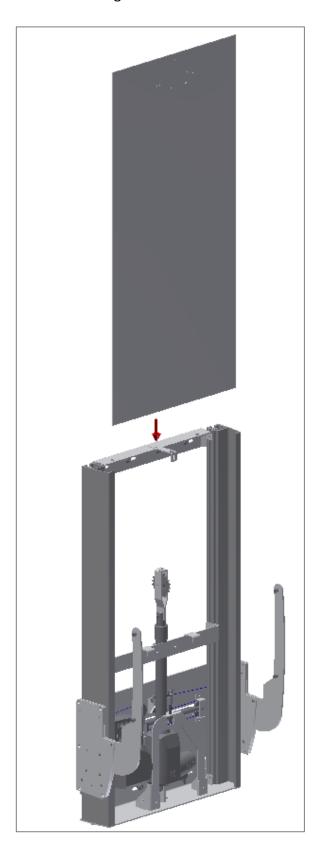

5. Mount the shelves on the Lifting unit.

Use 3 pcs. M6x6 screws to mount the shelves on both sides of the lifting unit.

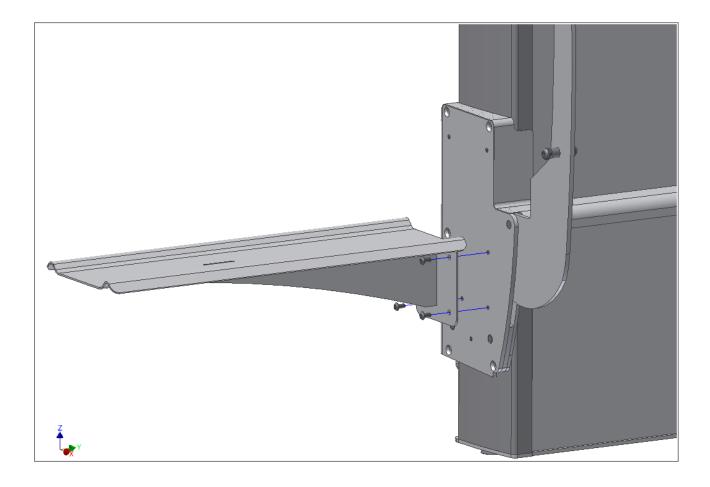


6. Mount the Shower tray on the Lifting unit.

Slide the Shower tray in between the Lifting unit and the support arms on the Lifting unit.




Fasten the Shower tray to the Lifting unit with 8 pcs. M8x12 through the bushings in the support arms.

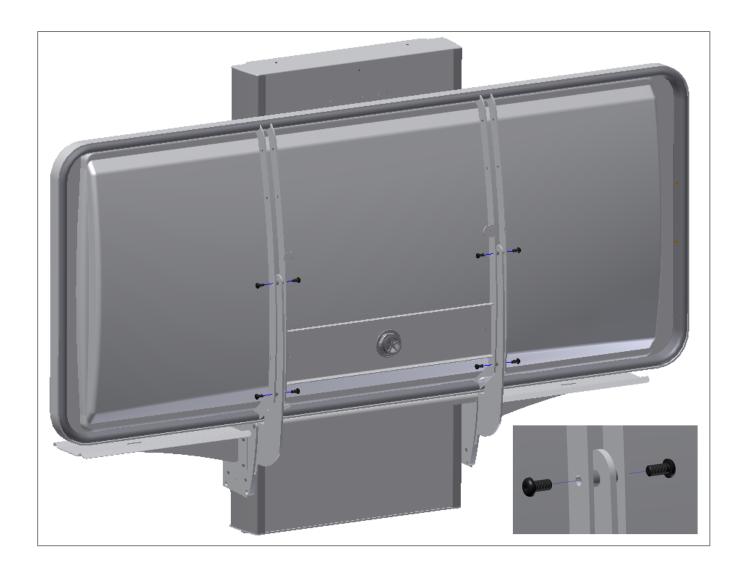


7. Mount the cover plate on Shower tray

Place the cover plate between the two support arms underneath the Shower tray.


Fasten it with 4 pcs M5x8

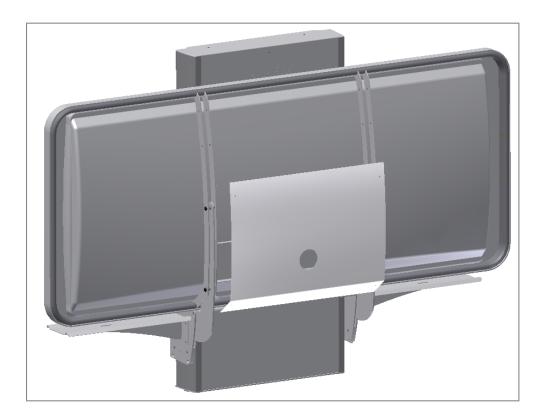


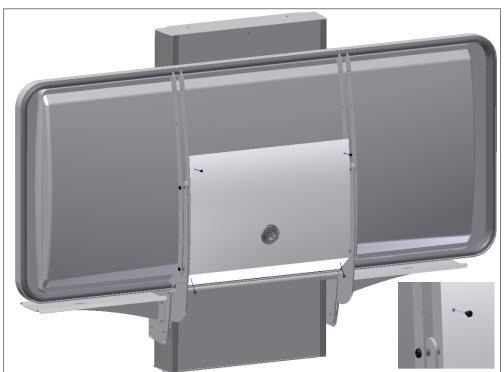

8. Mount the Drain and Flexible hose on the Shower tray

Screw the plastic nut on the scuttle and tighteen it firmly. Connect the flexible hose to the drain tube and connect it to the drain in the wall or on the floor.

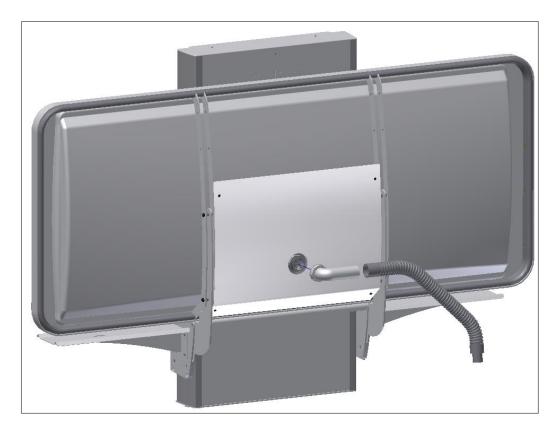


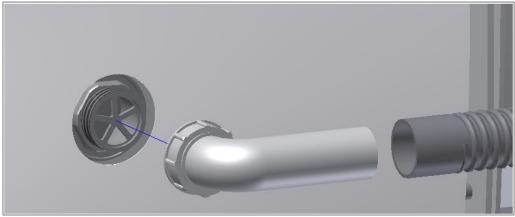

- After mounting, make sure that the chain is in correct position on the sprocket, also check that cable plugs is connected correctly to the control box.
- Check that no cables can be trapped or squeezed
- Connect the main power, and the shower bed is ready to be used.

Wire diagram

The shower bed is delivered with the control box already mounted on the frame. The diagram shows how the cables which are connected to the control box.

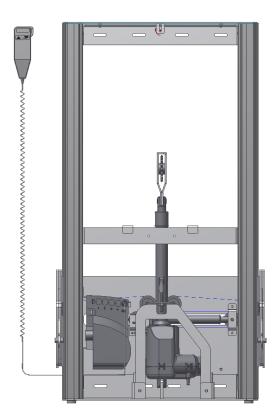


- 1. Control box
- 2. Mains cable 110V
- 3. Motor, connects to socket 1
- 4. End plugs, connects to socket 2, 3 & 4
- 5. Hand control, connects to HB socket



All the plugs are water tight, and therefore they can be a little tricky to connect / disconnect. Make sure that the plugs are positioned correctly before they are pushed into the sockets at the control box. The plugs must be pushed into the sockets until the "O-ring" is no longer visible.

Motor, control box and control switch apply to IP66. This means that they are dust tight and protected against heavy seas, or powerful jets of water.

The electronic system is designed for periodical use only, this means a max. duty cycle of 10%; 2min on /18min off.

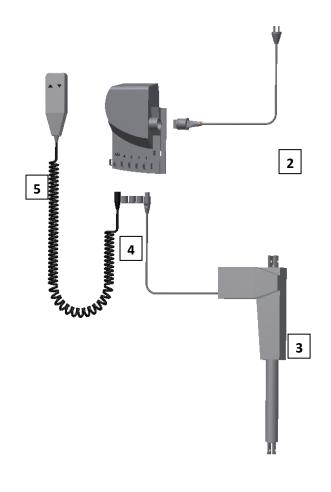

Electrical components

Connecting cables

The shower bed is delivered with the control box already mounted on the frame.

Motor cable connects to socket 1. Socket 2, 3 & 4 are not in use and are blocked.



Control switch connects to "HB".

When all cables have been connected, use the safe locking bracket. Pushed in until you hear a "click".

Mains cable connects to the socket at the side of the box.

Disconnecting cables


Mains cable can be pulled out (no safe locking bracket).

Before the other cables can be disconnected, the safe locking bracket needs to be released / removed (use a screwdriver to remove). Locking clips at each end of the safe locking bracket have to be released before the bracket can be removed, and the cables can be disconnected.

To obtain the full IP66 value, it is very important that all the plugs have been pressed firmly into the sockets and the safe locking bracket is mounted.

Placement of the hand control

Bracket for hand control can be placed at the wall as shown. To be mounted with the two screws which are delivered with the bracket.

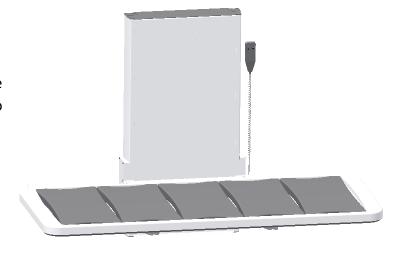




Hand control should be placed out of the reach of children

Label

Each shower bed has its own label as shown. The label contains information which is very important to specify when contacting your supplier. It also shows main power supply.

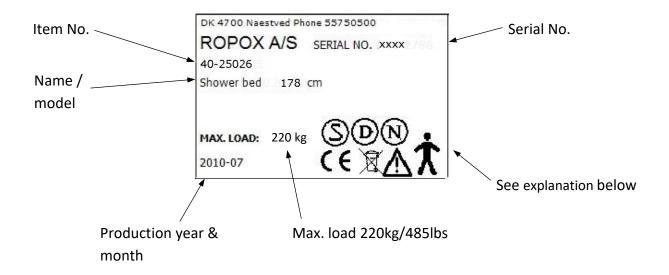

Symbol view



User Manual must be read before use



Waste must be sorted. Most parts for re-use.



Type B equipment. As for EN 60601-1



Class 2 equipment. Without earth connection.

CE-marking. This product complies with the demands for general safety and functionality in the relevant EU directives and standards.

EMC information



The shower bed complies with the demands for electromagnetic compatibility (EMC), if installation and daily use is according to instructions in this Manual.

In seldom situations the Shower bed might be affected by electronic signals from ex. Portable computers, cell phones etc.

Trouble shooting

The bed does not move up or	Check if:
down by when operating using the	Main power connected 110V.
hand control	Motor is connected to control box
	Hand control is connected to control box
	Motor is overloaded
Hand control defect	Check if:
	The product works with another hand control
By pressing the up or down button,	Plugs are not connected correctly.
no movement is achieved.	Dismount all cables and re-connect.

Waste handling



Used equipment shall be disposed of according to the National law. We recommend to dismount the product to the greatest extent possible, in a way that most parts can be reused. Be aware that this product contains electronically components. Example of groups this product can be sorted in: metal, plastic, cables, electronics and parts for destruction.

Factors affecting the operation

Temperature: 0°C to 40°C /32°F to 104°F

Relative humidity: 5% to 85% non-condensing

Factors affecting storage and transport



Spareparts

97000431	Actuator/motor Type: LA34.5 Stroke 350 mm/13.8in	
97000434	Control Box Type: CB6096-2	
97001655	Hand Control Type: HB310V0XB076-2102E1E000	
40-25030	Slat	
40*40090-315	Flexible plumbing kit	

Back support

The back support fits all shower bed lengths and uses two slats. No tools are required for mounting in the shower tray, as the back support has been designed to fit directly into the tray. The back support has three positions: 20°, 40° and 60°.

Item No. 40-25031



Recommendations for use

- Make sure that the unit has been placed correctly in the shower tray so that the two side rails of the bottom frame rest properly on the sides of the tray.
- See that the horizontal supporting rod has snapped into place before leaning on the back support.
- When folding down the back support after use, do not let go of the support, as it will fall down. Instead, carefully lower the back support to horizontal position.
- When folding down the back support, make sure that there are no fingers between the frame and the support in order to prevent fingers from getting caught.
- Note that the slats may become slippery when in contact with water and soap.

Bed guard

Bed guard is available as an option. The brackets will be mounted underneath the frame with the screws supplied.

Length of	Shower bed	Bed guard foldable	Bed end guard
Shower bed			Fixed
146cm/57.5in	40-25023 / 40-25073	40-25034	40-25038
178cm/70in	40-25026 / 40-25076	40-25036	40-25038
210cm/82.7in	40-25029 / 40-25079	40-25035	40-25038

Neck support

Neck Support is an ergonomically correct crafted neck support and it is available as an option. The Neck Support lifts and supports the user's neck, and gives a more pleasant experience. It can also be used with the Back Support.

The Neck Support is molded in one piece, of a bacterial and fungal retardant material, and is therefore extremely easy to maintain and easy to clean. The pad is furthermore waterproof.

Item No. 40-25090



Recommendations for use

Less suitable for people with uncontrolled movements.

The Shower & Changing Bed is maintenance free. For safety reasons it is recommendable to check all mounting screws and moving parts to be sure all is tightened ok, and that all attachment to the wall is in correct and safe condition.



To ensure proper function of moving mechanical parts make sure to actuate the bed to highest position and return to the lowest position. This will keep mechanical moving parts from drying out and will prolong the life time of the product.

Cleaning

Do not use a high pressure cleaner to clean this product.

Lime deposits are removed with household cleaning agent for bathrooms.

Always clean the shower tray after use to prevent dirt from sticking to the surfaces.

important:

Do not use abrasive powders or other abrasives as they will scratch the surfaces. Do not use pointy or sharp cleaning remedies.

Cleaning of slats

Lime deposits on slats are removed with household cleaning agent for bathroom.

Chemical resistance

Cleaning agents containing the following chemicals must not be used in cleaning

Acetone
Ammonium
carbonate
Ethyl acetate
Formic acid
Hydrochlorid acid
Isopropanol
Methyl acetate
Nitric acid
Phosphoric acid

See General Terms of Sale and Delivery on www.ropox.com



Ropox A/S

Ringstedgade 221 DK – 4700 Naestved

Tel.: +45 55 75 05 00 Fax.: +45 55 78 05 50

E-mail: info@ropox.dk www.ropox.com