









4Single Battery

User manual

This manual must always accompany the product

TF 200.01.0040 / Date 11-04-2022

Table of Contents

1.	Symbols	use	d in this manual	3
2.	General	safet	у	3
3.	General requirements			
	3.1	Pro	duct information	5
	3.2	Pro	duct description	5
	3.3	Inte	nded use	5
	3.4	Inte	nded population	6
	3.5	Inte	nded operator	6
	3.6	Bas	ic safety	6
4.	Charging	Charging the battery		
	4.1	Rec	harging procedure	7
	4.2	Cor	nection procedure	7
5.	Mounting	Mounting instructions		
	5.1	Ele	ctrical component diagram	8
	5.2 Mounting the control			8
	5.3 Testing		ting	9
	5.3	3.1	Before testing the frame	9
	5.3.2		Testing the frame	9
	5.3	3.3	Testing the safety-strip	9
6.	Troubles	shoot	ting	10
7.	Cleaning			10
8.	Maintena	ance		11
	8.1	Per	odic maintenance of the frame	11
	8.2	Per	odic maintenance of the battery	11
9.	Compon	ent p	part list	11
	9.1 Accessories, component part list		12	
10.	Environmental protection		13	
	10.1 Packaging, transport, and storage		13	
	10	.1.1	Packaging	13
	10	.1.2	Transport	13
	10	13	Storage	13

1. Symbols used in this manual

Indication of potentially hazardous situation. If not avoided, it can result in serious injury or death. Caution Symbol Indication of potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices. Notice Symbol This symbol is used to notify correct use and handling of the product.

2. General safety



This manual must be read and understood before use. Always keep this manual in close proximity of the product.

The use, installation and service of this product must follow this manual to avoid accidents and serious personal injury.

Never use or handle this product in other ways specified in this manual as it can result in personal safety hazards and/or cause damage to the product.

Anyone installing and/or using this product either as operator or user should have the necessary safety information and access to this manual.

If the supply cord is damaged, it must be replaced by the manufacturer, its service agent, or similarly qualified designated person to avoid a hazard.



Do not perform repairs, disassembly, or assembly operations, add-ons, re-adjustments, or modification of the product beyond what is described in this manual. These must be carried out by Ropox or Ropox authorized personnel. Do not perform service while in use.

This appliance can be used by children aged from 8 years and above and people with reduced physical, sensory, or mental capabilities, or lack of experience and knowledge if they have been given supervision, or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.

Do not use the product if it has defects or has become damaged before being repaired or replaced. If the control box makes unusual noise, or smells, switch off the mains voltage immediately.

Take care that the cables are not damaged.

Unplug the mains cable on mobile equipment before it is moved.

The products must only be used in an environment that corresponds to their IP protection. The products must not be used in explosion hazard environments.



The information in this manual is based on correct installation in accordance with installation instructions for this product.

Ropox cannot be held liable if the product is used in any way, that differs from stated in this manual and/or installation instruction.

Ropox reserve the right to amend this manual and reference documents without prior notice.

3. General requirements

3.1 Product information

Manufacturer			Ropox A/S , DK-4700 Naestved +45 55 75 05 00 nail: Info@ropox.com
	Part number	Model	Stroke length
Product models	50-47025 50-47026	Rapid stroke Heavy load	300 mm / 11.8 in 300 mm / 11.8 in
Maximum load			s. evenly distributed
Heavy load	250	kg / 550 lbs. evenly dis	stributed (accessory)
Operational capacity		30 - 40 lift	s when fully charged
Power supply		220-240	V AC / 2A / 50-60Hz
Battery		ut voltage – 25.2V DC; nput voltage – 5V DC;	
Charger		Qoltec – 5V DC / 2	.4A / 12W (Included)
Intermittence		Max. 10% = 2min	active / 18min pause
Stroke length			300 mm / 11.8 in
IP-rating			Control box – IP20
Usage temperature			+5 °C to +45 °C
Humidity		5-85	% (non-condensing)
Materials		Steel ar	nd misc. plastic parts
Surface treatment	Pow	der coating: st. CWS 8	31283 RAL 7021 mat

3.2 Product description

4Single battery is an accessory to an electrically adjustable 4Single Frame. By using the battery accessory, the 4Single Frame is freed from bothersome cables.

3.3 Intended use

The purpose of the product is to allow the 4Single Frame to be electrically adjusted to a seated or standing user. This way, they're able to use the table comfortably as a worktop. 4Single has a stroke length of 300mm / 11.8in.

By changing from fixed cabling to a battery, the 4Single achieves greater flexibility and becomes even more transportable.

3.4 Intended population

Anyone who can find use in having a tabletop or workstation with an adjustable height.

3.5 Intended operator

Anyone who has read and understood this manual can use the product. However, children over the age of 8 may only use the product under the supervision of an adult.

3.6 Basic safety

A 4Single Frame can be bought with a safety-strip, which must be mounted on the fascias. This way, collisions, and the danger of entrapment is reduced. It is always recommended to buy a safety-strip.

The safety-strip ensures that any collision with people or objects along the fascias are registered. When the safety-strip registers something, it stops the table, and retracts by 1-2cm / 0.4-0.8in.

4. Charging the battery

Battery status can be checked by pressing the status-button. The LEDs will turn on, and the greater the charge, the greater the number of LEDs will turn on.

Battery status

Batter status button is pressed. LEDs light up for 3 seconds.

100 - 76%	4 LEDs on
75 - 51%	3 LEDs on
50 - 26%	2 LEDs on
25 - 11%	1 LED on
10 - 0%	1 LED flashing



If the buzzer on the battery sounds twice for 0.5 seconds, when the status-button is pressed, it means that the battery has less than 25% charge. At this point, it is recommended to recharge the battery. The battery should be charged about once every 14 days, depending on usage, with heavy loads and frequent use typically requiring more frequent charging.

A charger is included with the battery as standard, with the following properties:

- Micro-USB type B
- > 5 V DC / 2.4A / 12W





Warning!

Charging during a thunderstorm, or other power surges, you risk serious injury or potentially death, due to electric shock.

Charging the battery with an incompatible charger can lead to serious injury or potentially death, due to the fire hazard of overheating.

4.1 Recharging procedure


- a) Unplug the battery from the docking bay.
- b) Plug the charger into the battery. Check to see if the LEDs are on. If the next LED starts to flash, the battery is recharging. If all the LEDs are on, but turn off after 3 seconds, the charger is incompatible.
- c) Wait 8-10 hours until all the 4 LEDs are on without flashing. The battery will sound the buzzer for 3 seconds when it is fully charged. If you have any doubts whether the battery is fully charged, and 4 LEDs are on, unplug the charger. The battery must not charge for more than 10 hours at a time.
- d) Unplug the charger.

4.2 Connection procedure

After the battery has been removed from the docking bay, it is important to follow the correct procedure for reconnecting the battery to the system, as otherwise, the system might not activate.


- a) Make sure that all the cables are connected, and that the safety-strip hasn't been activated.
- b) Plug the battery into the docking bay.
- c) Press the battery status button to activate the battery, and thereby the system.

5. Mounting instructions

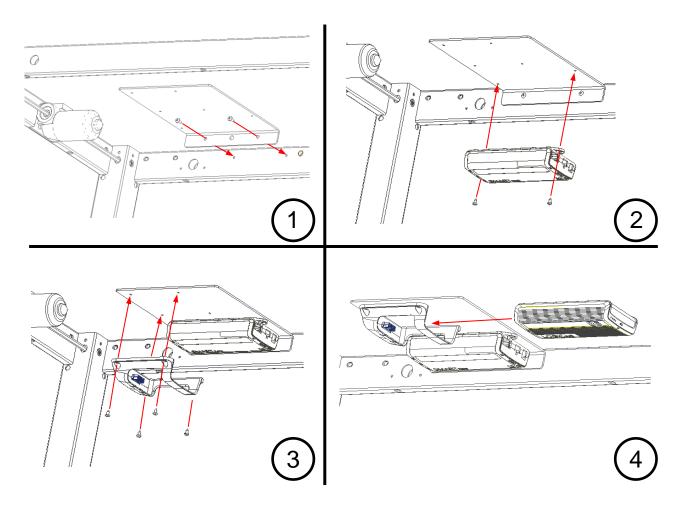

5.1 Electrical component diagram

The included cables must be mounted in accordance with the following diagram.

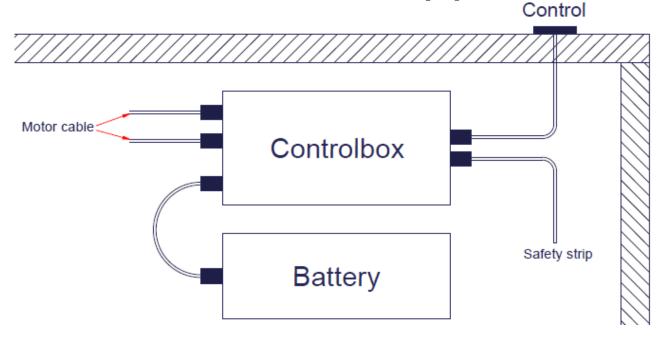



Notice!

Remember to activate the battery after mounting it in the docking bay, by pressing the battery status button.

5.2 Mounting the control

- a) Pull cable through hole in fascia and connect the control, via the control cable, to the control box
- b) Carefully remove the protection paper, on the back of the press pad, while keeping your finger between the cable-ribbon and the adhesive, as these mustn't stick to each other
- c) Place the press pad on the fascia





Notice!

If the cable-ribbon sticks to the adhesive, the control may be destroyed.

5.3 Testing

After mounting the battery on the 4Single Frame, and before putting it into use, the frame must be tested to ensure that it functions properly, and that the safety is upheld.

Afterwards, it is recommended to test the function of the 4Single Frame at least once a year, by qualified personnel.

5.3.1 Before testing the frame

- a) Ensure the battery has been mounted according to the mounting instructions
- b) Check that all bolt connections are fastened tightly
- c) Check that all the cables are mounted correctly, and the plugs are pushed in fully
- d) Remove any objects from on top of the tabletop, so that there is no load on the frame
- e) Make sure that the table can drive unhindered upwards and downwards, without the table hitting any objects

5.3.2 Testing the frame

- a) Press the DOWN button on the control, until the table is in its bottom position. Let go of the button and press DOWN again for 5 seconds. The legs should move below the bottom position. This resets the control box. Check if the movement is even and controlled.
- b) Press the UP button on the control, until the table is in its top position. Check if the movement is even and controlled.

5.3.3 Testing the safety-strip

a) If a safety-strip has been mounted under the fascias, press the DOWN button on the control, let the table drive 2-5cm / 0.8-2in, then press the safety-strip. The table should retract 1-2cm / 0.4-0.8in, then stop completely.

If all the items have been checked, and the table functions properly, it can be put into use.

6. Troubleshooting

	Check the battery status. If the battery status drops below 25%, and the table is driven, the buzzer will sound twice for 0.5 seconds. It is recommended to recharge the battery.
The frame cannot be	Check the battery status. If the battery status drops below 10%, and the table is driven, the buzzer will sound continuously for 0.5 seconds. It is recommended to recharge the battery.
height adjusted	Make sure the cables are connected correctly between the control box and the motors
	Make sure the cables are connected correctly between the control box and the control
	If a safety-strip is mounted, make sure it is not activated
	Check the battery status. If the battery status button is pressed, and
	none of the LEDs turn on, the battery must be fully charged. If the
	battery still doesn't work after a full charge, it must be replaced.
Battery doesn't work	If the battery shows more than 1 active LED, either flashing or static, during recharging, the battery is usable. Plug the battery into the docking bay and remember to activate the battery by pressing the battery status button.
	Remember to unplug the battery from the docking bay before recharging
	If the battery is plugged into an incompatible charger, the LEDs will turn on for 3 seconds, then turn off. Find an alternate charger, see 4.
Charging the battery doesn't work	If the buzzer sounds continuously for 2 seconds at a time, the battery is overheating. Unplug the charger and wait a couple of hours. Remember, the battery may only charge 10 hours at a time. If the buzzer keeps playing this sound, the battery or the charger may be faulty. Try another charger.
	The battery may be damaged. Plug the charger into the battery. If the buzzer sounds 5 times for 0.5 seconds, the battery needs to be fully charged, before it can be used again.
The frame is lopsided	The frame must be reset, see 8.1

7. Cleaning

The frame can be cleaned with lukewarm water and a suitable all-purpose cleaning agent. Use a cloth or soft sponge to wash the product. Dry the product off with a wrung cloth.

Electrical components, including the battery, may only be wiped down with a dry cloth and must always be disconnected from the mains power before cleaning of the frame can commence.



Notice!

Do not use cleaning agents containing abrasives e.g. scouring powder, steel wool, scouring sponge.

This product is not designed to be sterilized. Autoclaving and sterilization beyond normal cleaning may possibly change the product safety and function.

8. Maintenance

8.1 Periodic maintenance of the frame

It is recommended to test the function of the frame at least once a year, for operational and safety reasons. See 5.3.

Likewise, once a year, it's recommended to do a check-up of the frame by doing the following:

- > Check that all bolt connections are fastened tightly
- > Check that the table can move unhindered from bottom to top position. Check if the movement is even and controlled.
- Check that all the cables are mounted correctly and hasn't suffered any damage.
- ➤ Reset the frame Press the DOWN button on the control, until the table is in its bottom position. Let go of the button and press DOWN again for 5 seconds. The legs should move below the bottom position. This resets the control box.

8.2 Periodic maintenance of the battery

The battery isn't maintenance-free and will discharge itself over time. For this reason, it is recommended to recharge the battery regularly, approximately once every 14 days.

Likewise, the battery should be kept at above 25% charge, as the buzzer will go off otherwise during the driving of the table.

In addition, the battery should be maintained at least once every 4 months, doing the following:

- Clean/wipe off dust from the battery
- > Check the battery for cracks or other mechanical damage
- Discharge the battery fully, then recharge it to full charge. This extends the lifetime of the battery

Component part list



Notice!

Ropox will make part lists available for extraordinary maintenance and repair beyond what is described in this manual. Repair beyond what is described in this manual may only be done by Ropox designated service personnel.

Parts included



1x Control box SMARTneo-Pro 96000545





1x Battery 96000546





1x Docking bay 96000543







1x Mount for control box and battery 97701-850



6x Screw ø4,8x13 95091012




1x Control – 6x3cm 30-67840



2x Adapter cable 96000557

9.1 Accessories, component part list

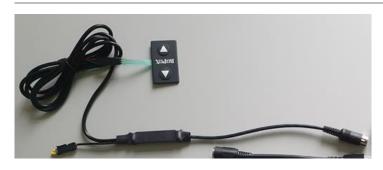

1x Connection – Safety-strip and Battery 50-47027

10. Environmental protection

The equipment must be disposed of in accordance with national legislation. It's recommended to dismantle the product as much as possible before disposal. Never attempt to dismantle the battery.

10.1 Packaging, transport, and storage

The battery must be packaged and transported according to the current regulations for lithium-ion batteries. These are also applicable when the product is re-packaged in case of a return delivery.

10.1.1 Packaging

The packaging must be labelled with a UN3480 lithium-ion battery label as shown. If packaged with other components, the LOGICcell must remain in its primary packaging. In this scenario, the battery label must be placed on the external packaging.



10.1.2 Transport

Regulations for transporting lithium-ion batteries are set for road transport (ADR), air transport (IATA) and sea transport (IMDG). The Battery Pack must be handled with care to minimize the risk of malfunction, damage, and injuries. IATA rules, as of April 2016, that lithium batteries may only be charged to a max. 30% if they are to be transported by air (passenger and freight aircraft).

10.1.3 Storage

It is important that batteries meant for long duration storage regularly are charged, with the time between charging varying between different storage conditions. Batteries that arrive by air must be charged prior to storage. In ideal storage conditions, the batteries must be recharged once every 4 months. Depending on temperature, the time between charging may be more frequent.

Temperature	Maximum time between charging
0 °C to +25 °C	4 months
+26 °C to +50 °C	3 months
+51 °C to +60 °C	< 1 month

