





Ropox StandardLine Washbasin

40-14770, 40-14771, 40-14772, 40-14773

User manual and Mounting instructions

This manual should always be in close proximity to the product.

TF 200.02.0026_eng / 17-05-2022

Table of Contents

1.	SYMBOLS USED IN THIS MANUAL	. 3
2.	GENERAL SAFETY	. 3
2.1	PRODUCT UNIT LABEL	. 5
3.	GENERAL REQUIREMENTS	. 6
3.2 3.3 3.4	PRODUCT INFORMATION INTENDED USE INTENDED OPERATOR ESSENTIAL PERFORMANCE	. 6 . 6 . 7
	NON-CLINICAL FUNCTIONS	
3.7	PRODUCT DIMENSIONS	. 7
4.	MOUNTING INSTRUCTION	10
4.1 4.2 4.3 4.4 4.5	LIST OF COMPONENTS	12 13 16
5.	FUNCTIONALITY TESTING	25
6.	SAFETY IN USE	25
7.	CLEANING	26
8.	TROUBLESHOOTING	26
9.	MAINTENANCE	27
9.1	Service schedule	28
10.	SPARE PARTS	29
11.	ENVIRONMENTAL PROTECTION	29
12.	ELECTROMAGNETIC COMPATIBILITY	30
12. 12. 12. 12.	2 ADJACENT AND STACKED USE	30 30
13.	COMPLAINTS	30



It is important to read this manual prior to mounting and use

1. Symbols used in this manual

Warning Symbol

Indication of potentially hazardous situation. If not avoided, it can result in serious injury or death.


Caution Symbol

Indication of potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices.

Notification Symbol


This symbol is used to notify correct use and handling of the product.

2. General safety

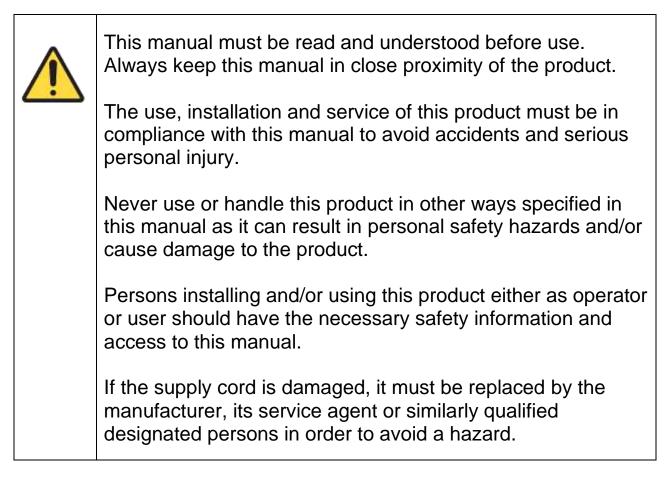

Ŵ	Do not perform repairs, disassembly or assembly operations, add-ons, re-adjustments or modification of the product beyond what is described in this manual. These must be carried out by Ropox or Ropox authorized personnel. Do not perform service while in use.
	This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.
	Do not use the product if it has defects or have become damaged before being repaired or replaced. If the control box makes unusual noise or smells, switch off the mains voltage immediately.
	Take care that the cables are not damaged.
	Unplug the mains cable on mobile equipment before it is moved.
	The products must only be used in an environment that corresponds to their IP protection.
	The information in this manual is based on correct installation in accordance with installation instructions for this product.
$\mathbf{\dot{\cdot}}$	Ropox cannot be held liable if the product is used in any way that differs from stated in this manual and/or installation instruction.
	Ropox reserve the right to amend this manual and reference documents without prior notice.

2.1 Product Unit label

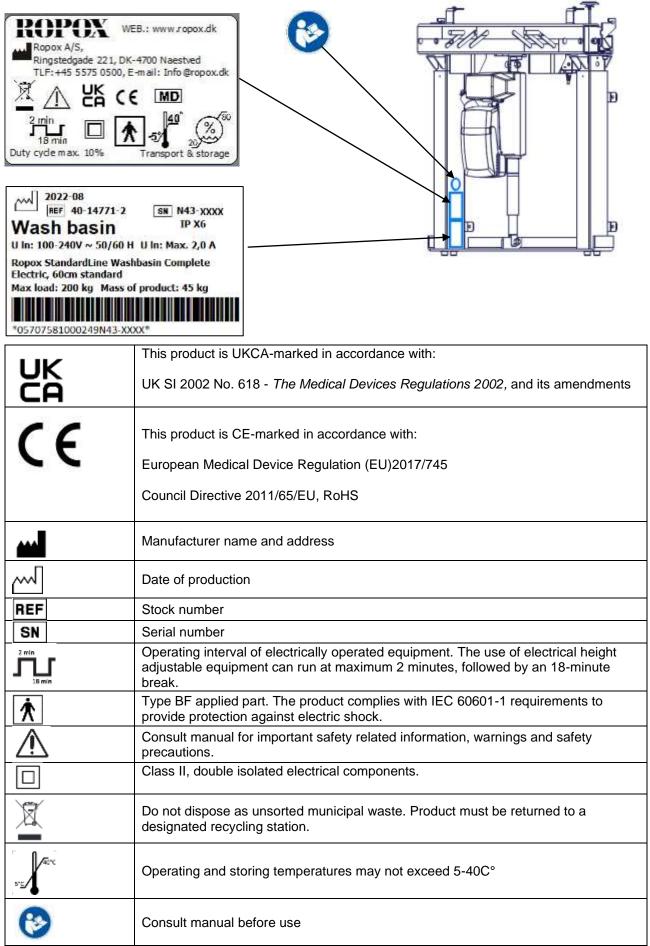

3. General requirements

3.1 Product information

Manufacturer	Ropox A/S Ringstedgade	221			Qserve Group UK, Ltd 49 Greek Street,	
	DK-4700 Naes				W1D 4EG London	
	+45 55 75 05 (United Kingdom	
	Info@ropox.co				+44 740 885 936 3	
	1110@1000.00	,,,,,			info@qservegroup.com	
Product models	Part number		Model	Configu		
i i oddol modolo	i art nambol		model	Comga		
	40-14770		StandardLine	Unit with	out washbasin	
	40-14771		StandardLine	Standar	d washbasin	
	40-14772		StandardLine		washbasin	
	40-14773		StandardLine	Hospital	washbasin	
Expected service life	5 years					
MDR Class 2017/745	Class I					
Applied part	Type BF appli	ed part				
classification		•				
60601-1						
MEE Class	Class II					
60601-1		-				
Height adjustment	40-14770 30cm range					
	40-14771 70 – 100cm					
	40-14772					
	40-14773	70 – 7	70 – 100cm			
Maximum user weight	200kg / 440lb					
according to DS/EN 17966:2016						
Power supply	230V ~50Hz					
l in	Max 2.5A					
Intermittence	2min use / 18n	nin pau	ISE			
IP rating	IPX6					
Ambient temperature			nsport and storage			
range	+5 °C to +40 °					
Relative humidity	20% to 80% -					
Intended environments	This product is	to be	used only in:			
	Professional H	lealthca	are Environment			
	Homecare Env					
	The device is not intended for use in special environments as defined by IEC					
	60601-1-2		·		-	
Materials in contact	Washbasin		Synthetic/mark	ole compo	site	
with patient	Frame		Welded steel tube St. 37, aluminum			
	Mixer tap		Chrome			
Accessories	40-14777		Hand control			
	40-14776	Plumbing kit				

3.2 Intended use

The StandardLine washbasin is especially suitable for private homes and institutions. The height is adjusted electrically. There are three types of washbasins to choose from so you can be sure to get the model that fit the users best. These include a standard wash basin, one that provides support through integrated handles, or one that's larger in size, intended for hospital use.

3.3 Intended operator

Reading and understanding this manual will be sufficient training for an intended operator. Children under the age of 16 or patient are not considered intended operator.

3.4 Essential performance

The product does not have essential performance, as the patient and user do not experience any risk in case of loss of product function/performance.

3.5 Non-clinical functions

Height adjusting offers better ergonomics to the user.


3.6 Clinical functions

The product does not have any clinical functions.


3.7 Product dimensions


40-14770

(30cm range)

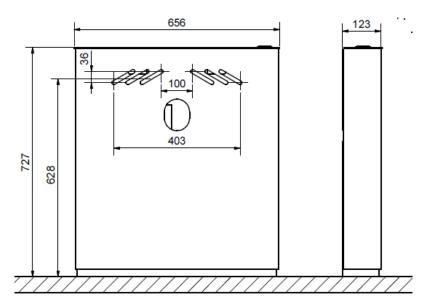


40-14771 (Height adjustment 700-1000mm)

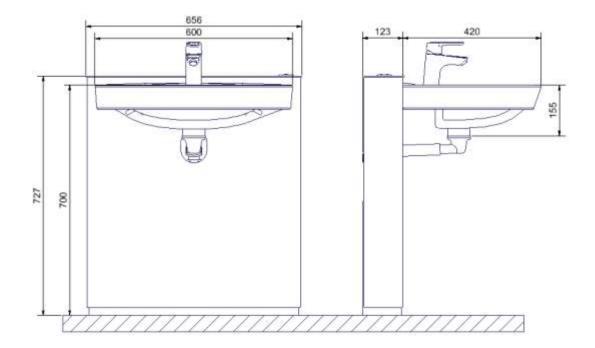

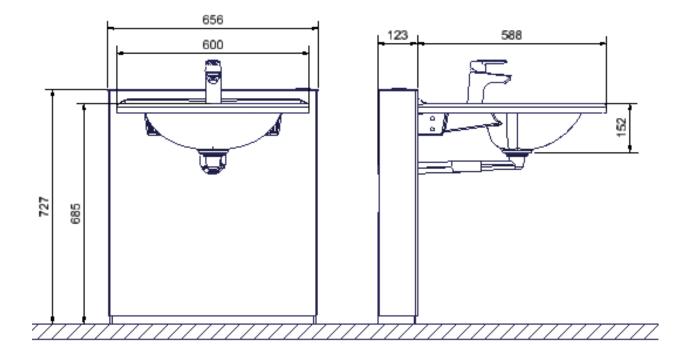

40-14773 (Height adjustment 700-1000mm)

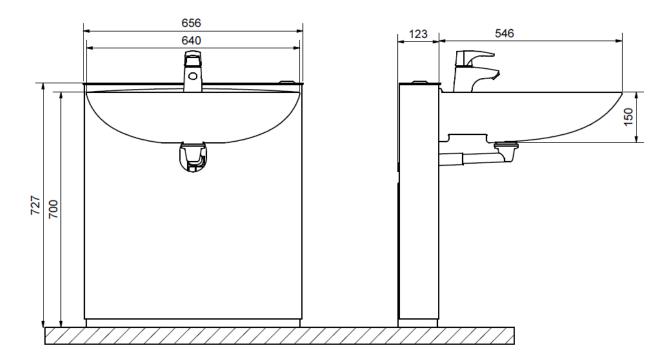

NB! All dimensions are in mm

3.8 Operation

Standard control

The washbasin height is adjusted by means of the fixed Ropox control button mounted in the top of the lifting unit.

The washbasin is raised/lowered by pressing the up/down symbol on the control.

The washbasin stops moving as soon as the button is released.



Ropox hand control with cable (accessories)

As an alternative, you can use a hand control with cable. The washbasin is operated in the same way. As soon as the button is released, the washbasin will stop moving.





Warning!

The product should always be able to travel the full range of actuation without colliding with objects. Failing to do so will compromise the product stability and introduce risk of squeezing body parts.

Special care should be taken in ensuring that no children or adults are sited under the product as severe injury from entrapment may occur.

4. Mounting instruction

4.1 Installation interfaces

Construction

The wall must be made of a material suited for screw mounting and approved for damp rooms. The hatched area is the minimum area for reinforcement of the wall.

Tested in accordance DS/ISO 17966:2016 with the following max. user weights: 40-14772: 200kg (440 lbs) 40-14773: 200kg (440 lbs) 40-14770: 200kg (440 lbs) (with porcelain wash basin)

Max. tensile load of upper screws

50kg/screw (110 lbs)

The washbasin is supplied with a set of screws and rawlplugs suitable for mounting on a concrete wall. The fitter should always consider the material, condition and strength of the wall and use screws and rawlplugs suitable for the specific wall type.

Plumbing

The drain is $Ø32 (1 \frac{1}{4})$ and ends in an $Ø32x88.5^{\circ}$ drain elbow.

The water supply ends in 2 (two) 90° shut-off valves with $\frac{1}{2}$ " outside thread pointing upwards (not included). The maximum protrusion of the shut-off valves and drain connection should be max 90mm (3 $\frac{1}{2}$ ") to allow for the covers.


The optimum location and orientation is as shown in green.


It is important that all hoses for water connection and drain are flexible to allow the frame to move freely within the height-adjustment range of 700-1000mm (27 1/2" - 39 2/5") (including washbasin).

For purchase of adapters for connection to various standard installations, see "Connection Adapters – Other Washbasins".

Power


Recommended location of socket in wall for optimum and hidden wiring is shown in red. If a different location is required, the cable from the control unit is 3200mm (126").

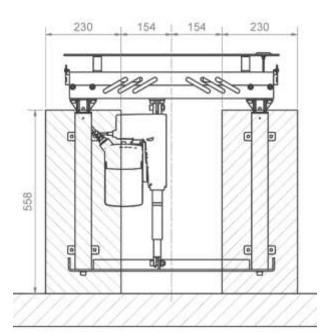

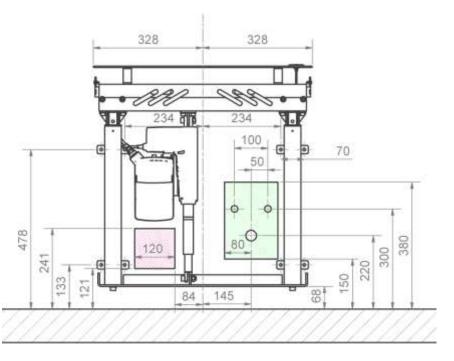


Water and drain placed in floor

If it is not possible to have the water supply and drain be placed in the wall, it is recommended to place it within the hatched green area in the floor. The drain is Ø32.

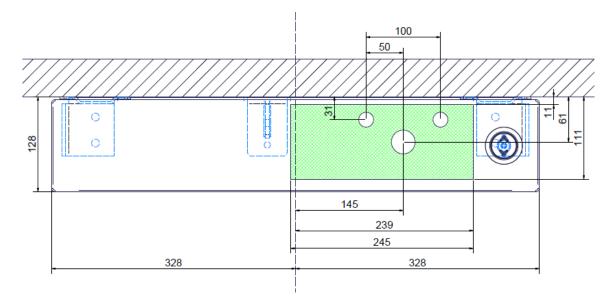

Water and drain placed outside the lifting unit

If the water supply and the drain is placed outs-ide the lifting unit, it is recommended to make a cut-out in the inner and outer covers, to allow water and drain to be brought inside the frame. If the height is adjusted correctly in relation to the adjustment screws, see page 17, the height of the cut-out in the outer cover mustn't be greater than 27mm. The total height of the cut-out should not exceed 50mm. The piping outside the lifting unit must be fixed, and should be brought inside the frame, as shown. Now, connect the flexible pipes and hoses to the fixed piping. The fixed piping can be brought inside the frame from both sides, but it is recommended to complete the installation in the right side.

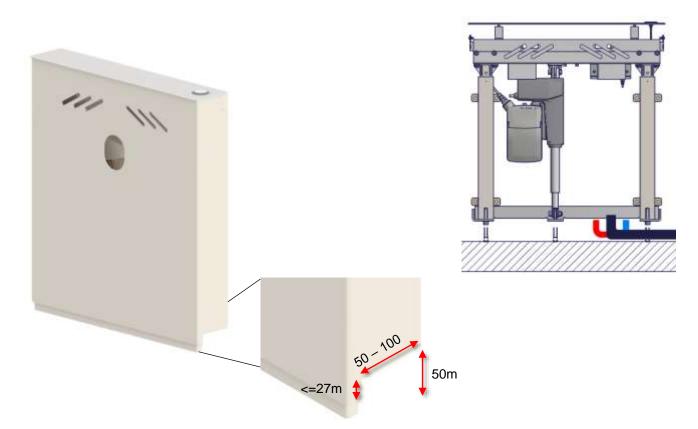

4.2 Connection adapters

The components illustrated below are available depending on the type of installation in the building.

Water connection adapters

Water connection adapters					
Hoses from wa	ashbasin	Adapter	Installation in room (not included)		
	No adapter needed	1/2" Thread			
1/2'	1/2" -> 3/8" Item no. 97001670	3/8" Thread			
		1/2" -> 15mm Item no. 97001666	15mm Pipe		
Nater hose extensions		F utur	h		
	Hose from washbasin Coupling piece		Extra hose 1/2" x 1/2", 0.5m, steel Item no. 97001123		
3	1/2" x 1/2", 1m, plastic Item no. 97001120				
Drain adapters					
Drain from wa	shbasin	Adapter	Installation in room (not included)		
	No adapter needed	Ø32			
Ø32	(22 × (40	Ø40			

Ø32 Ø32 Ø32 Ø32-> Ø40 Item no. 97001660 Ø32 -> Ø50 Ø32 -> Ø50 Item no. 97001661 Ø50 Ø50

Ø32 Ø32 Ø32 Ø32-Ø32 Item no. 97001662 Ø32 Flex hose, 32-75cm Item no. 97001172 Ø32 Flex hose, 40-110cm Item no. 97001161

4.3 List of components

StandardLine lifting unit with washbasin

40*14770-03	6	Actuator bracket	1x	6
30-67636	<i>Incl.</i> 30*65550-013 96000167	Flexi legs without spindle Adjustment screw Ferrule	2x	
40*14770-01	7	Alu profile 622mm	1x	
40*14770-02	8	Bracket for washbasin	1x	for the
40*14770-02	6	Alu top plate	1x	
40*14770-02 [,]	4	Outside cover	1x	
40*14770-02	5	Inside cover	1x	

97000488	Actuator	1x	and a
97001691	CA30 controlbox	1x	
97000427	Network cable for controlbox	1x	and a second
97001713	Bracket for controlbox	1x	
97000476	Retaining ring	1x	C
40*14770-044	Mounting parts for control	1x	
Incl.			
95091437	Taptite screw M3x10	2x	
97001800	Control	1x	
40*43043-133 96000165	Control cover Strips black 3,5/140mm	1x 2x	
40*14770-041	Mounting parts electrical	<u></u> 1x	
Incl.		17	
95011035	INSEX screw M10x35	2x	
95180510	Lock nut M10	2x	
96000170	Strips black 4,8/290mm	2x	
40*14770-043 Incl.	Mounting parts frame	1x	
40*14770-022	Top bracket	1x	
30*65500-003	Flat steel nut actuator mounting	2x	
30*65550-005	Flat steel nut leg section	2x	
95010814	INSEX screw M8x14	6x	
95170508	Chamfered washer M8	6x	
95011025	INSEX screw M10x25	4x	
95170510	Chamfered washer M10	4x	
96000810 40*14708-099	Sleeve for hole Ø9	2x 1x	
40~14708-099 Incl.	Mounting parts washbasin	IX	
95041250	Bolt M12x70	2x	
95170512	Chamfered washer M12	4x	
96000170	Strips black 4,8/290mm	2x	
97001467	Mounting parts for bolt mounted wash basin	1x	
30*65550-013	Adjustment screw	1x	
96000167	Ferrule	1x	
97001706	Oras Saga 3910F	1x	K

40-14776 <i>Incl.</i>	Plumbing kit for StandardLine	1x	
97001601	Prevex siphon	1x	0
97001120	Connection hose - ½"x ½" 1m	2x	
97001161	Flex drain Ø32/32 pipe	1x	10
97001035	32x88,5° white PP Wafix elbow HT	2x	
97001012	Nipple ¹ / ₂ " to 3/8"	2x	
96000170	Strips black 4,8/290mm	2x	
97001605	Prevex 100mm extension	1x	
95091070	Plate screw Ø4,8x70 cross recess A2		l l l l l l l l l l l l l l l l l l l
30*65500-00	08 Plate nut M6 INSEX screw M6x12	2x 2x	
95010612 40*14852-10		2x 1x	~
40 14832-10	J4 Blacket for siphon	IX	
40-15621 (For 40-14771)	Standard Washbasin	1x	
<i>Incl.</i> Insolation foam pad		1x	
97001582 (For 40-14772)	60cm Straight Support Washbasin	1x	
40-15622 (For 40-14773) Incl. Insolation foam pad	Hospital Washbasin	1x 1x	
ioam pad			

4.4 Installation of product

Prior to assembly check that all parts have been provided according to the list of components. See list of components.



Caution!

Mounting must always be carried out by qualified personnel




Warning!

Before use always make sure that mains cable and hand control cable will be able to travel freely without the risk of squeezing or disconnecting the product as result of the movement.



Caution!


When connected to the mains the product must be placed in a manner according to power outlets so that the operator will be able to disconnect the product during use in case of an emergency.

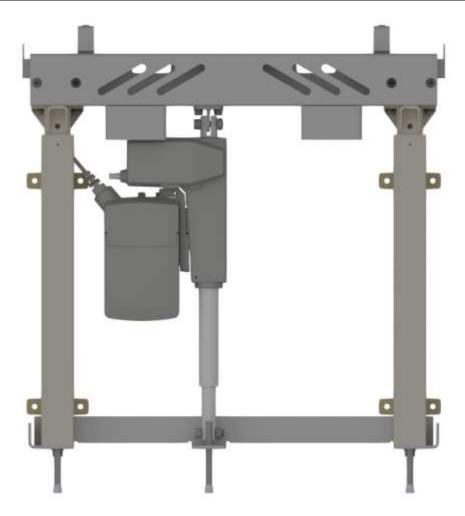


StandardLine comes partly pre-assembled.




Unscrew the adjustment screws 68mm below the bracket.

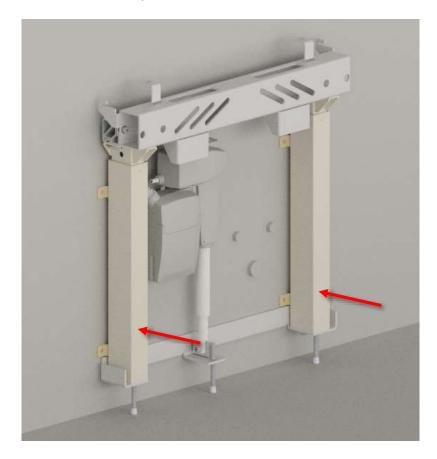


Place the lifting unit on the wall and mark holes for screws. Drill holes and use the enclosed **rawlplugs**.

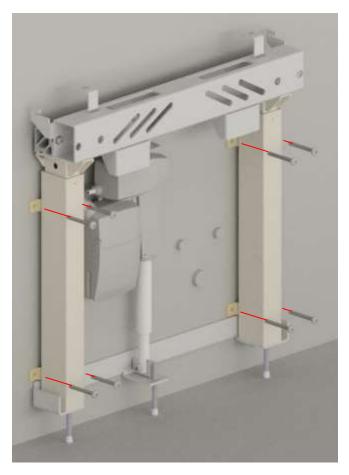


Fix the unit to the wall.

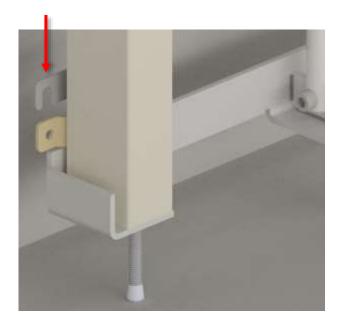


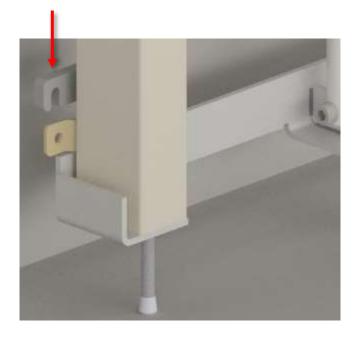


If the wall is uneven, or the unit needs be moved from the wall due to skirting boards, use the spacers.

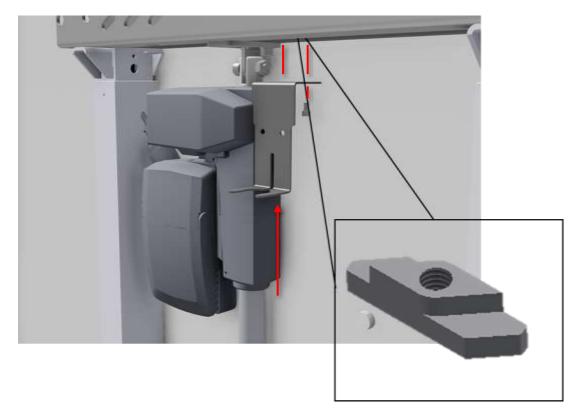


Place 2x M6 nuts into the profile and fix the bracket for the siphon with 2x M6x12.

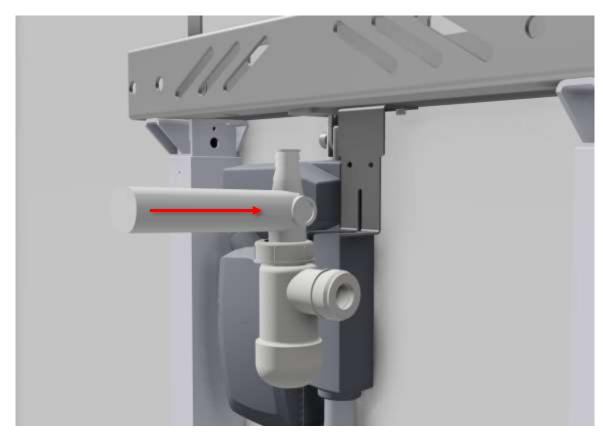


Push the siphon into the bracket and fasten the connecting pipe with strips through the two holes.

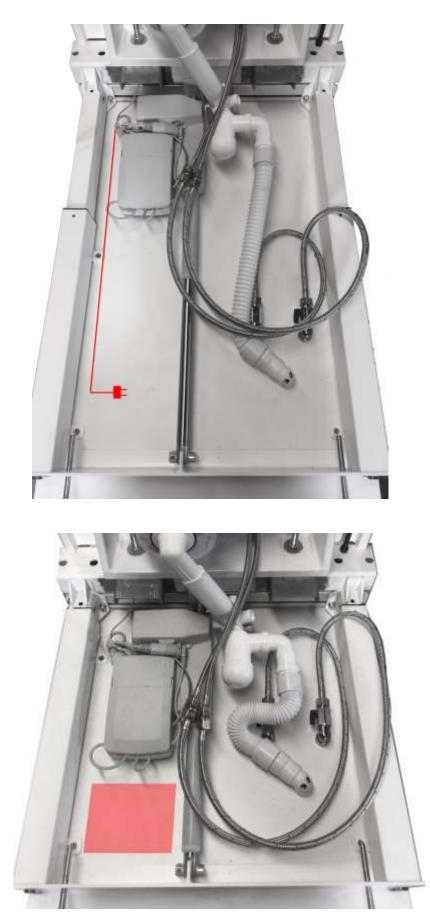

Assemble drain hoses, water, and power as shown on the picture. Alternatively, the water and drain are mounted in the floor.

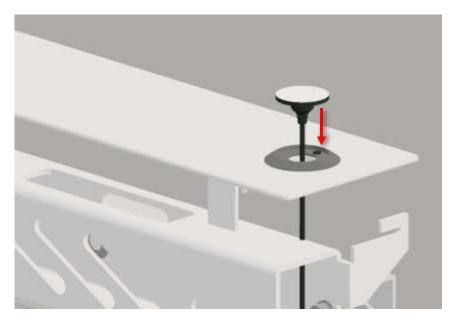


Remove the protective foil, and glue in the control on the top plate.

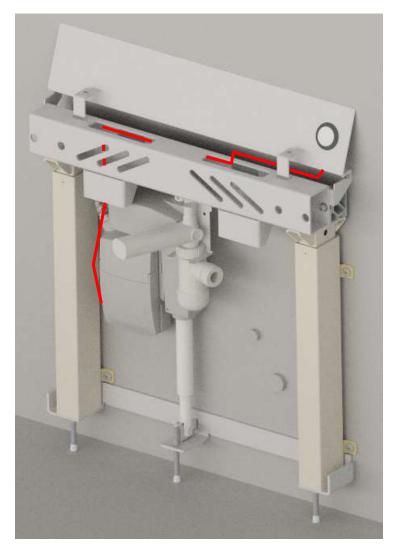


Plug the cable into the control box running through the frame as shown on the picture.

Perform a functionality test of the electrical system

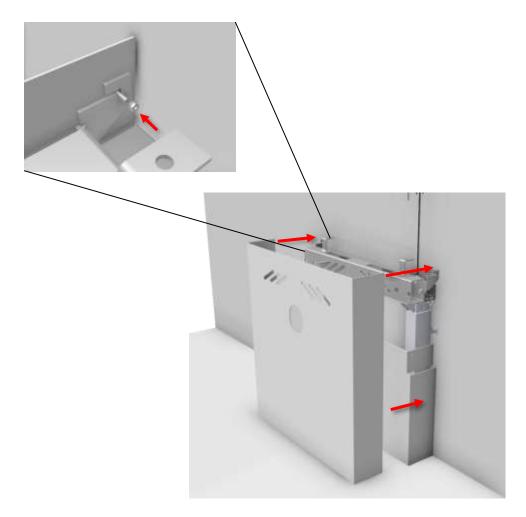

Push the inner cover in, so that it stands on the floor. Push the outer cover in place so that it hangs onto the two rods. Fix the outer cover onto the two rods with **M4 lock nuts** and **washers**.

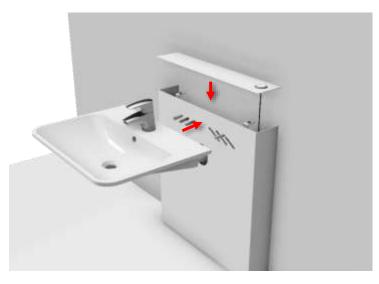


Fix the wash basin to the unit with the accompanying bolts and fix the top plate to the unit.




Caution!

When mounting a porcelain washbasin, an insolation foam pad **MUST** be placed between the washbasin and the unit. It is recommended to grout around the washbasin.

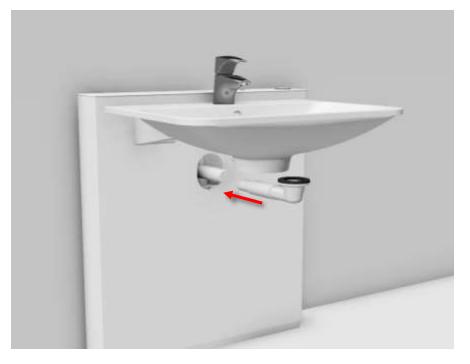


Mount the rest of the drain and connect the water to the tap.

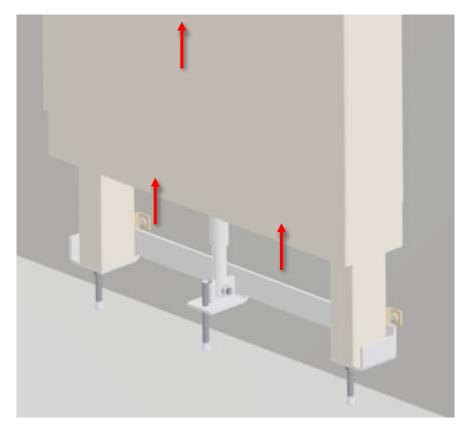

Move the unit to upper position and lift the inside cap to turn on the water. Now pull the inside cap back down to the floor to finish the installation.

4.5 Cable diagram

The round control pad is a standard accessory that comes with the StandardLine. Upon request, it's possible to get the StandardLine with a handcontrol with a flexible cord, as well. This handcontrol allows easier control of the unit, especially from range.

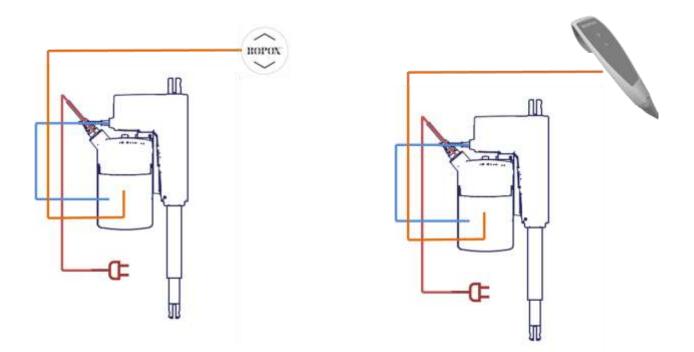

Cable diagram with both controls



5. Functionality testing

After completing the installation and before use, all functions of the StandardLine washbasin must be assessed. Subsequently, a functionality test must be conducted at least once a year by qualified personnel:

- 1. Check that the mounting instructions have been observed.
- 2. Check that all screws and bolts are fixed.
- 3. Check that all cables have been correctly connected, and that the plugs are pushed in properly.
- 4. Check that there is no load on the frame.
- 5. Check that there are no objects preventing the frame from moving freely within the height adjustment range.
- 6. Press both switches on the control and hold the pressure as long as a loud beeping sound is heard. When this sound stops, the frame can be moved to bottom position and is thus reset.
- 7. Press the DOWN button, move the frame to bottom position and check that the movement is even and smooth. Make sure that the hose connections follow the movements of the frame, and that they do not get squeezed.
- 8. Now press the UP button, move the frame to top position and check that the movement is even and smooth. Make sure that the hose connections follow the movements of the frame, and that they do not get squeezed.

If the above points are checked, the washbasin is ready to use. See the section "Safety in use".

6. Safety in use

- The StandardLine washbasin may only be used by persons who have read and understood these instructions.
- The StandardLine washbasin is a height adjustable washbasin and may not be used as a lifting table or as a person lifter.
- Always use the washbasin in a manner excluding the risk of damage to people or property. The person operating the StandardLine washbasin is responsible for avoiding damage or injury.
- If the washbasin is used in publicly accessible locations, where especially children or other persons with reduced observation capability may get near the StandardLine washbasin, the person operating the washbasin must pay sufficient attention in order to prevent dangerous situations.
- > Make sure that there is free space above and below the washbasin to allow free height adjustment.
- Be aware of water splashing from the washbasin during use, or actuation of the height adjustment. A wet floor will result in a slipping hazard.
- > Always check the water temperature before use. Too warm water may lead to scalding.
- If a handcontrol with flexible cord has been installed, be aware of the cord not getting caught on anything. This could lead to a tripping hazard.
- > Do not overload the washbasin and make sure that the load distribution is correct.
- > Do not operate the washbasin in case of errors or damage.
- > Do not use the washbasin in an explosive environment.
- Any modification to the StandardLine washbasin, which may influence its operation and construction, is forbidden.
- > Installation, service, and repairs may only be conducted by qualified personnel.
- Prior to use, ensure that the StandardLine washbasin has been mounted on a suitably strong wall and that there are no loose connections.
- > In case of inspections, service, or repairs, make sure that the washbasin is not loaded.
- If the StandardLine washbasin has not been installed in accordance with these mounting instructions, the warranty may be void.
- Only use original Ropox spare parts as replacements parts. If other parts are used, the warranty may be void.

7. Cleaning

During use, the product may get dirty. It is important that the washbasin and the lifting unit are cleaned in accordance with these instructions.

Ropox StandardLine may be wiped with a wrung cloth with lukewarm water and a mild detergent. Do not use corrosive/grinding fluids or scouring sponges and brushes.

Cleaning of washbasin

Wash the surfaces daily with water and wipe with a dry cloth. Do not use cleaning agents containing hard abrasives – such as steel wool, scouring powder, or sulfuric acid.

If the water has a high calcium content, calcareous deposits will often occur in the washbasin. These deposits should be removed at least once a week by using a detergent containing citric acid, acetic acid, or phosphoric acid. Leave the detergent to work for 5-10 minutes depending on the thickness of the deposits and clean thoroughly with a soft sponge.

Disinfection of the washbasin may be done with household disinfectants.



Notification!

Do not use cleaning agents containing abrasive e.g. scouring powder, steel wool, scouring sponge.

This product is not designed to be sterilized, autoclaving and sterilization beyond normal cleaning may change the product safety and function.

8. Troubleshooting

Problem	Solution	
The washbasin does not move up or down when pressing the hand control	 Check if: 230 V is connected, and if the power is on. The engine is connected to the control box. The control is connected to the control box. The actuator is overloaded. (NOTE: For every 2 minutes of activity, it needs 18 minutes of rest). 	
Hand control is defect	 Check if: The washbasin is height adjustable by means of another control. 	
When pressing the UP and DOWN buttons, only "click" sounds come out of the control box – no movement	 The plug may not be mounted correctly. Demount all cables from the control box, and mount again. A "Reset" might be needed. See the instructions below. 	
The frame seems loose or unstable	Tighten all screws of the frame, cf. mounting instructions.	
The washbasin cannot be height adjusted or the adjustment feels very difficult	Check that there are no jammed objects preventing the height adjustment.	
It takes too long for the water to leave the washbasin	 Check the siphon for clogging and clean it, if required. Check that the flexible hose has not been mounted so high on the wall that it acts as an additional siphon. Check that the connection hoses are not kinked or twisted on the hose. Adjust the hoses so that they are not kinked/twisted. 	

9. Maintenance

The frame is maintenance-free. Moving parts have been lubricated for life. For reasons of safety and reliability, we recommend inspection of the frame once a year:

At inspections, the following must be checked:

- > Check that all bolts and screws have been tightened securely.
- > Check that the frame can move freely and unhindered from bottom to top position without problems.
- Resetting the StandardLine washbasin: Press and hold down both buttons on the control. Keep the buttons depressed until the beep tone stops beeping. Then move the frame to bottom position for resetting. Check that the movement is even and smooth. Make sure that the network cable is able to move freely.
- > Check that any water connection hoses and drain hoses are tight and functional.
- > Check that all cables are mounted correctly and are in order and undamaged.

After each the inspection, the service schedule must be completed. See "Service schedule".



Failing to comply with periodic maintenance may result of degrading of product function and safety.

	-	
()
	÷	

Note!

Ropox will make parts list and circuit diagrams available for maintenance and repair beyond what is described in this manual. Repair beyond what is described in this manual may only be done by Ropox designated service personnel. Only use original Ropox spare parts as replacement parts. If other parts are used, the warranty may be void.

9.1 Service schedule

Service and maintenance	Serial no.	Service and maintenance	Serial no.
Date:		Date:	
Signature:		Signature:	
Remarks:		Remarks:	
Service and maintenance	Serial no.	Service and maintenance	Serial no.

Date:

Date:

Signature:

Remarks:

Signature:		
Remarks:		

Service and maintenance	Serial no.
Date:	
Signature:	
Remarks:	

Service and maintenance Serial no.
Date:
Signature:
Remarks:

Service and maintenance	Serial no.
-------------------------	------------

Date:

Signature:

Remarks:


Service and maintenance Serial no.

Date:

Signature:

Remarks:

10. Spare parts

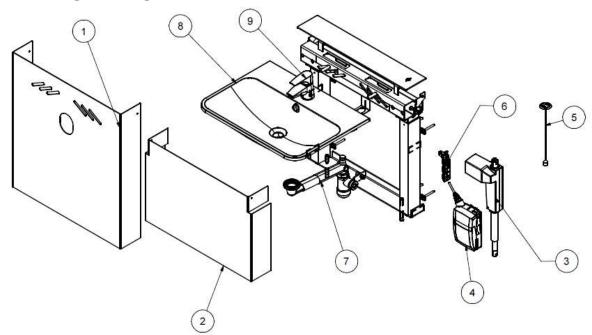

Position	Description	Number in product	Item no.
1	Outer cover	1	40*14770-024
2	Inner cover	1	40*14770-025
3	Motor LA31.6	1	97000488
4	Control box CA30	1	97001691
5	Control	1	97001700
6	Bracket for LA31/CA30	1	97001713
7	Plumbing kit for StandardLine unit	1	40-14776-1
8	Standard washbasin 60x42cm/	1	40-15621-2
	60cm Straight Support/	1	97001582
	Hospital washbasin	1	40-15622-1
9	Oras Saga 3910F	1	97001706

11. Environmental protection

The product is not intended to be disposed as municipal waste. Proper disassembly, sorting and disposal of components must be done by waste disposal professionals.



Part	Disposal recommendation
Wash basin	Ceramic
Main unit	Steel
Mixer tap	Steel
Actuator	Electronic
Control box	Electronic

12. Electromagnetic compatibility

12.1 Suitable Environments

The device is suitable for use at home, at daycare centers, at day centers for persons with physical or mental disabilities or at hospitals except near active HF surgical equipment and the RF shielded room of a medical electrical system for magnetic resonance imaging, where the intensity of electromagnetic disturbances is high.

12.2 Adjacent and stacked use



Warning!

Use of this equipment adjacent to or stacked with other equipment should be voided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally

12.3 Cables

Cable component	Length
Power cable	3,2m
Hand control spiral cable	Approx. 5m extended.



Warning!

Use of accessories, transducers, and cables other than those specified or provided by the manufacturer of this could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

12.4 RF portable equipment



Warning!

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

13. Complaints

Please see general terms of sale and delivery on www.ropox.com

