



CE



All-in-One Walking Trainer

User manual

This manual should always be in close proximity of the product

TF 200.0016 / Rev 03 / Date 03-07-2023

Table of content

1.	Symbols	used in this manual	. 4
2.	General	safety	. 4
	2.1	Product Unit label	. 6
3.	General	requirements	. 7
	3.1 F	Product information	. 7
	3.2 F	Product description	. 7
	3.3 I	ntended purpose	. 7
	3.4 I	ntended population	. 8
	3.5 I	ntended operator	. 8
	3.6 E	Essential performance	. 8
	3.7 E	Basic safety	. 8
	3.8 N	Nonclinical functions	. 8
		Product dimensions	
4.	Instructi	ons for use	10
	4.1	Installation of product	
	4.2	Installation of lifting unit and hand support	
	4.3	Operating the product	
		3.1 Operating instructions	
		3.2 Emergency situation	
	4.3	3.3 Stability function	
	4.4	Recharging the battery	
	4.5	Safety function, lifting motor	
	4.6	Safety function, control unit	
	4.7	Emergency lowering	
	4.8	Brakes	
	4.9	Size guide for Domino slings	
		9.1 Size information	
		9.2 Dimensions for slings	
	4.10	Transfer – walking trainer	
		10.1 Lifting from chair/wheelchair	
		10.2 Using the walking trainer	
		10.3 Placing in chair/wheelchair	
	4.11	Height adjustment of hand support	
	4.12	Electrical component diagram	
5.		hooting	
6.	-		
7.		ance	
	7.1	Periodic maintenance	
	7.2	Daily check	
	7.3	Monthly maintenance	
	7.4	Yearly inspection	
	7.5	Maintenance report mechanical parts	
	7.6	Maintenance report actuator/lifting motor	25

	7.7 Life	time of actuator/lifting motor	26
	7.7.1	Table to determine an actuators lifetime	26
8.	Component p	oart list	27
	8.1 Spare	e parts	27
9.	Environment	al protection	28
10.	Electromagne	etic compatibility	28
	10.1 Suita	able Environments	28
	10.2 Cabl	les	28
	10.3 RF p	portable equipment	28
11.	Complaints		29

1. Symbols used in this manual

Warning Symbol

Indication of potentially hazardous situation. If not avoided, it can result in serious injury or death.

Caution Symbol

Indication of potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices.

Notification Symbol

This symbol is used to notify correct use and handling of the product.


2. General safety

 This manual must be read and understood before use. Always keep this manual in close proximity of the product.
 The use, installation and service of this product must be in compliance with this manual to avoid accidents and serious personal injury.
 Never use or handle this product in other ways specified in this manual as it can result in personal safety hazards and/or cause damage to the product.
 Persons installing and/or using this product either as operator or user should have the necessary safety information and access to this manual.
 If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified designated persons in order to avoid a hazard.

Do not perform repairs, disassembly or assembly operations, add-ons, re-adjustments or modification of the product beyond what is described in this manual. These must be carried out by Ropox or Ropox authorized personnel. Do not perform service while in use. This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.
Take care that the cables are not damaged. Unplug the mains cable on mobile equipment before it is moved. The products must only be used in an environment that corresponds to their IP protection.
 The information in this manual is based on correct installation in accordance with installation instructions for this product. Ropox cannot be held liable if the product is used in any way that differs from stated in this manual and/or installation instruction. Ropox reserve the right to amend this manual and reference documents without prior notice.

2.1 Product Unit label



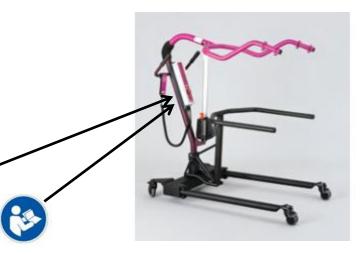

	This product is CE-marked in accordance with:
	European Medical Device Regulation (EU)2017/745
	Council Directive 2011/65/EU, RoHS
	Manufacturer name and address
	Date of production
REF	Stock number
SN	Serial number
ī	Consult manual before use
^{2 min} 18 min	Operating interval of electrically operated equipment. The use of electrical height adjustable equipment can run at maximum 2 minutes, followed by an 18-minute break.
*	Type B applied part. The product complies with IEC 60601-1 requirements to provide protection against electric shock.
\triangle	Consult manual for important safety related information, warnings and safety precautions.
	Class II, double isolated electrical components.
X	Do not dispose as unsorted municipal waste. Product must be returned to a designated recycling station.
30cm	Mobile phones or other portable RF emitting equipment should be no closer to the product than 30cm.
5°C	Operating and storing temperatures may not exceed 5-40C°
8	Consult manual before use

3. General requirements

3.1 Product information

Manufacturer:	Ropox A/S Ringstedgade 22 +45 55 75 05 00 Info@ropox.com	1, DK-4700 Naestveo	1	
Product models	Part number	Model	Configuration	UDI
	25-20035-8 25-20526-8	Walking trainer 150 Walking trainer 200	kg Convertible	57075810810S6 57075810811S8
MDR Class 2017/745	Class I			
MDD Class 93/68/EEC				
Applied part classification 60601-1	Type B applied p	art		
MEE Class 60601-1	Class II			
Intended				
environments	Professional Hea Homecare Envirc	Ithcare Environment		
	The device is not 60601-1-2	intended for use in s	pecial environments a	s defined by IEC
Maximum user weight according to DS/EN 10535:2007	150 / 200kg			
Power supply	100-240V ~50/60)Hz		
l in	Max 2.5A			
Intermittence	2min use / 18min	pause		
Height adjustment	150-190cm			
Speed of actuation	≈ 15mm/s			
Number of lifts per	150kg = Approx.			
charge (up and down)	200kg = Approx.	75		
IP rating	IPX4			
Ambient temperature		ransport and storage		
range	+5 °C to +40 °C (
Relative humidity:	20% to 80% - non-condensing			
Materials in contact with patient	Sling		olyester	
	Spreader bar	S	t37 tube construction v	vith powder coating

Accessories	
Item Number	Model/specification
25-20135-8	Walking Trainer kit

3.2 Product description

The All-in-One walking trainer system is a mobile personal lifter for people with limited mobility who need help with walking training. The medical staff, using different slings, can lift people up to 150/200 kg, depending on the chosen variant of lift.

3.3 Intended purpose

The product is used in situations where a patient must be lifted from a sitting position and up to a standing position. The product must not be used for patient transport. The product must always be used with Ropox suitable lifting slings.

3.4 Intended population

Patients with mobility difficulties.

3.5 Intended operator

Intended operator must have relevant health professional training before using the product, as well as have read and understood this guide. Children, patient, or lay person is not considered a suitable operator.

3.6 Essential performance

The lifting motor has a mechanical emergency lowering facility. In case of failure of electronics during use, this can be used to lower the patient to a safe height.

3.7 Basic safety


The battery must always be checked for capacity before starting work. An audible warning will sound if the battery capacity is too low. The product is equipped with an emergency stop, its function must also be checked before use.


3.8 Nonclinical functions


The product has no clinical effect. This is a patient lifter.

3.9 Product dimensions

	25-20035	25-20526
	150kg	200kg
Lifting capacity	150 kg	200 kg
Min. lifting height, lifting hook	150 cm	150 cm
Max. lifting height, lifting hook	190 cm	190 cm
Lifting travel	40 cm	40 cm
Min. height of handle	75 cm	75 cm
Max. height of handle	105 cm	105 cm
Width of handle	65 cm	65 cm
Length of undercarriage	117 cm	122 cm
Length legs	97 cm	98 cm
Min. inside distance between legs	50 cm	54 cm
Max. inside distance between legs	94 cm	118 cm
Min. outside distance between legs	65 cm	69 cm
Max. outside distance between legs	109 cm	133 cm
Height of undercarriage	12 cm	12 cm
Free height under legs	7 cm	7 cm
Wheel diameter	100/75 mm	100/75 mm
Turning circle	125 cm	131 cm
<u>Weight</u>		
Weight of spreader bar (lifting unit)	5.0kg	5.0 kg
Weight of handle unit	8.5 kg	8.5 kg
Weight of battery	3.0 kg	3.0 kg
Total weight	55.0 kg	61.5 kg
Approx. number of lifting operations per charging	100	75

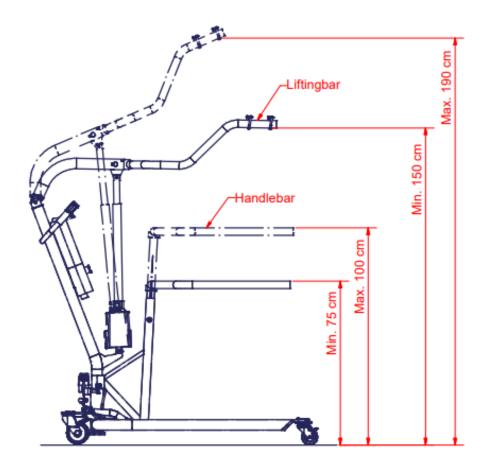

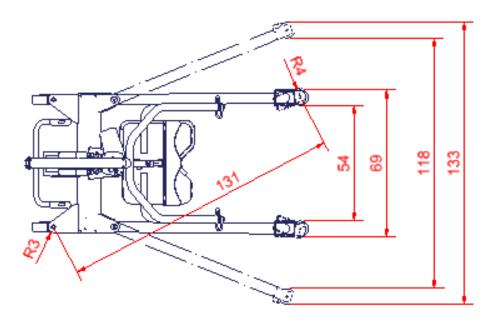


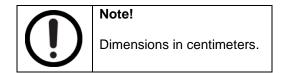


4. Instructions for use

4.1 Installation of product

- 1. Place the lifting mast in the mast holder of the frame. Tighten the two screws with the 5mm Insex wrench provided and use locking nut on opposite site.
- Connect the cable from the leg spreading motor to the control unit in output terminal 2, see section 4.12 "Electrical component diagram".

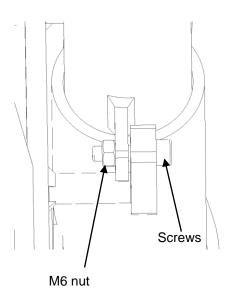

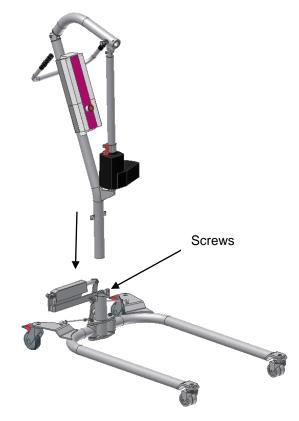




Note!

Assembly of 150kg and 200kg Walking Trainers is identical. Only difference is the undercarriage.

4.2 Installation of lifting unit and hand support

- 1. Brake the rear wheels.
- 2. Hold the lifting unit (turn it through 90°) so that the locking pin may be pushed into the corresponding slide rail of the lifting mast.
- 3. Push in the lifting unit.
- 4. Now turn it 90° clockwise until a click is heard from the locking screw.
- 5. Check that the arm has been properly locked.
- 6. Hook the hand support on to the mast holder.
- 7. Tighten the tightening handle on the side.
- 8. The walking trainer is now ready for use.

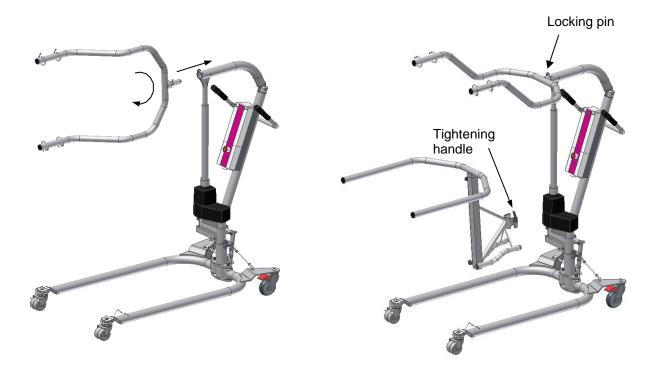

Dismounting

- 1. Brake the rear wheels.
- 2. Loosen the tightening handle.
- 3. Note! Do not unscrew it completely.
- 4. Lift the hand support off the mast holder.
- 5. **Note!** It may be stuck owing to the clamping plates.
- 6. Pull the locking screw while turning the spreader bar 90° counter-clockwise.
- 7. The lifting unit may now be pulled out.




Note!

Be aware of the weight of the lifting unit.

Mounting of directional steering of the front wheels

- 1. Dismount the M10x45 countersunk screw with washer and nut keeping the wheel in place.
- 2. Mount plate and holder for steering using the new <u>M10x50</u> countersunk screw.

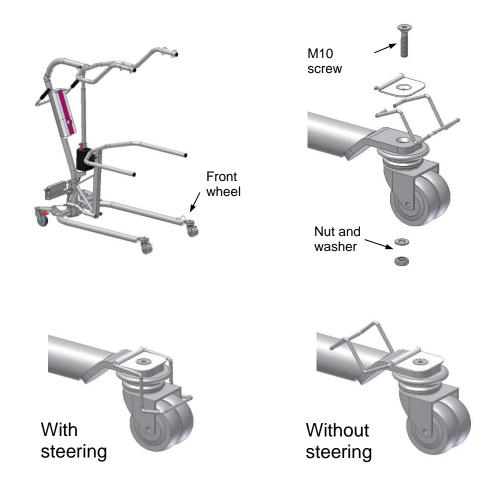

4.3 Operating the product

Note!

It is important that you are familiar with the use of the hoist and its functions, and that it works properly. For the best possible safety, all users of the hoist must read this user manual before using the hoist.

For the All-in-One you can use any of Ropox Domino slings.

4.3.1 Operating instructions

- > Check that the emergency stop has been released. If not, turn the button clockwise until release.
- The hoist is operated by means of the hand control unit. The two upper buttons control the lifting/lowering movement, whereas the two lower buttons control the leg spreading facility. It is not possible to use two functions simultaneously.
- Check that the sling provided is correct for the lifting operation to be performed. This applies to size as well as shape. Further information about the choice of slings will be found in "Domino Slings User manual".
- The battery should be recharged regularly, to ensure that the hoist is always available (e.g. every night).



Note!

In case of erroneous operation of the hand control unit (e.g. too quick change of direction) the hoist does not react. Let go of the push button – wait a few seconds and press again.





The product should always be able to travel the full range of actuation without colliding with objects. Failing to do so will compromise the product stability. Special care should be taken in ensuring that no children or adults are sited under the product as severer injury from entrapment may occur.

4.3.2 Emergency situation

Emergency stop and emergency lowering (electrical and mechanical) should be used in emergencies only. Should it be necessary to activate the emergency functions, contact the distributor before using the hoist again.


4.3.3 Stability function

The lift has a leg spreading function, which is designed to increase the stability of the lift during lifting of a patient. While parallel, the legs have a minimum inside distance between each other of 50cm or 54cm, depending on lifting capacity, but can be increased to 94cm or 118cm. This allows the lift to get as close as possible to a person in a wheelchair.


It is IMPORTANT that during lifting of a patient, that the legs are spread out as much as possible, as you might otherwise risk the lift having reduced stability.

This is not applicable to transporting a patient in the lift, in which case you'd like the legs to be parallel, to be able to get in through doors and be free of obstacles.

150kg

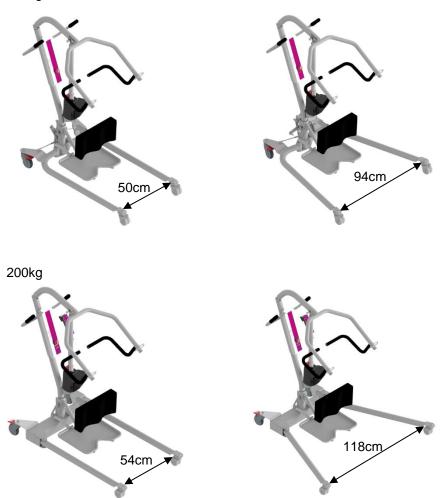

The leg spreading function is controlled with the same remote as the lifting arm.

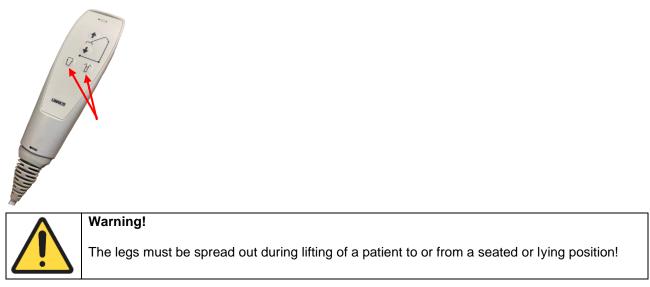

4.4 Recharging the battery

- The battery <u>must</u> be recharged after approx. 35/50 lifting operations, depending on lifting capacity i.e. at 50% battery capacity. In case of continued use you will hear an acoustic signal. The battery can still be used, but it will affect is recharging capability.
- We recommend charging as soon as the hoist is not being used for a longer period of time, e.g. every night. The battery cannot be over-charged and it is only "healthy" for the battery to be recharged often as it increases the battery life.
- Charging is made directly in the control unit by means of the mains cable provided. Insert the cable into the wall outlet and the control unit, see section 4.12, and switch on power. The hoist cannot be used when connected to mains voltage.
- A green light on the control unit indicates connection to mains voltage. A yellow light indicates that the battery is being recharged.
- > When the battery has been fully charged, the yellow light will be out.
- > Full recharging takes 4-5 hours.
- If charging is made via a wall-mounted charging station (optional) a battery may be constantly charged in the station. The batteries may be easily switched by means of the snap system on top of the battery box.

Note!

Make sure that the plug from the battery is correctly inserted in the control box, if the plug has been pulled out, before using the lift again.



4.5 Safety function, lifting motor

The lifting motor is provided with a safety function, spline, against jamming. If the lifting arm meets with resistance during lowering, the movement of the arm will stop automatically. Thus, a person cannot get jammed between the lifting arm and the bed.

4.6 Safety function, control unit

- > Protective motor switch for lifting and leg spreading, protecting against overloading
- Short-circuit protection
- Acoustic signal sounding when the batteries need charging
- Electrical emergency lowering if the hand control unit does not work
- Built-in charger



Warning!

Owing to the safety function on the motor the lifting arm and spreader bar may fall freely to the spindle of the motor. Therefore, make sure that the lifting arm does not "hang", that is has always been pressed down to ensure that it rests on the spindle of the motor before and after all lifting operations. This is important as otherwise an injury may result if the lifting arm "falls" down.


Emergency stop

If the hoist is to be stopped owing to an emergency – press the red button on the control unit. When the button has been depressed the electrical functions of the hoist cannot be activated and the battery indicator has been switched off.

4.7 Emergency lowering

If the hoist is to be lowered and the hand control unit does not work, there are two possible solutions:


- The electrical emergency lowering of the control unit (marked emergency), see section 4.12. Press with a ball pencil or the like and the person will be lowered slowly.
- > The mechanical emergency lowering of the lifting motor. Pull up the red handle, see drawing.

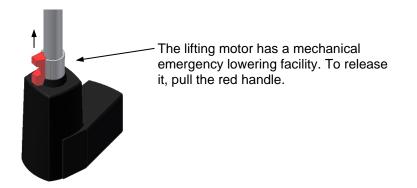

4.8 Brakes

The hoist may be braked on the two large rear wheels. Press down the rear part of the brake with the foot to brake the hoist. Release the brake by pressing the front part of the brake with the foot.

Note!

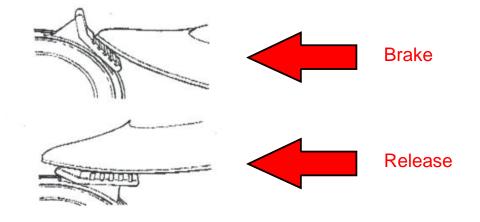



Never brake the hoist during lifting except when used in stand-up mode.

4.9 Size guide for Domino slings

In choosing the correct sling to use, the "Facts-Label" on the sling can be used. Alternatively, the following instructions can be used. This section can also be used to determine which slings to choose, when acquiring the product. It's recommended to use the Domino slings exclusively with the Ropox All-in-One hoist.

4.9.1 Size information

Туре	Walking Trainer Slings		
Height	145 – 175cm	165 – 185cm	175 – 195cm
Weight	40 – 60kg	60 – 80kg	80 – 100kg
Size	Å Small	M edium	Å Large

4.9.2 Dimensions for slings

The walking trainer slings come in three different sizes. For these, the waist size (W) is provided in order to, together with height and weight, choose the best sling for the patient. The waist size is indicated as the **maximum size**. The dimensions are as follows:



Туре	Size	W
	<u>Small</u>	90 cm
Walking Trainer Slings	Medium	110 cm
	Large	125 cm

4.10 Transfer – walking trainer

- > Before lifting a person, it is important to be familiar with the walking trainer and that it works satisfactorily.
- Also, it is important to choose the right size of sling for the lifting operation to be performed. Further information about the use of slings will be found in the "Walking Trainer Manual for Slings".
- We recommend that the walking training take place on a horizontal surface. However, the walking trainer may be used on surfaces with gradients up to 5°.
- > Only brake the hoist when it is parked or standing on a sloping surface, e.g. in bathrooms.
- Beds or wheelchairs, which the client may be transferred from, must be locked in place during lifting.
- Never lift a person higher than necessary.
- Always be aware the maximum load of the hoist.-

Before lifting a client, try the hoist yourself.

Risk assessment shall be carried out to ensure that the correct size, type, and shape of slings is being used for the patient

4.10.1 Lifting from chair/wheelchair

- 1. Place the sling around the client sitting in the chair/wheelchair. Close the body belt and tighten it so that it is still comfortable.
- 2. Place the leg straps between the thighs and pull the lifting straps placed in front of the shoulders through the black loops of the leg straps. Make sure that the four grey lifting straps are in the same height before attaching them to the spreader bar.
- 3. Increase the width between the legs of the hoist and move it towards the client.
- 4. Place the feet of the client on the floor.
- 5. Move the hoist so close to the client as possible.
- 6. Make sure that the spreader bar is in the lower position.
- 7. Place the lifting straps of the sling correctly in the hooks of the spreader bar. Check that all straps have been fitted correctly.
- 8. Lift the client a few centimeters above the seat STOP and make sure that the sling is placed correctly.
- 9. When the client is comfortable, lift him/her from the chair to standing position.
- 10. Slowly pull the hoist away from the chair STOP adjust the sling and lifting height until the client is standing comfortably.
- 11. You may now start walking training.



The wheelchair must be braked during transfer to and from the chair.

4.10.2 Using the walking trainer

- The carer may walk in front of or beside the walking trainer, instructing or supporting the client.
- Raising or lowering the sling frame will relieve the client depending on the phase of walking training.
- Use the hand support to begin with in order to give the client something to hold on to and to use for pushing/steering.





- During the last phase of the walking training the client may use the hand support only without the support of the sling.
- The steering devices of the two front wheels contribute to steering the walking trainer, e.g., when persons with hemiplegia start walking training.

4.10.3 Placing in chair/wheelchair

To place a person in a chair/wheelchair, do the following procedures:


- 1. Increase the width between the legs of the walking trainer and move it to the chair.
- 2. Make sure that the client is positioned correctly above the chair and lower the hoist slowly.
- 3. Check that the client is positioned correctly during the entire operation.
- 4. When assisting a client in getting back into the chair, lower the client so that he/she just touches the chair.
- 5. If the chair can be tilted a little backward it is easier to place the client correctly.
- 6. If two assistants are present, one may press gently on the knees of the client.

4.11 Height adjustment of hand support

Press the red button, then pull locking tap. Hand support can now be pulled upwards or pushed downwards for adjustment of correct height for user.



4.12 Electrical component diagram

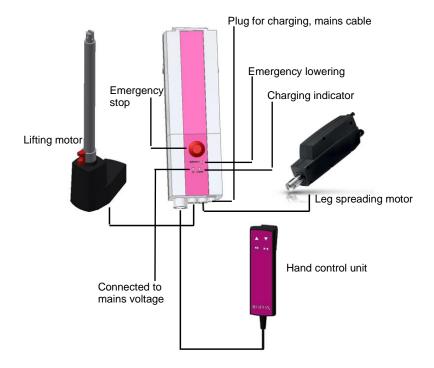

5. Troubleshooting

If the walking trainer does not work, check, and test the following procedures before contacting an authorized distributor.

- Has the battery been charged?
- > Has the emergency stop been released?
- > Have all plugs been inserted correctly?
- > Does the hand control unit work properly and has it been mounted correctly?
- > Try with another hand control unit or charger.
- If the lifting arm cannot be lowered by means of the hand control unit, use mechanical or electrical emergency lowering, and the lifting arm will come down.

Symptom	Possible cause	Action
Indicator for mains voltage does not light up	 Mains voltage not connected Control unit defective 	 Connect mains voltage Replace the control unit
The motors do not start. A click is heard from the relays of the control unit	 The plug has not been correctly inserted into the control unit Motor defective Control unit defective 	 Insert the motor plug correctly into the control unit Replace the motor Replace the control unit
The motors do not start. No click is heard from the relays of the control unit	 Control unit defective Hand control unit defective 	 Replace the control unit Replace the hand control unit
The control unit is completely dead. No click is heard from the relays of the control unit	 Battery completely discharged Battery defective 	 Charge the battery Replace the battery
The control unit is in order except for one direction of one channel	- Hand control unit defective - Control unit defective	 Replace the hand control unit Replace the control unit

6. Cleaning

Clean the walking trainer with a damp cloth wrung in warm water with a mild detergent. Electrical components may be dried with a firmly wrung cloth. Do not use water directly on these components. Never use solvents.

	Note!
\bigcirc	Do not use cleaning agents containing abrasive e.g. scouring powder, steel wool, scouring sponge.
$\mathbf{\overline{\cdot}}$	This product is not designed to be sterilized, autoclaving and sterilization beyond normal cleaning is may possibly change the product safety and function.

Preapproved cleaning agents

Trade name	Solution
Acticlor plus	1000ppm
S90 sanirens	1-3% peracid
FUTUR, Alkalisk rens	1% solution [pH 9,5]
Overfladedesinfektion	70% Ethanol
Suverent Universalrengøring	1% solution [pH 8]

7. Maintenance



Note!

Failing to comply with periodic maintenance may result of degrading of product function and safety.

7.1 Periodic maintenance

Always make sure that the mechanical and electrical systems of the hoist operate satisfactorily. In case of malfunction stop the hoist and check or repair it, as required. Malfunction may be a symptom of a defect, which may become worse and present a safety risk in critical situations. It is therefore important to check the hoist as described below.

7.2 Daily check

<u>Sling</u> 1. Check that the slings are clean, if not, send them to the wash.

- 2. Check for wear and that the seams are intact and not frayed. A worn sling must be replaced.
- <u>Hoist</u> 3. Check that the hand control unit works satisfactorily and is mounted correctly.
 - 4. Check that cables have been correctly inserted into the control unit.
 - 5. Check visually that shaft and bolted joints run smoothly and silently.
 - 6. Keep the hoist clean. A clean hoist is safer and has a longer life.

7.3 Monthly maintenance

- Sling 1. Inspect the slings carefully and replace them in case of visible defects or wear.
- Hoist 2. Check that hand control unit, control unit and battery have no visible defects.
 - 3. Check that cables have been inserted correctly and are undamaged.
 - 4. Check that the lifting motor is not bent and is undamaged.
 - 5. Check that all shaft and bolts joints are undamaged and stable. Worn parts should be replaced.
 - 6. Check that the wheels run smoothly. Clean them once a month.

7.4 Yearly inspection

Preventive service <u>must</u> be made once a year. Follow the procedures below must be followed.

- Forms to service report

Take a copy off the forms on page 24 and 25 and use them when filling out the service report. We recommend scaling them up to A4 size.

- Visual inspection

Inspect hoist frame and welds for wear, skews, or any other damage.

- Testing

Check all functions with and without load.

Check emergency stop.

Check battery and charger.

- Mechanical inspection, se also section 7.5

Inspect that all parts are moving freely and without any abnormal noise.

Note! Bolt M6x50 holding Spreader bar should be dismounted and inspected visually.

Note! Axle holding lifting arm should be dismounted and inspected visually .

Inspect, that bolts are tight.

- Electric inspection, also see also section 7.6

Check actuator for damage, noise, and the fixtures.

Check the battery voltage with max load and a full lifting cycle. If below 19V the battery needs to be changed. Replace defective battery, if any.

Check wires for damage.

Check all plugs and sockets.

7.5 Maintenance report mechanical parts

Invoice address:		Delivery	/ address:			
Repair	Mounting		Service		Warranty	
Lift type:			Serial No.			
Inspection carried o	ut		ОК	Defective	Repaired	Replaced
Lubrication of wheels						
Lubrication of load be hooks	aring shafts	and				
Welds						
Bolts / bushings						
Mast and arm connec	ction					
Arm and lifting ascess	sories					
Test of the leg-spreading function						
Switches and safety functions						
Test of batteries and recharging function						
Test of the seams of the Domino slings						
All bolts						
Lifting motor and leg- See separat pages re						
Other						
Functional test of the lift						
Consumption of spare	e parts					
Spare part No:	Products:					
Comments:						
Milage: Time consump			tion:			
Date: Performed by:						

7.6 Maintenance report actuator/lifting motor

Invoice address:		Delivery address:						
Repair	Mounti	na	Servi	се		Warranty		
Lift type:		5	Serial No.					
Inspection carrie	dout		OK Not OK Remarks					
			UN	NOL OK	Remain	.5		
Powercomsumptio								
Powercomsumptio	on with m	nax. load						
Noise								
Plastic								
Wires and cords								
Straightness and on the actuator is con	outbendi moletly c	ng when						
Looseness	<u>p.e</u> j e							
Fixing points								
Marks and dents								
Other	Other							
Consumption of sp	oare part	S						
Spare part No:	Pro	ducts:						
Comments:								
Milage:		Time cor	sumpt	ion:				
Date: Performe		ed by:						

7.7 Lifetime of actuator/lifting motor

A service technician fills out the below points from the user information

How often is the lift used pr. day:

Actuators production date:

Expected changing of actuator:

7.7.1 Table to determine an actuators lifetime

Note!! The lifting actuator should be changed when it is max. 6 years or after 10.000 strokes.

Cor	ntinue	e normal use Consider changing Actuator should be changed							anged		
Lifts		Age in years									
pr. da	ay	1	2	3	4	5	6	7	8	9	10
•	1	365	730	1095	1460	1825	2190	2555	2920	3285	3650
	2	730	1460	2190	2920	3650	4380	5110	5840	6570	7300
	3	1095	2190	3285	4380	5475	6570	7665	8760	9855	10950
	4	1460	2920	4380	5840	7300	8760	10220	11680	13140	14600
	5	1825	3650	5475	7300	9125	10950	12775	14600	16425	18250
	6	2190	4380	6570	8760	10950	13140	15330	17520	19710	21900
	7	2555	5110	7665	10220	12775	15330	17885	20440	22995	25550
	8	2920	5840	8760	11680	14600	17520	20440	23360	26280	29200
	9	3285	6570	9855	13140	16425	19710	22995	26280	29565	32850
	10	3650	7300	10950	14600	18250	21900	25550	29200	32850	36500
	11	4015	8030	12045	16060	20075	24090	28105	32120	36135	40150
	12	4380	8760	13140	17520	21900	26280	30660	35040	39420	43800
	13	4745	9490	14235	18980	23725	28470	33215	37960	42705	47450
	14	5110	10220	15330	20440	25550	30660	35770	40880	45990	51100
	15	5475	10950	16425	21900	27375	32850	38325	43800	49275	54750
	16	5840	11680	17520	23360	29200	35040	40880	46720	52560	58400
	17	6205	12410	18615	24820	31025	37230	43435	49640	55845	62050
	18	6570	13140	19710	26280	32850	39420	45990	52560	59130	65700
	19	6935	13870	20805	27740	34675	41610	48545	55480	62415	69350
	20	7300	14600	21900	29200	36500	43800	51100	58400	65700	73000
	21	7665	15330	22995	30660	38325	45990	53655	61320	68985	76650
	22	8030	16060	24090	32120	40150	48180	56210	64240	72270	80300
	23	8395	16790	25185	33580	41975	50370	58765	67160	75555	83950
	24	8760	17520	26280	35040	43800	52560	61320	70080	78840	87600
	25	9125	18250	27375	36500	45625	54750	63875	73000	82125	91250
	26	9490	18980	28470	37960	47450	56940	66430	75920	85410	94900
	27	9855	19710	29565	39420	49275	59130	68985	78840	88695	98550
	28	10220	20440	30660	40880	51100	61320	71540	81760	91980	102200
	29	10585	21170	31755	42340	52925	63510	74095	84680	95265	105850

8. Component part list




Note!

Ropox will make parts list and circuit diagrams available for extraordinary maintenance and repair beyond what is described in this manual. Repair beyond what is described in this manual may only be done by Ropox designated service personnel.

8.1 Spare parts

Position	Description	Number in product	Item no.
1	Battery	1	25*29030-002
2	Control box	1	150kg = 25*29020-002
			200kg = 25*29530-001
3	Lifting motor	1	25-20235-9
4	Hand control unit	1	25-29050-9
5	Leg spreading motor	1	25-20212-9
6	Brake wheels	2	25-98202-9
7	Wheels	2	25*98212-002

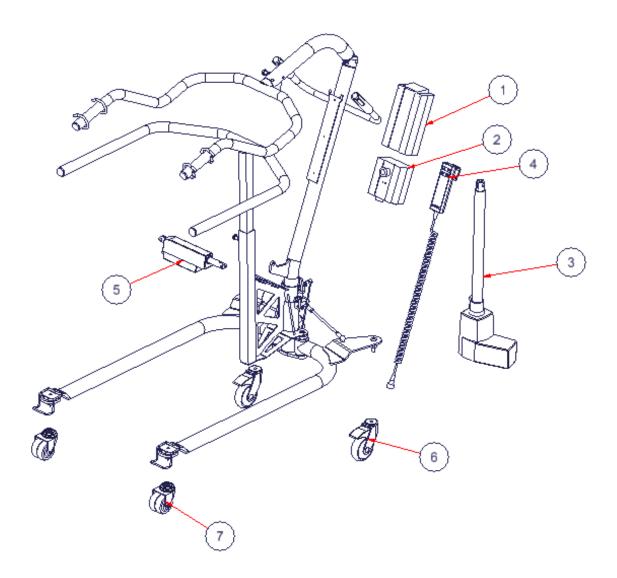

9. Environmental protection

The product is not intended to be disposed as municipal waste. Proper disassembly, sorting and disposal of components must be done by waste disposal professionals.



Part	Disposal recommendation
Control box	Electronics
Motors	Electronics
Handcontrol unit	Electronics
Battery	Electronics
Handle	Hard plastic
Frame	Steel waste
Wheel	Hard plastic

10. Electromagnetic compatibility

10.1 Suitable Environments

The device is suitable for use at home, at daycare centers, at day centers for persons with physical or mental disabilities or at hospitals except near active HF surgical equipment and the RF shielded room of a medical electrical system for magnetic resonance imaging, where the intensity of electromagnetic disturbances is high.

10.2 Cables

Cable component	Length
Hand control spiral cable	Approx. 2.4m extended.



Warning!

Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

10.3 RF portable equipment



Warning!

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

11. Complaints

See our general terms of sale and delivery on www.ropox.com



Ringstedgade 221 DK – 4700 Næstved Tel.: +45 55 75 05 00 Fax.: +45 55 75 05 50 E-mail: <u>info@ropox.dk</u> <u>www.ropox.dk</u>