







# Ropox Diagonal, 30-409xx-1

User manual Mounting instruction

Keep this folder with product at all times!

# Table of Contents

1.		3
2.	COMPLIANCE WITH EU-DIRECTIVES AND UK-DIRECTIVES	3
3.	APPLICATION	3
4.	TECHNICAL DATA	4
5.	SCHEMATIC DIAGRAM OF FRAME	5
	<ul> <li>5.1.1 Dimension sketch</li></ul>	6 7
6.	MOUNTING INSTRUCTIONS	8
	<ul> <li>6.1. Connection of cables</li></ul>	14 16 18 19 21 23 23
7.	MOUNTING INSTRUCTIONS FOR 2 UNITS	
	<ul> <li>7.1 Connection of cables</li></ul>	26 28
8.	PERFORMANCE TEST	30
9.	LIST OF COMPONENTS FOR DIAGONAL, L=40-180CM / 15,7 – 70,8 IN	31
10.	OPTIONS FOR DIAGONAL	33
11.	PLACING OF LED SPOT	35
12.	SAFETY IN USE	36
13.	CLEANING AND MAINTENANCE	37
	<ul> <li>13.1 Cleaning of frame</li></ul>	37 37
14.	TROUBLE SHOOTING	39
15.	14.1 The height of the frame cannot be adjusted	

# 1. Introduction

Ropox Diagonal is an electric height adjustable frame for wall units. Diagonal is especially suitable for wheelchair users or persons who normally have difficulties reaching upper shelves in wall units.

This document must ALWAYS accompany the product and be read by and available to users

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.

Children should be supervised to ensure that they do not play with the appliance.

All users must follow these instructions. It is very important that the instructions have been read and understood prior to operation of the product.

This instruction must be accesseble to the user and keept with the product if it is moved to another place.



If this product is electrically adjustable in height there is a risk of trapping. The product must therefore always be operated by or under the guidance of an experienced adult, who has read and understood the importance of section 12 "Safety in use"

The correct use, operation and inspection are decisive factors for efficient and safe performance.

# 2. Compliance with EU-Directives and UK-Directives

This product has CE-marking according to the current Machinery, EMC and Low Voltage Directives and thus complies with the basic safety requirements.

This product has UKCA-marking and thus complies with the basic safety requirements.

If these frames are assembled or otherwise connected with other electrical components, this will be considered a new unit. Consequently, the assembled unit must be subject to a risk assessment, after which the CE mark may be awarded.

## 3. Application

Diagonal is designed to obtain optimum working height for the user. Diagonal has been designed for height adjustment of wall units. Diagonal must NOT be used as a lifting table or person lifter.

The product must be used indoor, under normal room temperatures and humidity as stated in section 4.



The control unit complies with IP32/II and must always be installed in accordance with the national Heavy Current Regulations or corresponding national or international standards.

# 4. Technical data

Product name:	Ropox Diagonal		
Item numbers:	Item numbers:		
	Width of wall units 40-180 cm / 15,7 – 70,8in :30-409xx-x		
Material:	Welded steel tubes St. 37 and various plastic components.		
Surface treatment:	Blue chromate, powder coating.		
Power supply:	220-230V~ / 50-60Hz, 5,0A		
Standby primary:	0,3W		
Control voltage:	24VDC		
Duty circle.:	Max. 2 min. active then 18 min. pause		
Max. load of frame:	120 kg / 264 lb evenly distributed over the frame		
Speed:	Approx. 2 $^{mm}/_{sec}$ - Approx. 23 sec for a complete stroke		
Temperature:	5-45° C		
Air humidity:	5-85% (non-condensing)		
Complaints:	See Complaints, page 40		
Producer:	Ropox A/S, DK-4700 Naestved, Tlf.: +45 55 75 05 00 E-mail: info@ropox.dk - <u>www.ropox.com</u>		

# 5. Schematic diagram of frame

#### 5.1.1 Dimension sketch

It's important that electrical connections to control unit are flexible and can move freely within the range of adjustment of frame.


#### Conditions for schematic diagram:



Height of wall units = <u>704 mm / 27,7 in</u>- (with another height of wall unit regulate diagram accordingly)

(Wall unit heights that can be used, 704-920 mm / 27,7 - 36,2 in)

Height of upper edge of wall unit = <u>2120 mm / 83,4 in (</u>if another height required, regulate diagram accordingly)

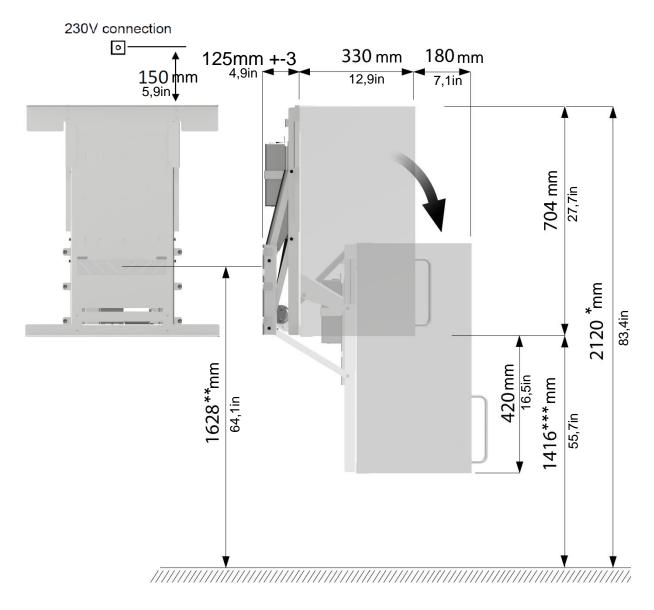

- \* Height can be individual adjusted
- \*\* Depends of height to top of cabinet
- \*\*\* Depends of height to top of cabinet and cabinet size

#### 5.1.2 Construction – see dimensions on the following page



Minimum area for reinforcement of wall. Wall material must be suitable for screw mounting.




Area for electric installation.

Area for conduit.

Pull down cable from wall units to edge of worktop for placement of control button. We recommend a build-in pipe in wall where the cable can be pulled through (ø3cm).

If a high wall unit is placed right next to a Diagonal, it is an advantage to pull the cable for the control trough wall unit then a pipe in wall can be avoided.

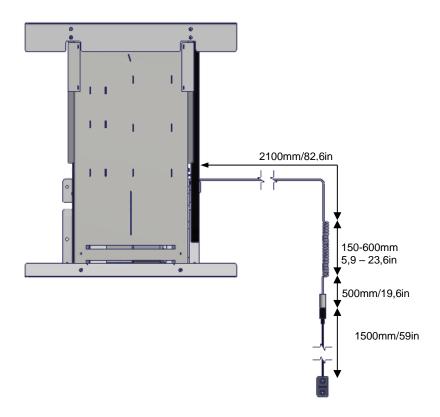

#### 40 – 120 cm / 15,7- 47,2 in

Max. load 120kg / 264 lb evenly distributed, according to DS/EN 12182, based on 10 screws/bolts.


#### 130 – 180 cm / 51,2 – 70,8 in


Max. load 220kg / 485 lb evenly distributed, according to DS/EN 12182, based on 20 screws/bolts.


#### Minimum load capacity per screw/bolt:

45 kg / 99 lb


The fitter should always consider the material, condition and strength of the wall and use screws and rawlplugs suitable for the specific wall type.

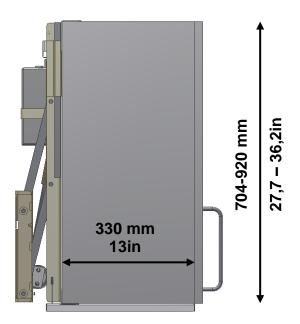

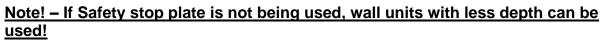

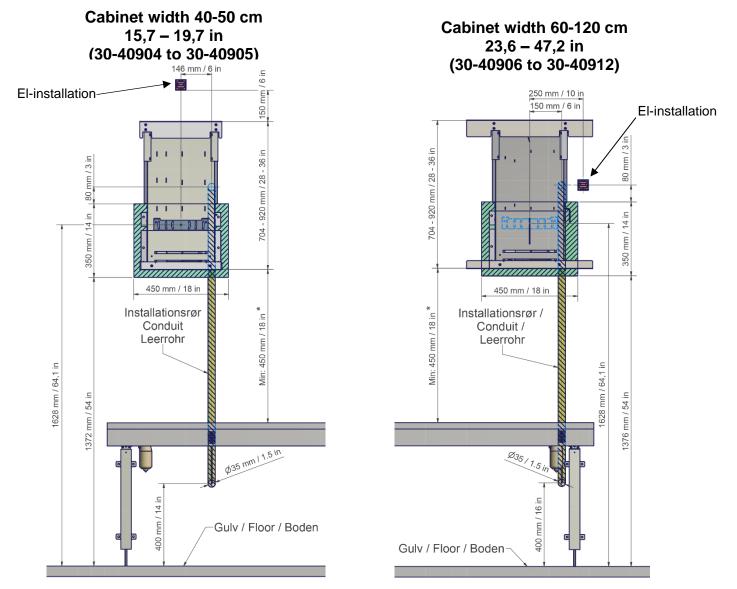


#### Cabinet width 130-180 cm 51,2 - 70,8 in (30-40913 to 30-40918)

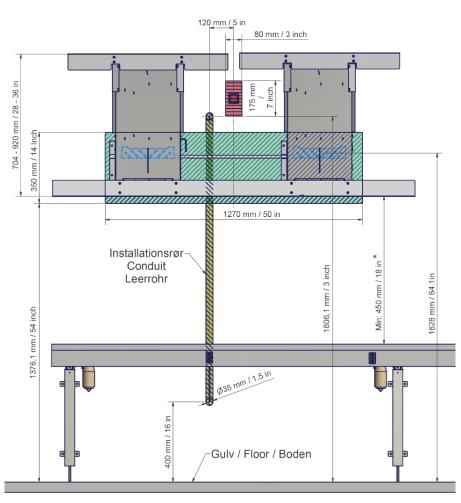

5.1.4 Fixed cupboards

An Installation of Diagonal frame shifts & places the wall cupboards 115mm +-3mm / 4,5 in out from the wall. If you require/wish for any existing fixed cupboards to be in line with Diagonal we recommend an installation of the Diagonal first. In this way the best result is ensured.

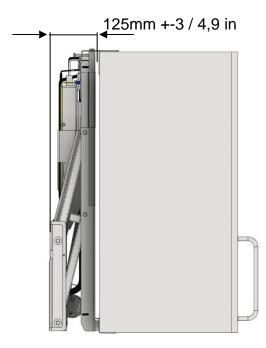



Assembly must always be carried out by competent personnel


Always consider the integrity of the wall when installing. For particularly soft or unstable walls, it is recommended to use a plywood or OSB sheet backing. We recommend to span the reinforcement across at least two wall studs for optimal stability


Prior to assembly check that all parts have been provided. See list of components, section 9


#### Height of top wall unit (edge) = 2120 mm / 83.4 in - see schematic diagram page 5

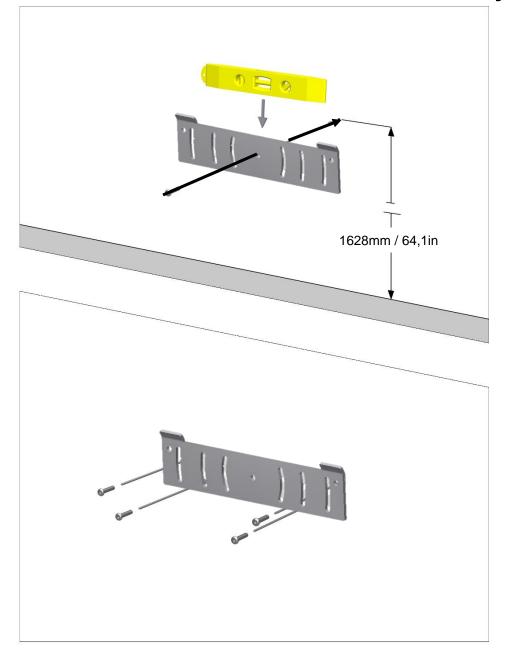

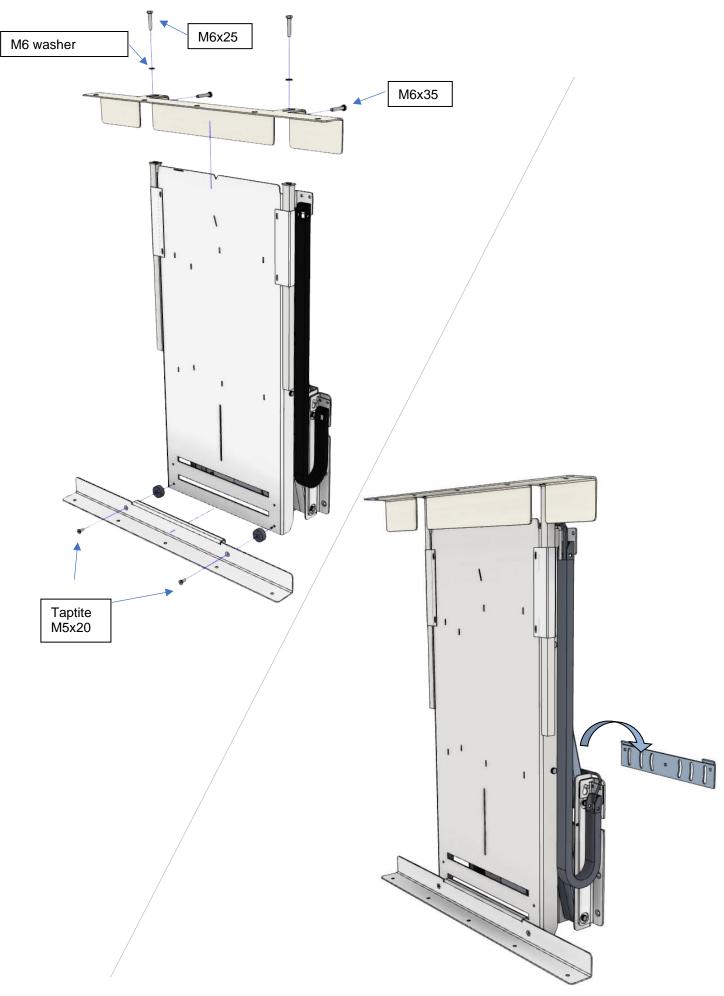


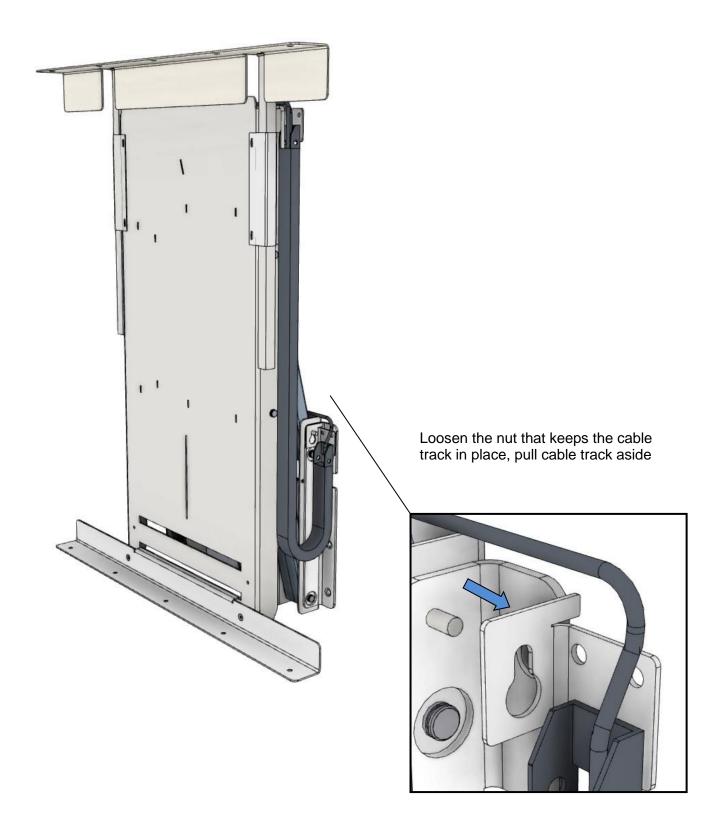

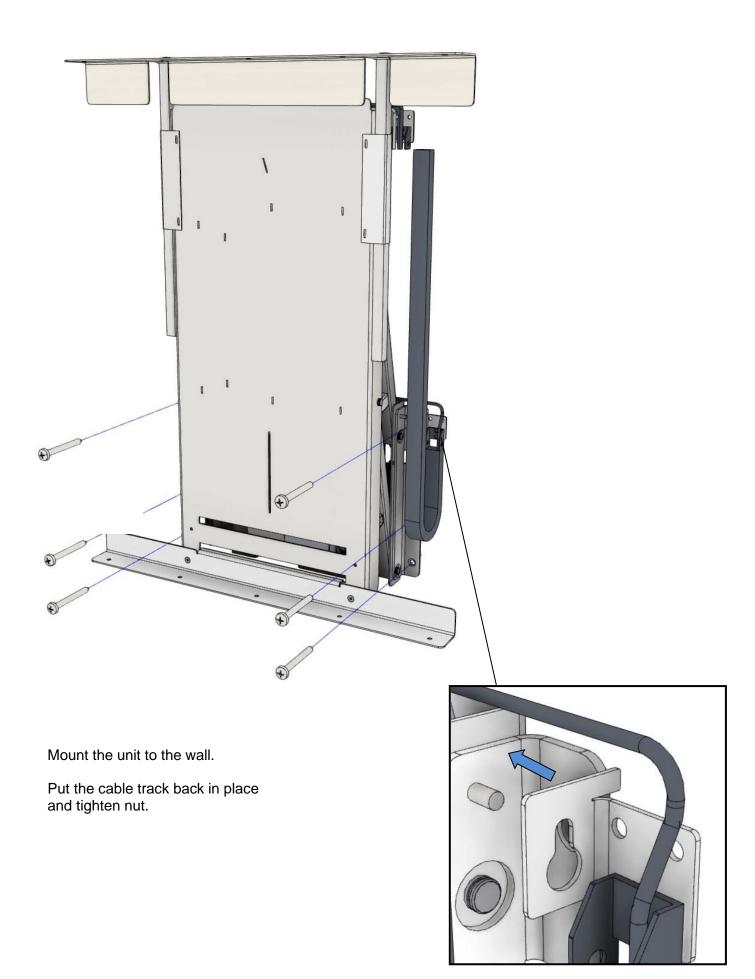


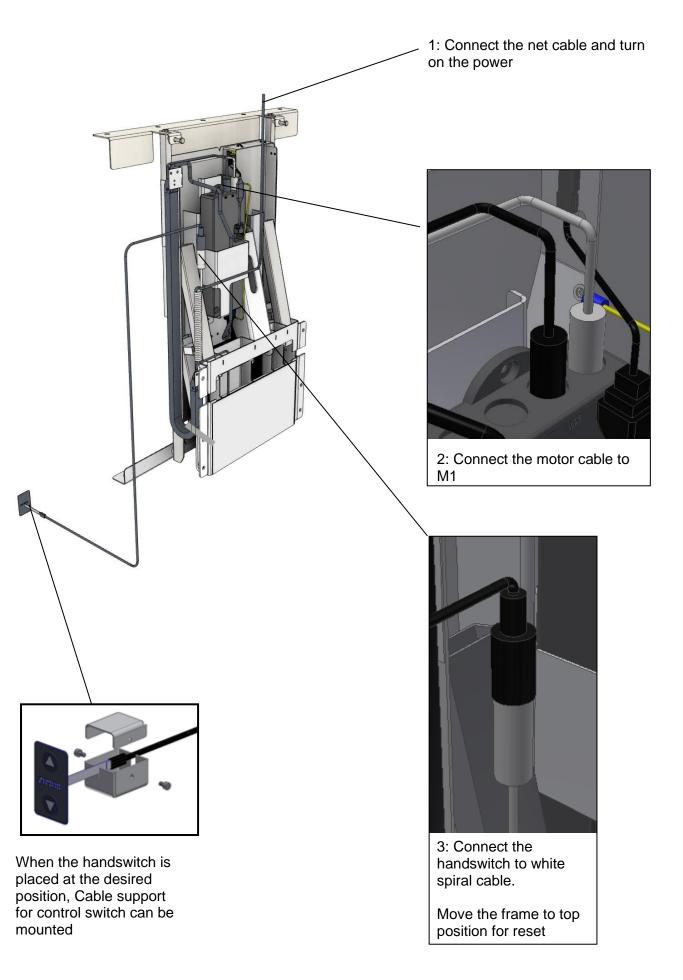

#### 6.2 Mounting of cable support for net cable

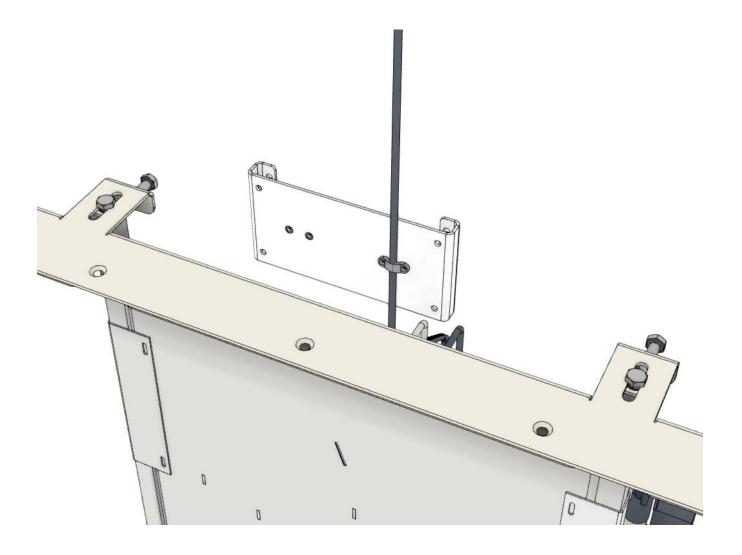


# Recommended placement, seen from above: To the left of the box

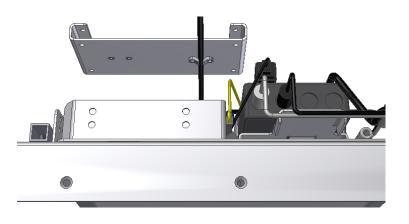



# Recommended placement, seen from side:

Lower than the cabinet plate. The cable support will be hidden in top position

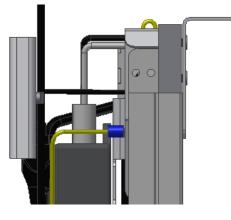

Attach cupboards to sliding frame and fasten

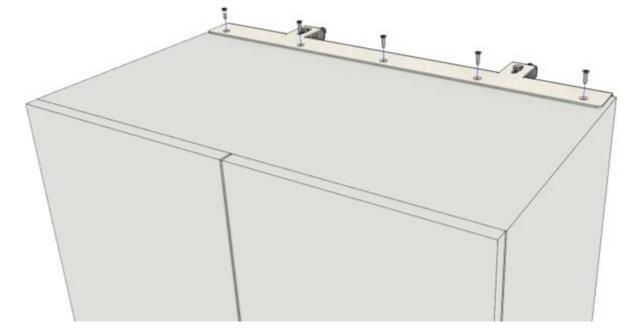

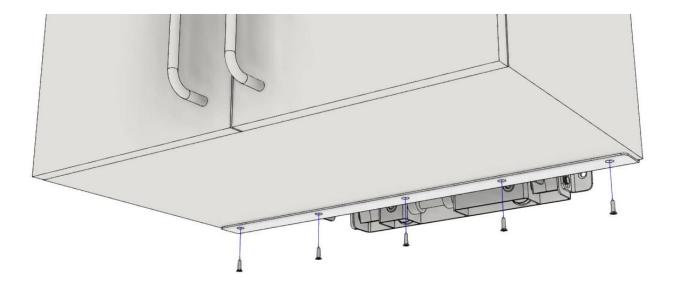


Fasten cupboard at the bottom


#### 6.4 Leveling cabinet

In case the cabinet is not fully plumb after installation, it is possible to level out the system by performing the following steps:

1. Loosen the two top screws approximately one revolution, to allow them to move freely in the track

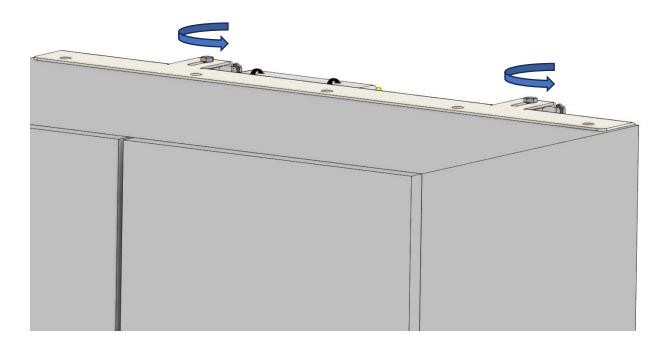


2. Adjust the leveling screws from behind the unit

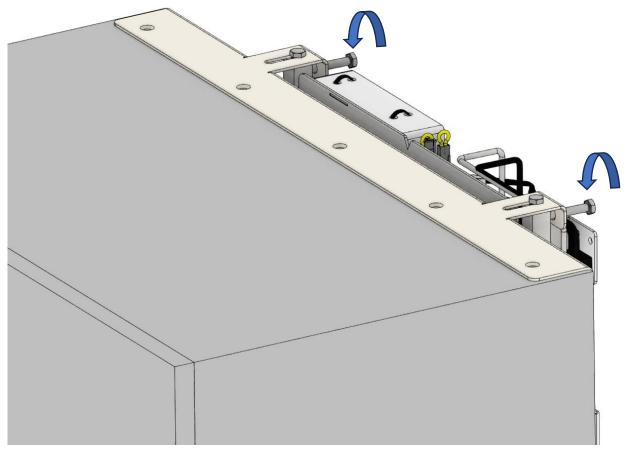

Adjust until the cabinet is plumb




3. Tighten the two top screws again to lock the system

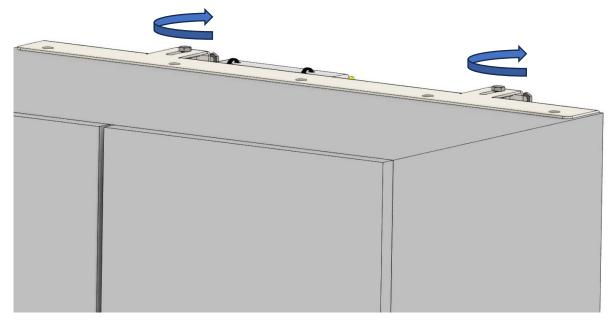


#### 6.5 Mounting of safety stop

Safety Stop is an accessory for diagonal


### 6.5.1 Placement options

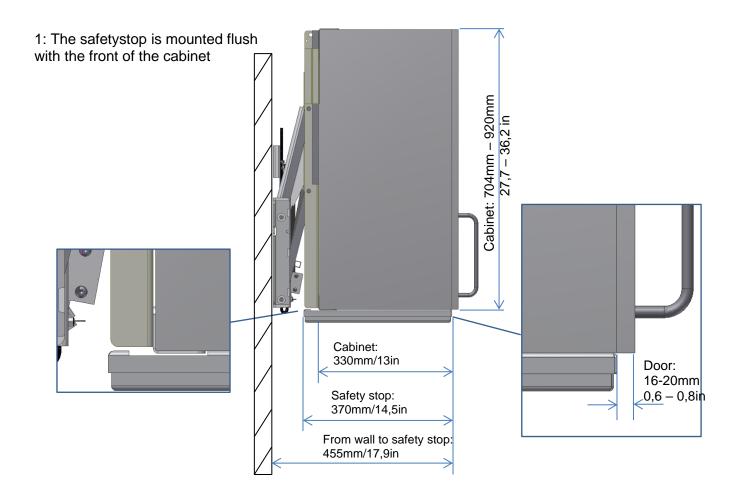

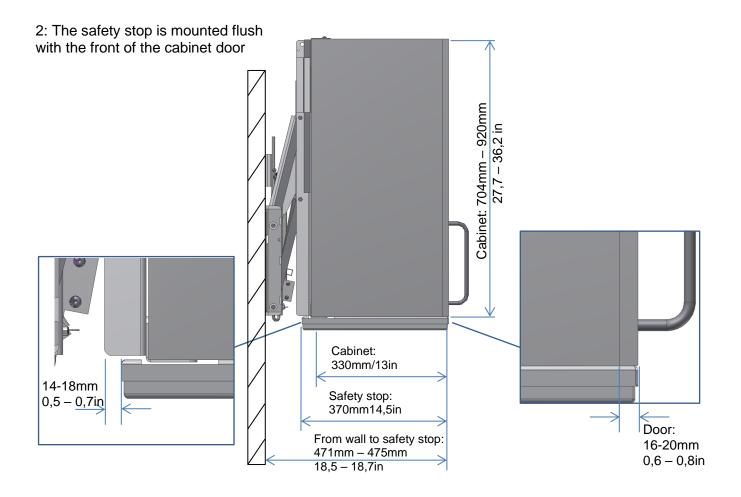



#### 6.5.2 Mounting on cabinet

Place the safety stop in center, under the cabinet. The safety stop can be mounted flush with cabinet or cabinet door. Hold the safety stop in place with a clamp.

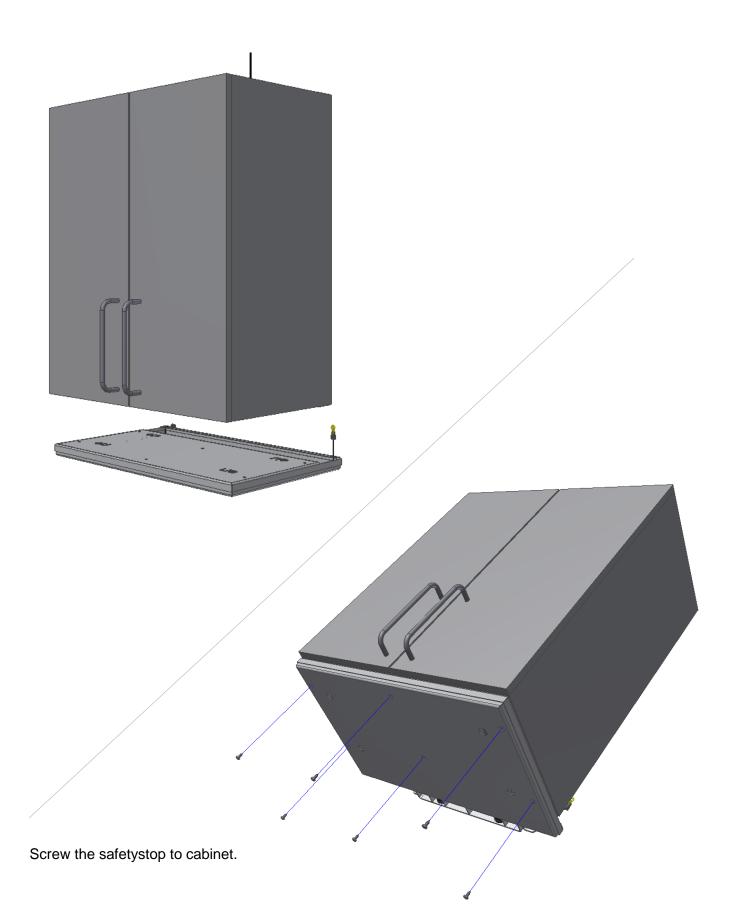


In the right side of the safety stop there is a yellow 2,2k $\Omega$  resistor mounted.

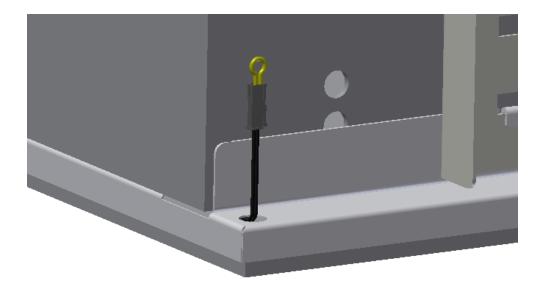

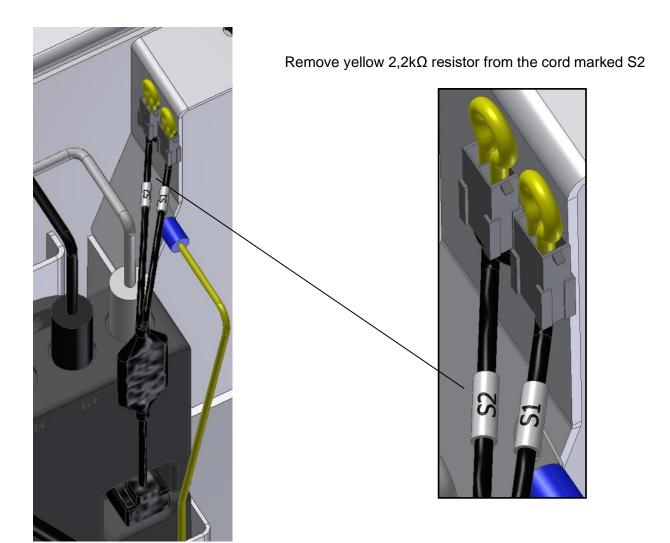

TF 200.0087\_eng / Rev 007

Connect extension cord to the cord marked S2 and fasten it with cable binders as shown.

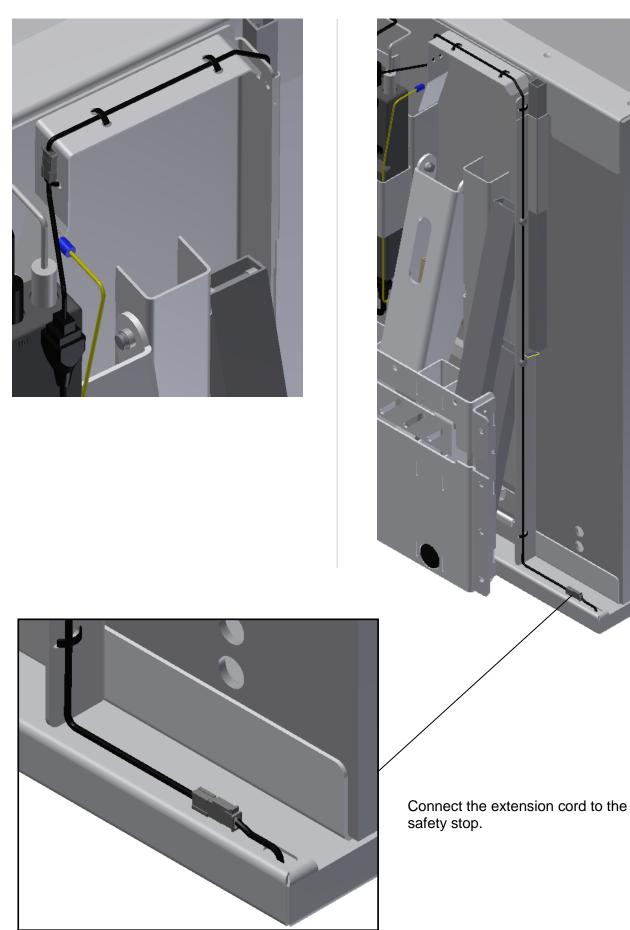


6.7 Mounting the Bluetooth Adapter for Ropox Connect App



6.7.1 Determine the placement of the adapter, then grab the Bluetooth adapter and the split-cable

6.7.3 If the adapter is to be fixed, it should be done now

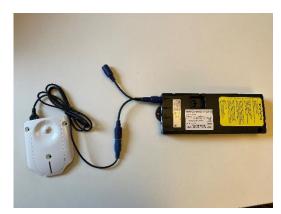

6.7.2 Insert the split-cable in the plug "**HS**" in the control box. Then, insert the adapter in one end of the split cable, and the button that came with the product, in the other

### 6.8 Downloading the Ropox Connect App


As an option to operate certain Ropox products, it's possible to download the free Ropox Connect App, which makes it possible to operate the product wirelessly. The Ropox Connect App allows you to operate the product upwards or downwards and has the option of saving height settings, allowing users to easily change from a standing height to a sitting height. To see an expanded explanation of functions, accessibility settings, and more, see the enclosed manual, TF 200.0054. To download the Ropox Connect App, follow the instructions below:

#### How to download the Ropox Connect App


#### For iOS

- 1. Scan the QR code or open the app: App Store.
- 2. In the App Store, press 'Search'.
- 3. Search for "Ropox Connect".
- 4. Find the Ropox Connect App, and press 'Get'.




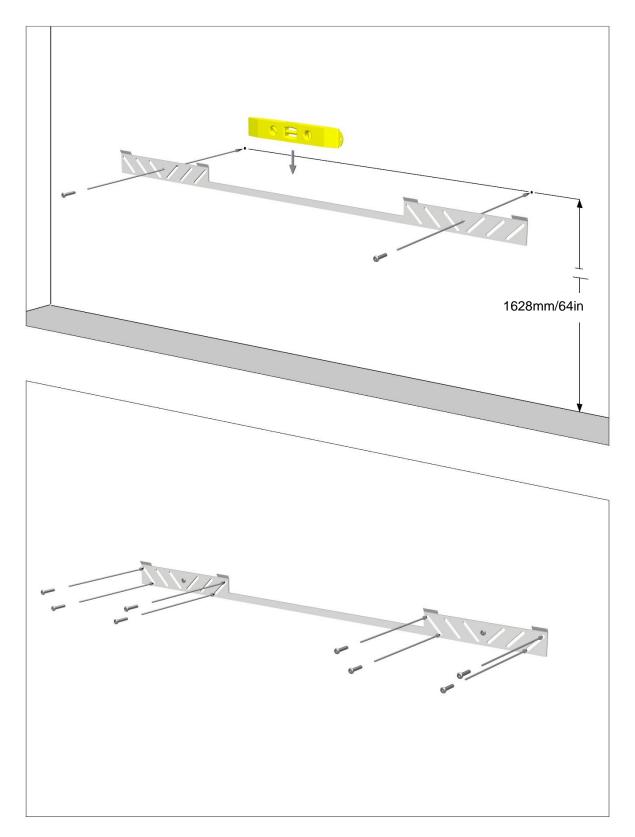

Mount the 2 units as described on page 10-12

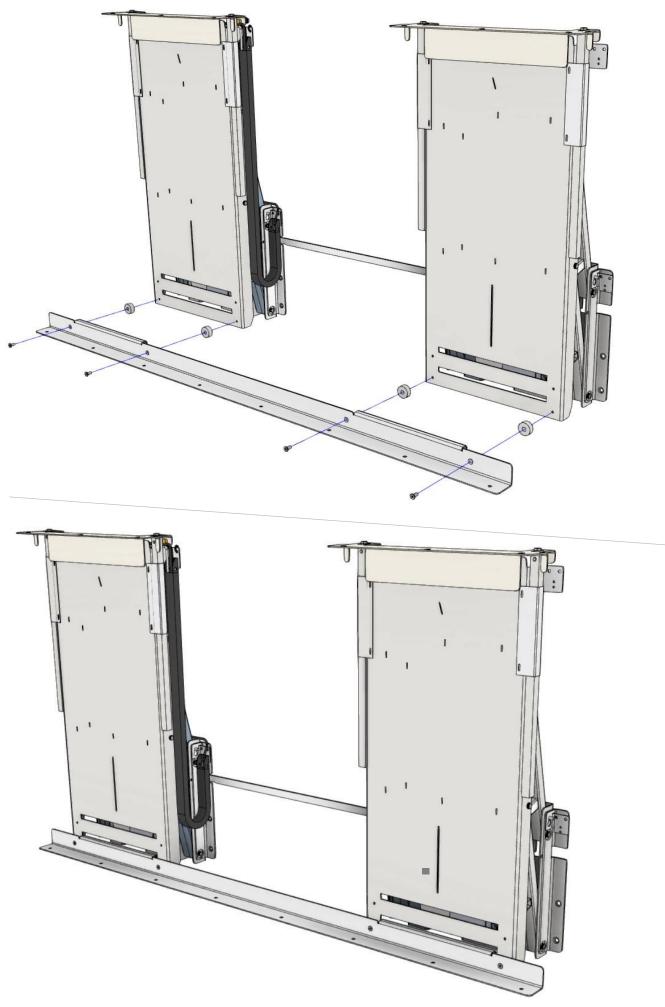


#### 7.1 Connection of cables

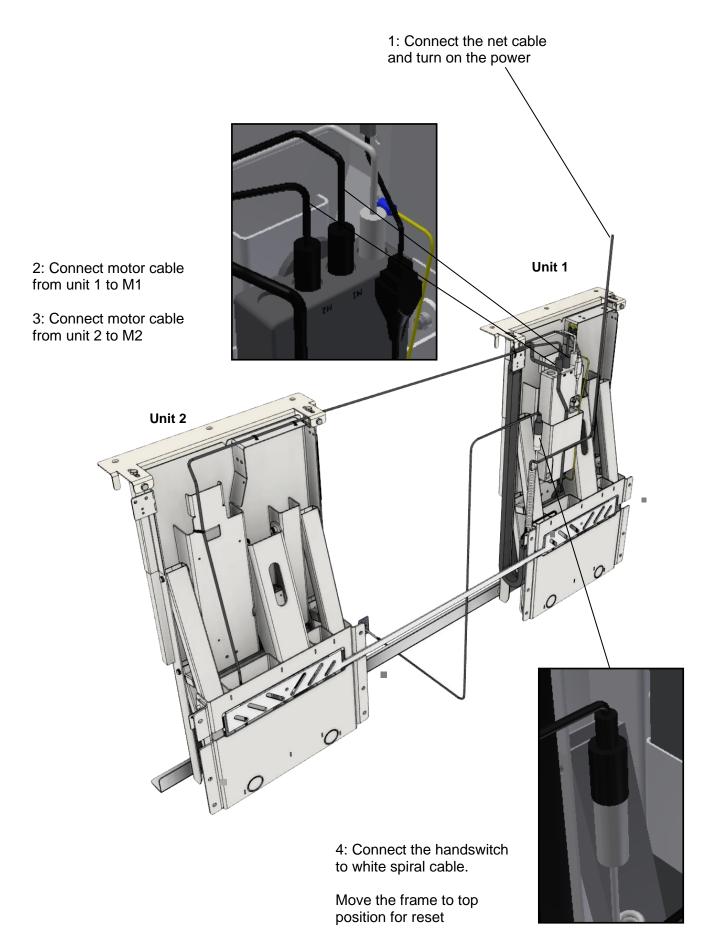

# 7.2 Mounting of cable support for net cable


See page 14.

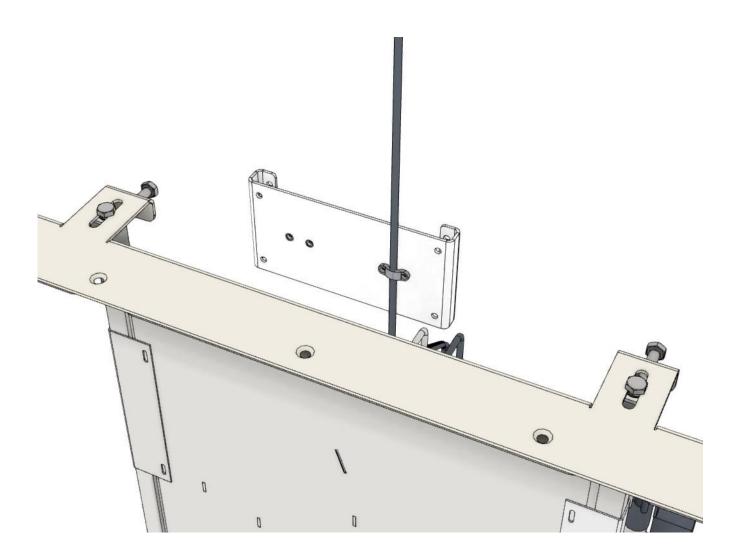


# 7.3 Mounting of cabinets


See page 15




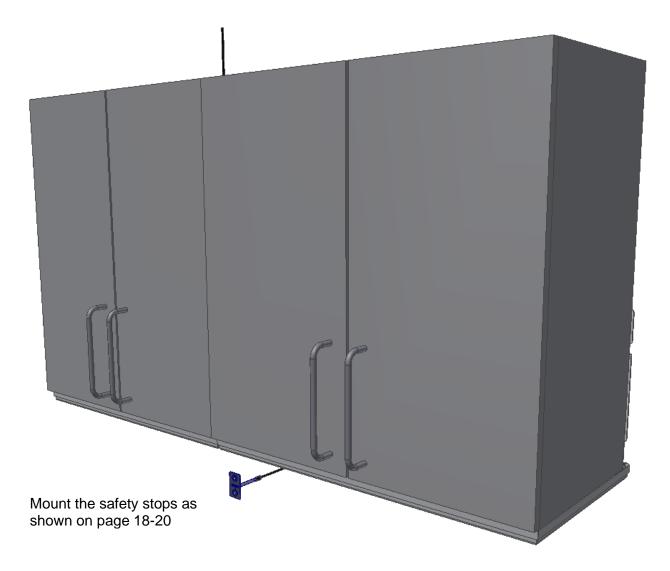

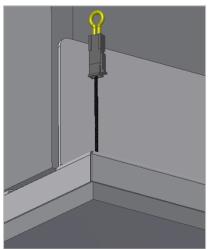

Place plug with yellow  $2,2k\Omega$  resistor on the right side of safety stop plate (shown from the front)

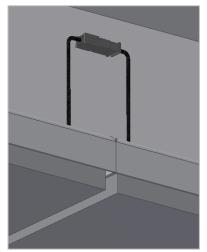

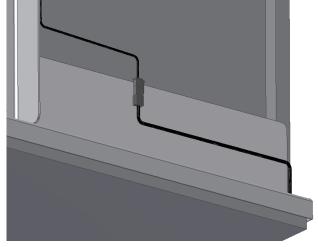

Connect the two cables for safety stop plates in the middle

Connect extension cable of control box to safety stop plate on the left side (shown from the front) and fasten it with cable binders as shown on page 21

## 8. Performance test

After installation and prior to use all functions of unit must be tested. The test must be carried out by competent personnel. Subsequently the test shall be carried out once a year.

Testing prior to connection to mains voltage:

- 1. Check that mounting instructions have been observed
- 2. Check that all bolts have been tightened.
- 3. Check that all cables have been connected correctly and that the plugs have been pressed home.
- 4. Check that there is no load on the frame.
- 5. Check that there is nothing preventing the frame from moving freely within the height adjustment range.

#### Start-up Procedure

- 1. Connect the net cable and turn on the power
- 2. Connect the motor cables to M1 (M2)
- 3. Connect the control switch to HS
- 4. Press DOWN on the control switch, move the frame to bottom position, check that the movement is even and smooth. Make sure that the cable connections follow the movement of the frame and that they do not get squeezed. The frame is now reset.
- 5. Press UP on the control switch, move the frame to top position and check that the movement is even and smooth. Make sure that the cable connections follow the movement of the frame and that they do not get squeezed.

If safety stop has been mounted under the wall unit it must be tested as follows:

Press DOWN on control switch and let frame move 2-5 cm / 0,7 - 2in downwards, activate the safety stop. The frame must stop downwards movement and move 1-2 cm / 0,4 - 0,7 in upwards and stop.

If a smart box has been mounted – (is used if a FlexiElectric has been mounted below this unit) follow additional instructions for smart box.

If all test are ok, the frame is ready for use. See section 12 "Safety in use".

9. List of components for Diagonal, L=40-180cm / 15,7 - 70,8 in

#### Diagonal unit 30-40904 to 30-40912:

1 pc.

1 pc.

2 pcs.

2 pcs.

1 pc. 1 pc.

1 pc. 1 pc. 1 pc.

1 pc. 1 pc. 1 pc. 1 pc. 1 pc. 1 pc. 1 pc.

6 pcs

2 pcs.

2 pc.

2 pc.

<i>The Diagonal comprises:</i> 30*40999-098 Mounting profile L=40-120cm/15,7 – 47,2in			
96001037	Distance pieces		
30*40999-170	Tube for top-profile		
Top profile accord 30*40999-354 30*40999-355 30*40999-356 30*40999-357 30*40999-358 30*40999-359 30*40999-360 30*40999-361 30*40999-362	ding to width of cabinets: Top profile, cabinet width 40cm/15,7in Top profile, cabinet width 50cm/19,6in Top profile, cabinet width 60cm/23,6in Top profile, cabinet width 70cm/27,5in Top profile, cabinet width 80cm/31,5in Top profile, cabinet width 90cm/35,4in Top profile, cabinet width 100cm/39,3in Top profile, cabinet width 110cm/43,3 Top profile, cabinet width 120cm/47,2		
	cording to width of cabinets: Bottom profile, cabinet width 40cm/15,7in Bottom profile, cabinet width 50cm/19,6in Bottom profile, cabinet width 60cm/23,6in Bottom profile, cabinet width 70cm/27,5in Bottom profile, cabinet width 80cm/31,5in Bottom profile, cabinet width 90cm/35,4in Bottom profile, cabinet width 100cm/39,3in Bottom profile, cabinet width 110cm/43,3in Bottom profile, cabinet width 120cm/47,2in		
95091442 Screws for profiles Diagonal unit 30-40913 til 30-40918:			

#### The Diagonal comprises:

30\*40999-358

30\*40999-358

30*40999-148	Mounting profile L=130-180cm/51,1-70,8in	1 pc.
96001037	Distance pieces	4 pcs.
30*40999-170	Tube for top-profile	4 pcs.
Top profile accord 30*40999-354 30*40999-355 30*40999-356 30*40999-357	ling to width of cabinets: Top profile, cabinet width 130cm/51,1in Top profile, cabinet width 140cm/55,1in Top profile, cabinet width 150cm/59in Top profile, cabinet width 160cm/63in	2 pc. 2 pc. 2 pc. 2 pc. 2 pc.

Top profile, cabinet width 170cm/66,9in

Top profile, cabinet width 180cm/70,8in




11/177







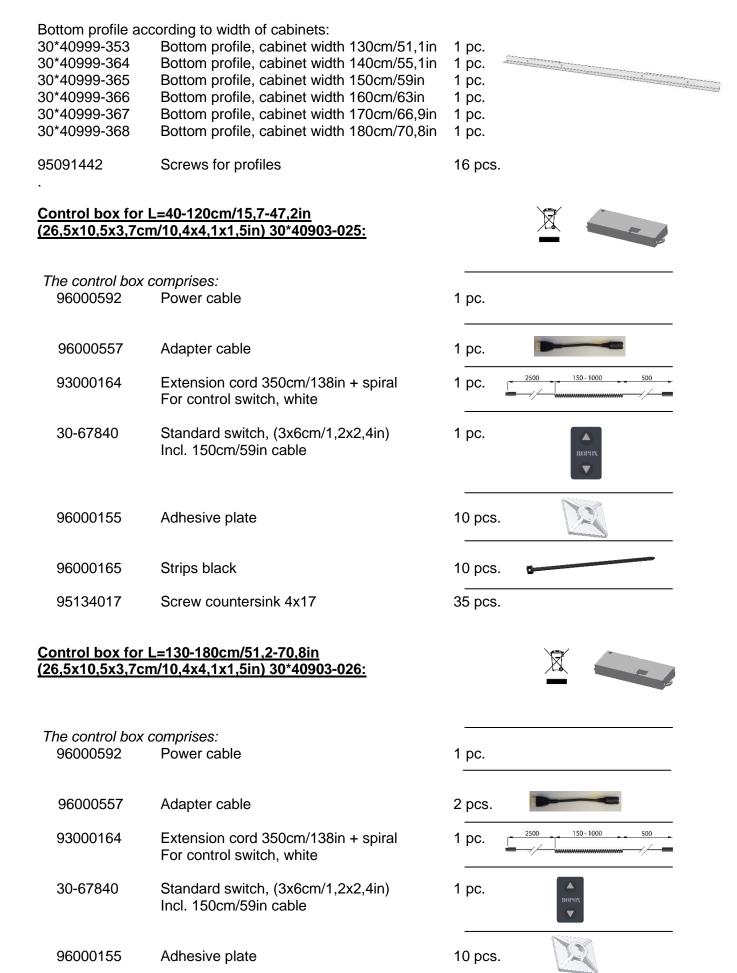

96000165	Strips black	10 pcs.	6
95134017	Screw countersink 4x17	35 pcs.	

# 10. Options for Diagonal


<u>Split cable 96000629:</u> To be used if more than one control	1 pc.	$\sim$
Large press pad (14x7cm/5,5x2,7in) 30-67841: For disabled people, incl. 150cm/59in cable	1 pc.	
Safety stop plate 30-416xx: Safety circle with extension cable (cable 250cm/98in) Steel plate with mounting parts and plate.	1 pc.	
<u>Smartbox 30-69002-1:</u> Option for safety stop plate To be ordered if Diagonal is mounted above a FlexiElectric	1 pc.	and the set of the set Here and the set Here a
Extension cord 30-67870: 250cm/98in (black) for safety stop	1 pc.	
<u>Spiral extension 30-67871:</u> L =25-100cm/9,8-39,3in (black) for safety stop	1 pc.	
Bluetooth Adapter for Ropox Connect App 30-90591: Option, if the product needs to be remotely controlled via the Ropox Connect App	1 pc.	


# Incl. cables 30-42604 Led for 40cm/15,7in Safety stop 1pc 30-42605 Led for 50cm/19,7in Safety stop 1pc 30-42606 Led for 60cm/23,6in Safety stop 1pc 30-42607 Led for 70cm/27,5in Safety stop 2pc 30-42608 Led for 80cm/31,5in Safety stop 2pc 30-42609 Led for 90cm/35,4in Safety stop 2pc 30-42610 Led for 100cm/39,3in Safety stop 2pc 30-42611 Led for 110cm/43,3in Safety stop 2pc 30-42612 Led for 120cm/47,2in Safety stop 2pc 30-42613 Led for 130cm/51,2in Safety stop 3pc 30-42614 Led for 140cm/55,1in Safety stop 4pc 30-42615 Led for 150cm/59in Safety stop 4pc 30-42616 Led for 160cm/63in Safety stop 4pc 30-42617 Led for 170cm/66,9in Safety stop 4pc 30-42618 Led for 180cm/70,8in Safety stop 4pc

Safety stop with LED spot 1W Incl. transformer.

# 11. Placing of LED spot

If LED spots are purchased, they will be positioned as shown on drawing below.

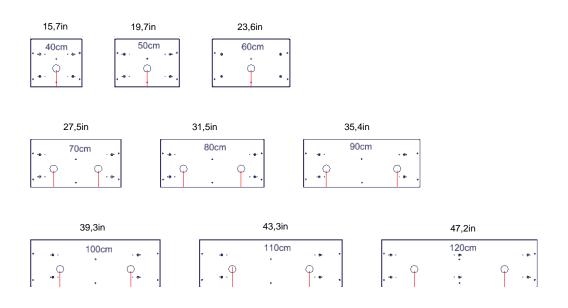

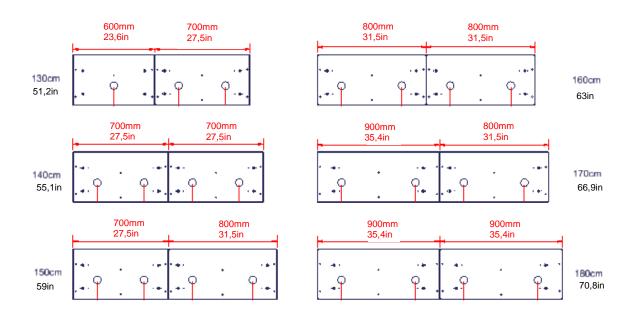

# 12. Safety in use

- The Diagonal should only be used by people who have read and understood these instructions..
- The Diagonal is a height-adjustable frame, and we recommend the filling of a safety stop beneath in order to guard against trapping. Even though a safety stop plate has been mounted always make sure that there are no people, animals or objects under the frame during height adjustment.
- > The Diagonal is a wall unit and must not be used as lifting equipment or person lifter.
- Always use the frame so there is no risk of damage to people or property. The person, who operates the frame, is responsible for avoiding damage or injury.
- Children and people with reduced observation must only operate the frame under surveillance.
- If the frame is used in publicly accessible locations where children or people with reduced observation ability may get close to the frame, the person operating the frame must pay sufficient attention in order to prevent dangerous situations.
- > Make sure that there is free space above and below the frame to allow height adjustment.
- > Do not overload the frame and make sure that the load distribution is correct.
- > Do not operate the frame in case of errors or damage.
- > Do not use the frame in an explosive environment.
- In case of inspections, service and repairs make sure the frame is not loaded, and that main power is disconnected
- Any modification to the frame, which may influence its operation or construction, is forbidden
- Installation, service and repairs must only be carried out by competent personnel.
- If the frame has not been installed in accordance with these mounting instructions, the guarantee may become void.
- Only use Ropox original spare parts as replacement parts. If other spare parts are used, the guarantee may become void.

# 13. Cleaning and maintenance

#### 13.1 Cleaning of frame



The frame must not be connected to the mains voltage during cleaning. Do not flush electrical components with water



Clean the trame with a damp cloth using lukewarm water and ordinary cleaning agents. Do not use Corrosive/abrasive liquids or abrasive cloths, brushes or sponges.

After cleaning dry the frame.

#### 13.2 Maintenance



Inspections, service and repairs must be carried out by competent personnel

The frame is maintenance-free and moving parts have been lubricated for life. For reasons of safety and reliability we recommend inspection of the frame once a year:


- > Check that all bolts have been tightened.
- > Check that the frame moves freely from bottom to top position without problems.
- > Check that all cables have been fitted correctly and are undamaged.

After each inspection the service schedule shall be filled in.

Only use Ropox original spare parts as replacement parts. If other parts are used, the guarantee may become void

#### 13.3 Label

Each unit has its own label as illustrated. This label contains important information to be stated in connection with inquiries to the supplier. The label is located as illustrated.

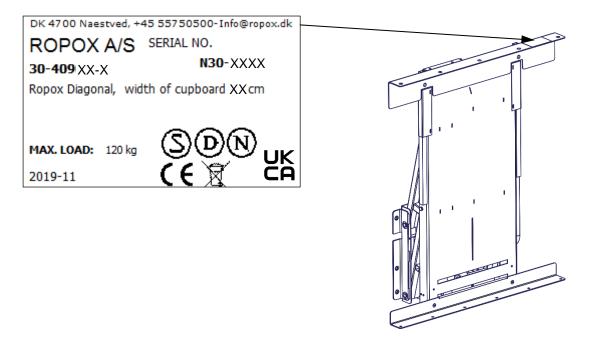

# Symbols:

CE

UK

The user manual must be read before use.

Waste must be sorted - if possible, for recycling.

Ropox is the manufacturer of the product in question.

CE marking. This product meets the requirements for General Safety and Functionality of the current EU Directives and Standards.

UKCA marking. This product meets the requirements for General Safety and Functionality of the current UK Directives and Standards.

#### 13.4 Service schedule, operation and maintenance

Service and maintenance	
Serial No.:	

Date:

Sign:

**Remarks:** 

Service and maintenance Serial No.:

Date:

Sign:

**Remarks:** 

Service and maintenance Serial No.:

Date:

Sign:

Remarks:

Service and maintenance Serial No.:

Date:

Sign:

Remarks:

## 14. Trouble shooting

#### 14.1 The height of the frame cannot be adjusted

- 1. Check mains and voltage to the control unit and that the power is switched on.
- 2. Check cables and plug-in connections between control box and motors.
- 3. Check cables and plug-in connections between control box and control switch.
- 4. Check cable connections between safety stop plate, and control box. (see mounting instructions).
- 5. Check that safety stop has not been activated.

#### **Reset procedure**

- 6 If none of the above has helped reset the unit:
  - Remove all load from cabinet
  - Disconnect motor cables
  - Disconnect mains cable and wait 5 sec
  - Connect mains cable
  - Connect motor cable
  - Move the unit to bottom position for reset.

# 15. Complaints

We refer to our general Terms of sale and delivery on our homepage www.ropox.com

# Keep this folder with product at all times



Ringstedgade 221 DK – 4700 Naestved Tel.: +45 55 75 05 00 Fax.: +45 55 75 05 50 E-mail: <u>info@ropox.dk</u> <u>www.ropox.com</u>