



# Domino Slings




## User manual

**This manual should always be in close proximity of the product**


## Table of content

<b>1. Symbols used in this manual .....</b>	<b>3</b>
<b>2. General safety .....</b>	<b>3</b>
2.1 Product Unit label .....	5
<b>3. General requirements .....</b>	<b>6</b>
3.1 Product information .....	6
3.2 Product description .....	6
3.3 Intended purpose .....	6
3.4 Intended operator .....	6
3.5 Contraindications .....	6
3.6 Essential performance .....	7
3.7 Clinical functions .....	7
3.8 Complaints and adverse events .....	7
<b>4. Instructions for use .....</b>	<b>7</b>
4.1 Operating the product .....	7
4.2 Model/Type .....	8
4.3 Size guide for Domino slings .....	9
4.3.1 Size information .....	9
4.3.2 Dimensions for slings .....	9
4.4 Slings used with Stand-up hoist .....	10
4.5 Transfer from chair .....	10
4.6 Color coding on the straps .....	11
<b>5. Cleaning .....</b>	<b>12</b>
<b>6. Maintenance .....</b>	<b>12</b>
6.1 Periodic maintenance .....	12
6.2 Daily maintenance .....	12
6.3 Monthly maintenance .....	12
<b>7. Storage .....</b>	<b>13</b>
<b>8. Environmental protection .....</b>	<b>13</b>

# 1. Symbols used in this manual

<b>Warning Symbol</b>  Indication of potentially hazardous situation. If not avoided it can result in serious injury or death.	
<b>Caution Symbol</b>  Indication of potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices.	
<b>Notification Symbol</b>  This symbol is used to notify correct use and handling of the product.	

## 2. General safety

	<p>This manual must be read and understood before use. Always keep this manual in close proximity of the product.</p> <p>The use, installation and service of this product must be in compliance with this manual to avoid accidents and serious personal injury.</p> <p>Never use or handle this product in other ways specified in this manual as it can result in personal safety hazards and/or cause damage to the product.</p> <p>Persons installing and/or using this product either as operator or user should have the necessary safety information and access to this manual.</p>
---	---



Do not perform repairs, disassembly or assembly operations, add-ons, re-adjustments or modification of the product beyond what is described in this manual. These must be carried out by Ropox or Ropox authorized personnel. Do not perform service while in use.

This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.

Do not use the product if it has defects or have become damaged before being repaired or replaced.



The information in this manual is based on correct installation in accordance with installation instructions for this product.

Ropox cannot be held liable if the product is used in any way that differs from stated in this manual and/or installation instruction.

Ropox reserve the right to amend this manual and reference documents without prior notice.

## 2.1 Product Unit label

All slings are equipped with a “Quick-Guide” and “Facts-Label” on the reverse side.

The “Quick-Guide” enables you to determine quickly what is up and down as well as in and out on the sling.

Furthermore, the safe load and washing instructions are also indicated.



It is also possible to write the name of the client or department on the sling and date when it was first taken into use.



On the “Facts-Label” is it easy to see which size, type/name and the time of production (year and quarter).

In general, there are two to four sizes of slings, small, medium, large and X-large.

The label is also used to determine the size to be used for a person based on the person’s height and weight. These indications are only intended as a guide as the individual anatomy of a client can result in another categorization of size.

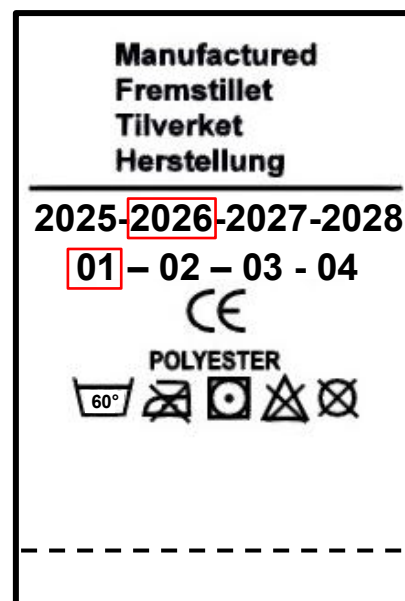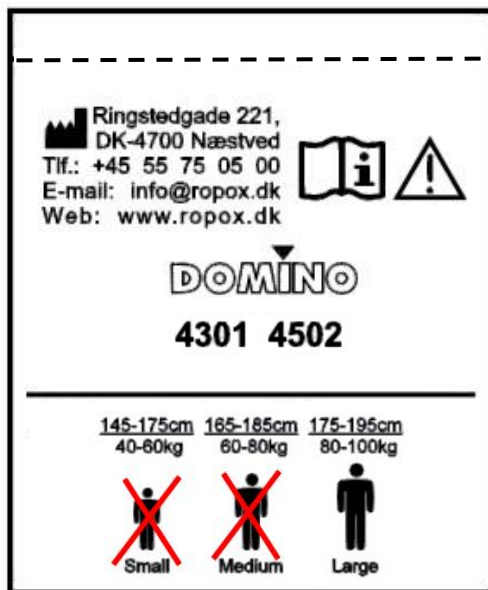

## 3. General requirements

### 3.1 Product information

Manufacturer	Ropox A/S Ringstedgade 221, DK-4700 Naestved +45 55 75 05 00 <a href="mailto:Info@ropox.com">Info@ropox.com</a>			
Product models	Part number	Model	Configuration	UDI
	25-43012	Thorax (with padding)	M	5707581001017
	25-43013		L	
Expected service life	2 years			
MDR Class 2017/745	Class I			
MDD Class 93/68/EEC				
Intended environments	This product is to be used only in:  Professional Healthcare Environment Homecare Environment			
Maximum user weight according to DS/EN ISO 10535:2007	200kg			
Ambient temperature	5-40C° Operation and storage			
Materials in contact with patient	Sling		Polyester	

### 3.2 Product description

Domino slings are produced of a polyester material which gives strength and comfort. The material has an emery finish on the inside. This gives a cotton-like surface which ensures that the sling does not slide and at the same time is comfortable for the user. The material is fire-retardant.

### 3.3 Intended purpose

Domino slings can be used on Ropox All-in-One hoists. They find use in situations, where a patient is to be transferred from one stationary bed onto another. The product must not be used for patient transport.

### 3.4 Intended operator

Intended operator should be a health professional, educated in the proper use of such products, in addition to having read and understood these instructions for use. Children, patients, or lay people are not considered intended operators.

### 3.5 Contraindications

If the patient does not have the upper body strength to sit in the sling (with or without holding onto the handles), they have a higher risk of slipping out of the sling. This could lead to a fall.

### 3.6 Essential performance

The device does not have any function related to basic safety or essential performance.

### 3.7 Clinical functions

This product does not offer clinical benefit for patients, when used as intended by the manufacturer.

### 3.8 Complaints and adverse events

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is located.

**In case of inquiries concerning slings, please state the following information:**

Customer: \_\_\_\_\_

Sling model/size: \_\_\_\_\_


Sling number: \_\_\_\_\_

Year and month: \_\_\_\_\_

## 4. Instructions for use


### 4.1 Operating the product

It is important for the user to be familiar with the operation of the slings and its facilities, and that it works without problems. Therefore, in order to obtain optimum safety, this user Manual must be read carefully before use.

	<p><b>Caution!</b></p> <p>Try the lift on yourself, before you lift another person. Before use - Carry out a risk assessment to ensure that the correct size, type and shape of body-support unit are being used for the patient.</p>
---	---

Before you start to lift a client go through the following check list:

1. Determine proper sling size by using the guide- and facts labels which is on all slings.
2. Inspect selected sling for damage. Do not use the sling if damaged or worn. Special attention should be given to lifting straps.
3. Make sure you are familiar with the hoist you are going to use, and that it is working properly.

	<p><b>Caution!</b></p> <p>Be aware of any client apprehension towards the lift. Some people new to mechanical lifting may be quite uneasy or become distressed. Therefore, always inform the client that he/she is going to be lifted and about the procedure. Reassure the client during the transfer.</p>
---	---

## 4.2 Model/Type



**Domino Thorax**, available in four sizes

Only for stand-up hoist. Unique design and padding.  
Provides optimum support for stand-up lifting. Can also be  
supplied without padding under the arms.



### **Warning!**



Domino slings can lift up to 200kg - If the client to be lifted exceeds this weight, do not use our standard slings.



### 4.3 Size guide for Domino slings


In choosing the correct sling to use, the “Facts-Label” on the sling can be used. Alternatively, the following instructions can be used. This section can also be used to determine which slings to choose, when acquiring the product. It's recommended to use the Domino slings exclusively with the Ropox All-in-One hoist.

#### 4.3.1 Size information

Type	Domino Thorax (w/ padding)	Domino Thorax (w/ padding)
Height	165 – 185cm	175 – 195cm
Weight	60 – 80kg	80 – 100kg
Size	 Medium	 Large

#### 4.3.2 Dimensions for slings

Different slings come in two different sizes. For these, the waist size (W), are provided in order to, together with height and weight, choose the best sling for the patient. The waist size is indicated as the **maximum size**. The dimensions are as follows:

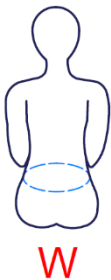

Type	Size	W
Thorax (w/ padding)	Medium	100 cm
	Large	130 cm

## 4.4 Slings used with Stand-up hoist

**Domino Thorax** sling - only for transfer with stand-up hoist

1. Transfers with Stand-up hoist must only be performed on clients to and from sitting positions.
2. The Stand-up hoist should be braked during lifting and lowering.
3. Clients should not be dressed in slippery clothes or underwear.
4. Clients must be able to bear weight on at least one leg.

## 4.5 Transfer from chair

1. Apply Thorax lift sling firmly to very firmly around the waist/lower back of client.
2. Increase the width between the legs of the hoist and move it close up to chair.
3. Place the feet of the client on the standing platform.
4. Move the hoist so close to the client as possible and brake the hoist.
5. Adjust the knee support so that the kneepad is just below knees.
6. Lower the lifting arm.
7. Place the lifting straps in hooks of the fork shaped lifting bar. Client's arms can be inside or outside the sling, and they can hold on to the stand-up bar depending on their ability (not crucial to hold on if impossible to do).
8. Encourage the client to look up and lean back.
9. Lift the client a few centimetres above the seat - STOP - make sure that all straps are securely attached to the hooks on the lifting arm and if necessary, adjust the sling.
10. Bring the client into standing position. We recommend lifting the client 3/4 of the way i.e., with knees slightly bent. Support the client if necessary.
11. In order to prevent sling from riding up, the two lifting straps can be passed through an extension loop.



### **Warning!**

For Stand-up hoist: Please observe that when you lift or lower your client the wheels on the hoist must be locked, and on the wheelchair they must be locked.

## 4.6 Color coding on the straps

The loops on the straps have color coding, which makes it easier to select the correct position.

There are three different loops/colors options on the shoulder and leg straps, which allow the sitting position to be altered so that lift is as comfortable as possible.

It is important that the same color code is used on both sides e.g. on the shoulder straps. It may differ between shoulder and leg straps.

The positions when using the three different colors are shown on the illustrations below. They are only intended as a guide, as the sitting angle is a combination of the adjustment of both the legs and shoulder strap loops and differs from client to client.




**Grey** - Reclining  
Outermost suspension



**Green** – Semi reclining  
Middle suspension



**Red** – Sitting upright  
Innermost suspension

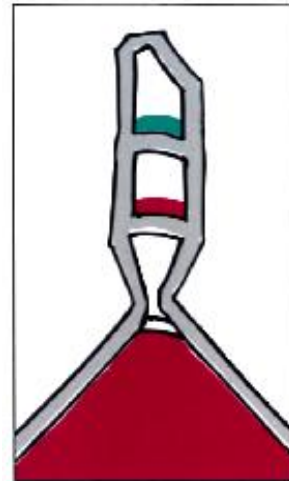

### **Warning!**

Always check the condition of the sling before use.

## 5. Cleaning


Wash the sling in warm water at up to 60° with a mild detergent, never use solvents.


Ensure that any cleaning agents are removed by rinsing thoroughly in warm water.

Drying is best done naturally at room temperature. It takes about an hour.


Drying can also be done in tumble dryer at low warmth.

Wash separately the first time to avoid discoloration.

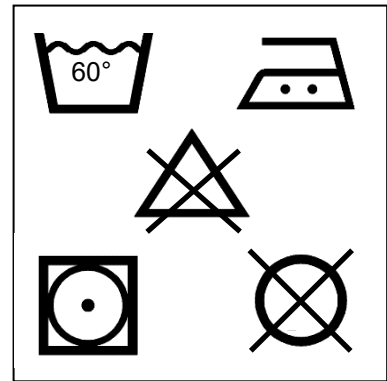

	<p><b>Notification!</b></p> <p>Do not use cleaning agents containing abrasive e.g. scouring powder, steel wool, scouring sponge.</p> <p>This product is <u>not</u> designed to be sterilized, autoclaving and sterilization beyond normal cleaning as this may possibly change the product safety and function.</p>
---	---

## 6. Maintenance

	<p><b>Notification!</b></p> <p>Failing to comply with periodic maintenance may result of degrading of product function and safety.</p>
--	--

### 6.1 Periodic maintenance

Domino slings should be checked for signs of wear or fatigue before each use. Special attention should be given to the loops on the lifting straps and the stitching at the junction between lifting strap and the sling.

A damaged or worn sling should not be used. Even minor damage can quickly become worse, and a dangerous situation can occur. Therefore, is it important to check the sling before a lifting job.

If there is any doubt, take the sling aside for a closer inspection

A detailed and overall visual inspection of the slings should be carried out on a regular basis, preferably every month.


### 6.2 Daily maintenance

- Check for wear and that the seams are intact and not frayed. A worn sling must be replaced.
- Check that the slings are clean, if not, have them washed.

### 6.3 Monthly maintenance

The conditions mentioned below are potentially dangerous and must lead to instant discontinuing use of the product:

- Cuts, holes, or burns in the webbing
- Chemical attack
- Broken or worn stitch patterns
- Abrasion
- Buckling of the locking/unlocking mechanism

	<p><b>Warning!</b></p> <p>If in doubt about the condition of a sling, do not use it.</p>
---	--

## 7. Storage

Do not store or place Domino Slings in strong and direct sunlight or in excessive heat or humidity, as this can shorten the lifetime of Domino Slings.

The best way to store Domino Slings is simply to hang them in the straps on a wall-mounted hook. Do not fold Domino Slings in the foam-padded sections.

## 8. Environmental protection

The product is not intended to be disposed as municipal waste. Proper disassembly, sorting and disposal of components must be done by waste disposal professionals.

