GRIPO

BRUKSANVISNING

NORSK

ENGLISH

SVENSKA

DANSK

DEUTSCH

CE

HEPRO

Forord

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Notater

Foreword

Gripo is a sturdy and flexible support tool that provides safety for getting up and moving around the home. Gripo fits most homes with ceiling heights between 210-300 cm and is very easy to install. For ceiling heights lower than 210cm and higher than 300cm, hepro offers Gripo Spesial and Gripo Lang. The support bar works great on its own, but can also be complemented with useful accessories.

With Gripo, we combine quality and the environment. The product is made of recyclable materials that are gentle on the skin and resistant to corrosion, which also makes it ideal for use in wet rooms and humid environments. At Hepro, we think holistically about the environment. That is why Gripo is delivered in recyclable, environmentally friendly packaging marked with Grønt Punkt – a conscious choice to protect both people and nature.

The pole is tested and approved according to the relevant parts of EN 12182:2012 and declared compliant to EN 21856:2022. It is made of aluminium and plastic and is designed to be clamped between floor and ceiling in any room where the ceiling and floor are strong enough to withstand a point load of 100 kg.

The pole is quickly and easily installed without screws or tools, and securely fixed between the floor and ceiling. With a wide range of accessories, it can be adapted to different user needs. Hepro is always working to develop new accessories based on user needs and now offers new bed rails and new accessories for weaker roofs.

Hepro is a manufacturer and can therefore offer customized solutions when needed. Hepro is a Norwegian industrial company that develops, manufactures and sells aids for the elderly and disabled. The company was established in 1987 and currently has around 85 employees. The company's head office and factory are located at Rognan in Nordland. Hepro AS, Tømmerveien 10, 8250 Rognan.

IMPORTANT!


**Read the user manual carefully before
using the pole**

Gripo

Item no. 35200 Gripo

Item no. 35210 Gripo Lang

Item no. 35202 Gripo Spesial

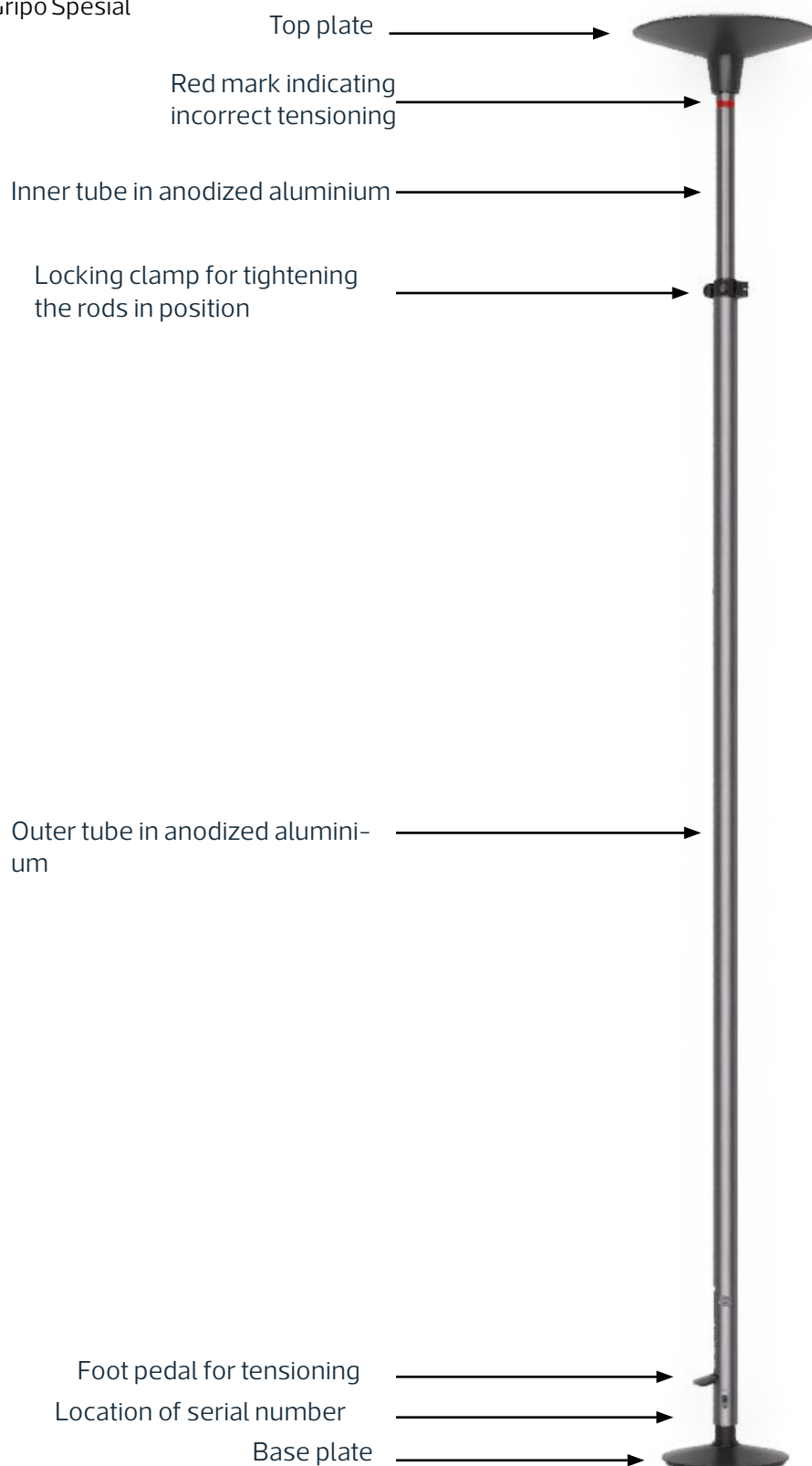Safety and security

This product is intended for use in private homes or institutions.

- Do not use the product if the red safety ring at the top of the pole is visible.
- Always consider the risk of crushing between the pole and other furniture/objects. For more information, read points 4 and 5, pages 9-10 in this manual
- Do not mount the product on loose objects that are not securely fastened to the ceiling or floor, such as floor mats, plates or similar.
- Do not allow children to play with the product.
- Do not place the product on surfaces that are not flat or parallel.
- When installing in wet zones, use anti-slip protection, Item no. 35007
- Do not modify or alter the product in any way that is not described in this manual.
- Do not use this product with other accessories that do not belong to Gripo/the product
- Gripo can be installed in a shower, but the product should not be permanently immersed in water or other liquids.
- Be careful when opening the packaging so that neither you nor the product is damaged.

GRIPO CONSISTS OF:

1 x Base plate (the smallest)
1 x Top plate (the largest)
1 x Pole
1 x Gas spring
1 x User manual

Item no.	Description	Ceiling height
35200	Gripo	210-300 cm
35210*	Gripo Lang	260-340 cm
35202	Gripo Spesial	Lower than 210cm

* In addition to extra height, this version has a 15 cm longer outer tube, which is suitable for those who need to have accessories attached high up on the pole, or who want to use the pole for training/activity.

Contents

Foreword	2
Gripo	3
Safety	4
Contents	5
1 Explanation of symbols	6
2 Package contents	6
3 Environment	7
4 Risk assessment of distances during installation	9
5 Safety during installation	10
6 Instructions for use	11
7 Installation	13
8 Dismantling	17
9 Maintenance routines	17
10 Guarantee	17
11 Revisions to this manual	18

1 Explanation of symbols

Manufacturer

Serial number
CE marking

Warning notice


Medical equipment

Production date

Read the instructions carefully before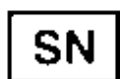

use

2 Contents of the package

ROD

TOP PLATE / BOTTOM
PLATE

GAS SPRING

3 Environment

3.1 MATERIALS AND RECYCLING

Gripo is recycled according to local regulations, please contact your local recycling station for more information.

When Gripo has reached the end of its service life, sorting the components is quick and efficient.

Share	Materials	Sorted as
Aluminium profiles (anodized)	Aluminium	Metal
Bottom and top plate, as well as other plastic details	PA6	Plastic
Gas spring	Stainless steel	Metal
Foot pump	Aluminium	Metal
Fasteners (screws, nuts, washers, sleeves)	Steel, copper, aluminium	Metal

Long life and quality:

During the development of Gripo, we have placed great emphasis on choosing high-quality materials. The product is designed with durable materials and a robust construction. Gripo has been tested by Hepro and users for over 20 years for maximum performance and longevity. This reduces the need for frequent replacement, helping to save resources and promote more sustainable use – to the benefit of both you and the

environment.

Repair and reuse:

The design allows for the replacement of parts. We offer spare parts during the contract period and five years after, which reduces waste.

ALUMINUM POLE:

Gripo's anodized aluminium rods are 100% recyclable and can be recycled endlessly without loss of quality. Recycling aluminium requires only 5% of the energy needed to produce new aluminium, making it a highly sustainable choice.

The anodizing provides a hard, durable surface that protects against both scratches and corrosion, making the bars ideal for demanding environments, including optimal use in wet rooms. The smooth, glossy finish makes the material easy to clean, while ensuring long durability and optimal functionality.

PLASTIC PARTS IN PA6:

The plastic parts we use are made of PA6, a highly durable material, tested by Hepro for maximum performance and longevity. The material can withstand both high and low temperatures, even under heavy loads. The material is both strong and durable as well as easy to clean, ensuring good hygiene throughout the life of the product. The material is a sustainable choice, combining high quality, reliability and environmental friendliness in one material.

4 Risk assessment of distances during installation


Always consider the risk of crushing between Gripo and other furniture/items. Weather pay special attention to the distance between bar and other furniture, and assess the distance so that the user cannot become trapped between Gripo and the furniture.

To avoid the risk of crushing, the installation of Gripo is carried out with the correct safety distance to objects used together and in connection with Gripo. Critical areas for crush hazard are torso, neck and head. Ensure that the "Recommended distances between objects" are observed. These distances are essential to ensure safe use and prevent serious injuries.

RECOMMENDED DISTANCES BETWEEN OBJECTS

To avoid the risk of crushing:	Recommended dental care for adults	Recommended dental care for children
Head / Neck / Torso	Less than 120 mm or more than 300 mm	Less than 60 mm or more than 300 mm
Feet	Less than 35 mm or more than 120 mm	Less than 25 mm or more than 120 mm

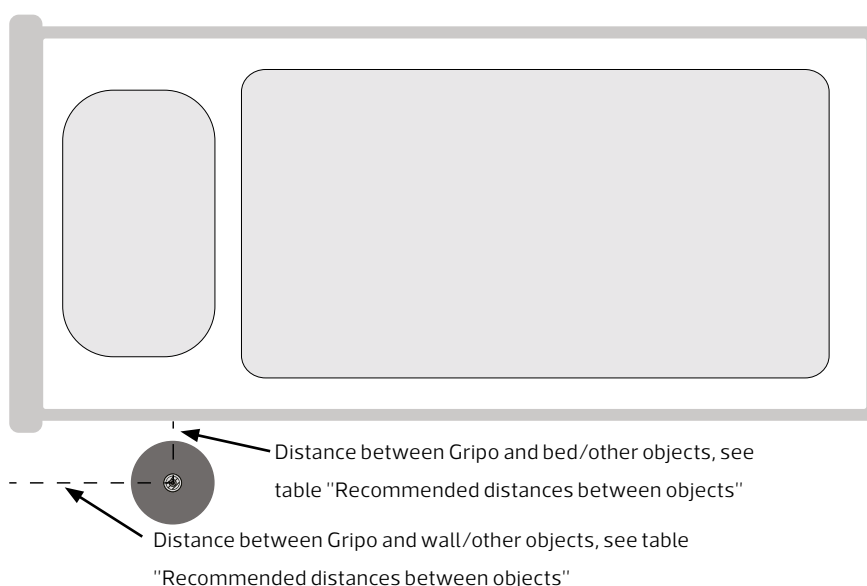
5 Safety during installation

Personnel installing Gripo must pay attention to the following:

To avoid cases where the user can get stuck between Gripo and other objects, make sure to follow the recommended distances.


Please note that adjustable and/or movable objects may cause the distance between Gripo and the object to change. Examples of movable and/or adjustable objects are lifting chairs or beds that can be raised and lowered or rolled.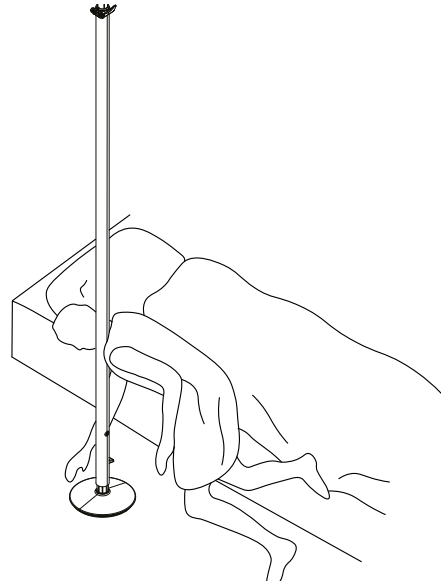

Note that Gripo is a sturdy and flexible support tool that provides security when standing up and moving around the home. If an aid is needed to prevent the user from falling out of bed, accessories suitable for this purpose should be used and not the Gripo pole itself.

Always consider the risk of crushing between Gripo and other furniture during installation.

To prevent the risk of crushing between the bed/chair and Gripo, a circular grip D30, or similar accessory, can be fitted as shown in the illustrations.

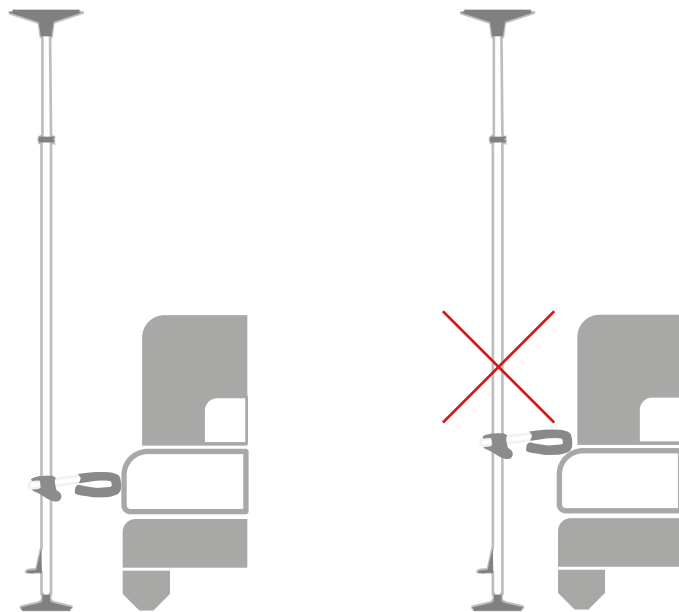6 Instructions for use

DEVELOPED AND MANUFACTURED IN ACCORDANCE WITH:

- MDR 2017/745
- Relevant parts of EN 12182:2012 in accordance with EN 21856:2022

OPERATING ENVIRONMENT

This product is intended for use in private homes or institutions. The Gripo pole should be at room temperature before it is set up.

USER WEIGHT

Maximum user weight 210 kg.

DIMENSIONS GRIPO

Weight	4.7kg
Height	210cm to 300cm
Width top plate	29cm

TRANSPORT

During transportation, the product must be secured so that it cannot endanger passengers or other objects.

INTENDED USE

Gripo support bar is versatile and can be used in various situations where you need support when standing up. Whether it's in the bathroom, at the bedside, in the hallway or anywhere else, Gripo will be there to support you. The support bar works great on its own, but can also be complemented with useful accessories to suit your individual needs.

6.1 INSTALLING GRIPO AGAINST THE ROOF

Before installing Gripo, the roof structure should be examined with regard to strength and the possibility of placing the pole in the desired location. Ceilings and floors must be strong enough to withstand a point load of 100 kg.

For plasterboard ceilings, it is very important that Gripo is installed as shown in the sketch. Avoid placing Gripo between roof beams and on chipboard roofs. When installing Gripo on plasterboard ceilings or other weak ceilings, it is recommended to use accessories for weak ceilings that provide extra reinforcement between the top plate of the pole and the ceiling.

Not like this

Use a large roof panel

Like this

7 Installation

1

Before you start with assembly.

1. Place the rod on the small base plate so that the rod fits in the groove.
2. Place the gas spring on top of the inner tube and push the gas spring towards the red safety mark.
3. Attach the large top plate to the top of the gas spring so that the rod fits in the groove.


The force acting on the ceiling is 100 kg


1

2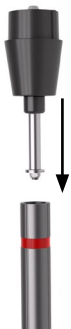

3

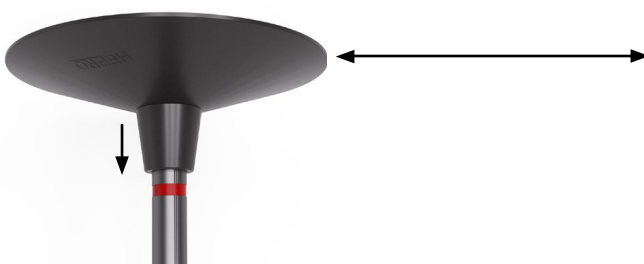2


Place the pole in a suitable location.

1. Pull out the foot pedal.
2. Open the locking clamp.

1

2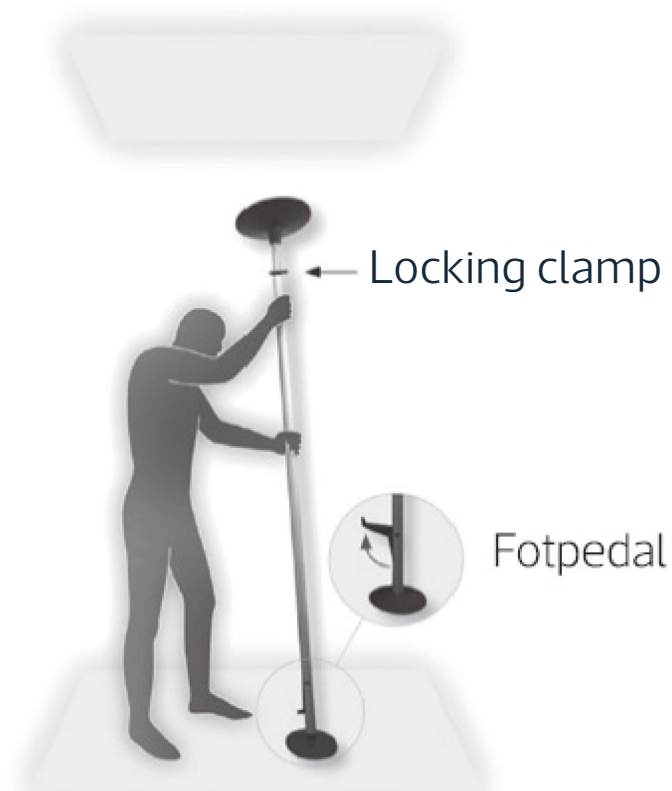

3

Push the inner tube up until the top plate reaches the ceiling, while still keeping the bottom plate in contact with the floor.

Ensure that the pole is vertical.

4

Rotate the inner tube clockwise so that the inner tube locks into the outer tube.

5

Lock locking clamp.
NB: Make sure the screw on the locking clamp is tightened.

⑥

Press down on the foot pedal. If you are unsure whether the roof can withstand the load, make sure that the roof does not crack while gently depressing the foot pedal.

Check that the red marking is gone.
If the red marking is visible, dismantle the rod and start from point 2.

Make sure that the red marking is not visible during use.

If it comes out – take the bar down and put it back up again.

Check that the tightening screw on the locking clamp is not loose and that the inner tube is screwed on securely. If the red mark is still visible, one possible reason may be that the roof structure is not strong enough.

8 Dismantling

Release the locking clamp and pull up the foot pedal. If necessary, turn the inner tube anti-clockwise.

9 Maintenance routines

9.1 CLEANING

Wash the product with water at max. 85 °C for a minimum of 3 minutes, or disinfect chemically with an approved disinfectant. Some disinfectants may discolour the product, but will not affect its function.

9.2 REUSE

Gripo is designed for a long service life and can therefore be reused. In order to reuse Gripo, the top and bottom plates and gas springs must be replaced. In addition, the aluminium rods must be disinfected inside and out. Gripo is then ready for a new user with minimal maintenance.

10 Guarantee

Under normal use in accordance with the operating instructions, Gripo has a 5-year service life and 5-year warranty. The warranty period for accessories is 2 years. In the event of complaints or inquiries about the product, the product's serial number must always be quoted as a reference.

In case of product faults, please contact

Hepro, telephone number: +47 46 74 55 55.

11 Revisions to this manual

Auditing	The date	Author	Note
1.0	20 Feb 2025	Anders Gloslie	Established
2.0	22 May 2025	Alexandra Parti	Combined all languages in one document
3.0	06 Feb 2026	June Nanette Hansen	Changed text regarding red mark

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