


ROPOX™

BETTER WAYS TO BETTER DAYS

- A PART OF AddLife®

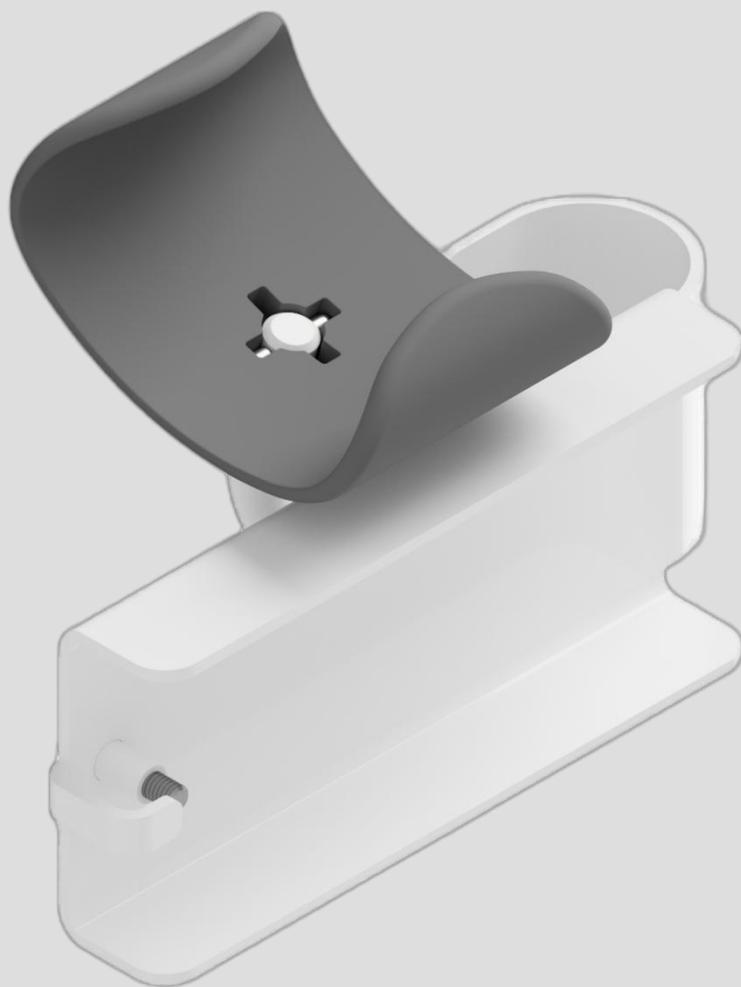

Control Holder for **Toilet Lifter**

User Guide

40-40912

Table of Content

- Application 3
- Component Parts List..... 3
- Right or left handed 3
- Mounting Instructions 4

Application

The Control Holder for Ropox Toilet Lifter is an accessory, which is designed to hold the remote control in place, to make it handier to use while on the toilet. The Control Holder is easy to mount, and has the ability to lock the remote control in place using a locking screw. The remote control can be removed from the control holder freely, if the locking screw isn't used, should it be necessary to activate the Toilet Lifter remotely.

Component Parts List




1x Control Holder
40-40912

Includes
INSEX screw M4x16
95010416




Right or left handed

The Control Holder is supplied as standard for mounting on the left side, but can be reconfigured to make it fit on the right side instead. This is done by loosening the screw in the bushing, and turning the grip part around 180°. The screw is then re-fastened. Use a 2mm Allen key for this (not supplied by Ropox).

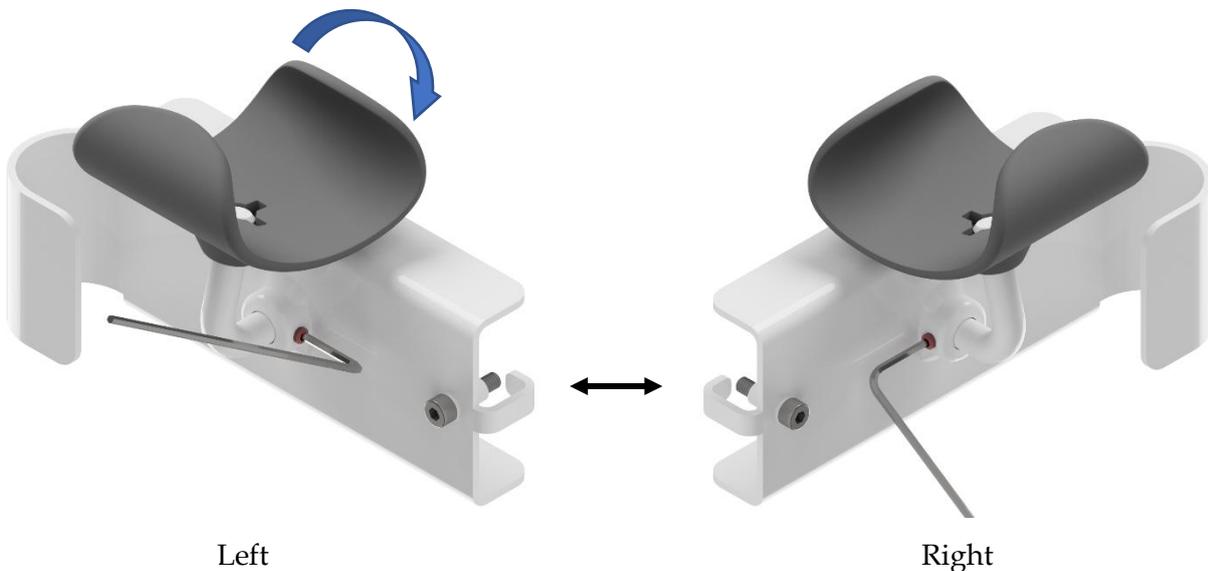


Mounting Instructions


- a) Determine the best placement for the Control Holder relative to the user. The Control Holder can be placed anywhere along the entire length of the support arm.

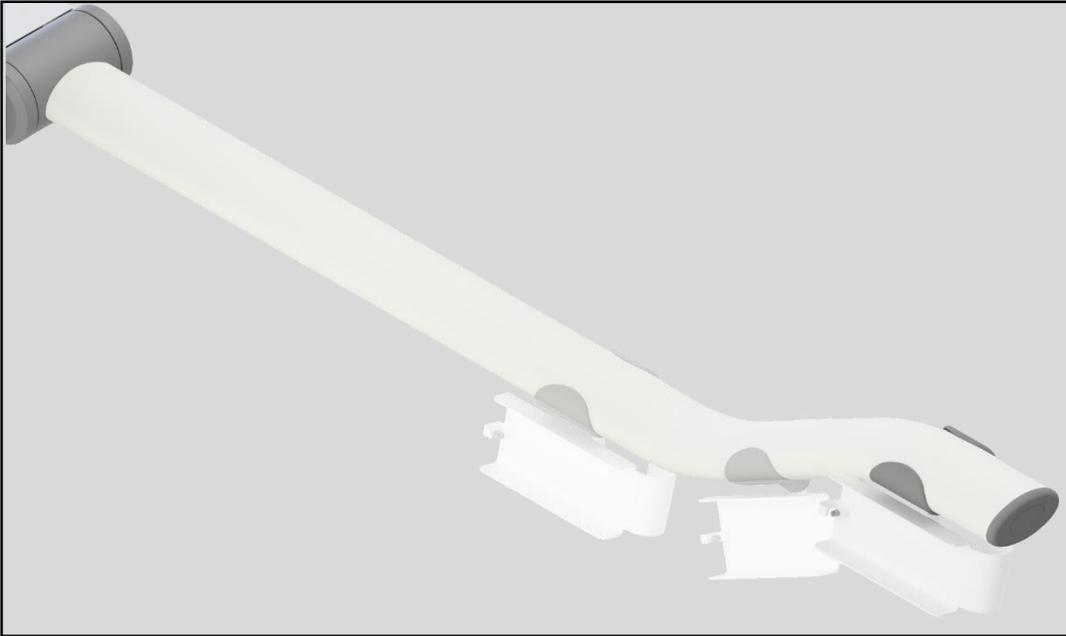


- b) Remove the protective paper from the tape on the grip, and click the grip onto the support arm.

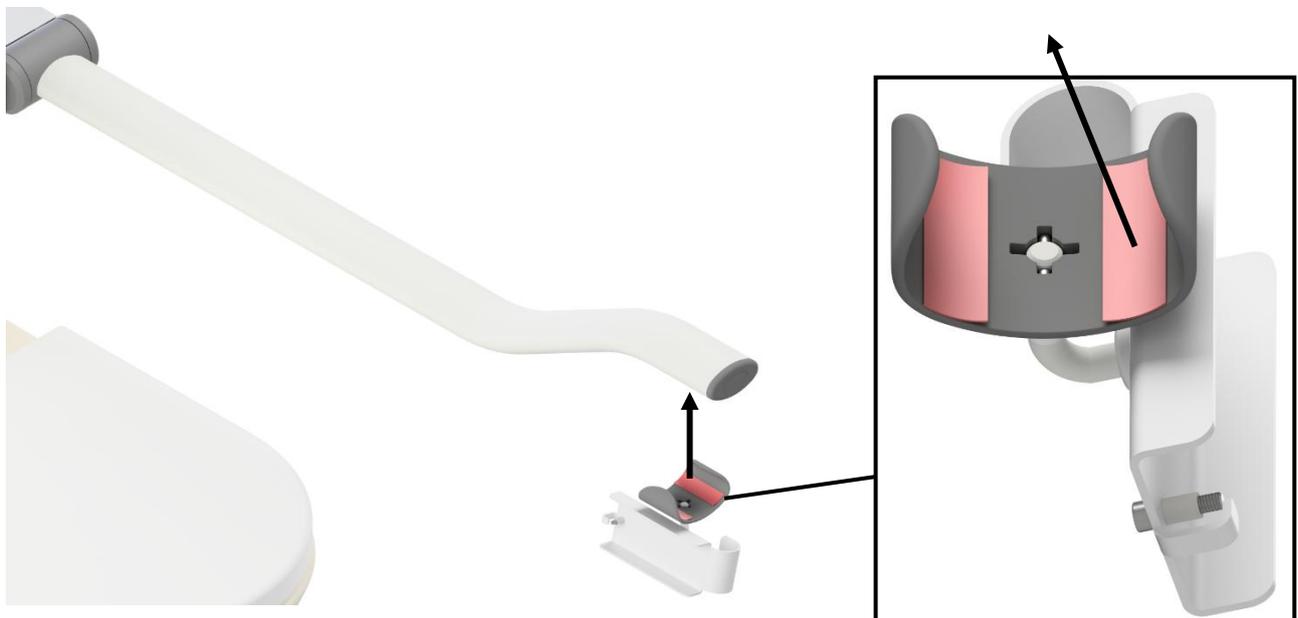

- c) Unscrew the locking screw from the Control Holder, so the remote control can be mounted on it. This is only fastened by hand. Don't throw away the screw yet!

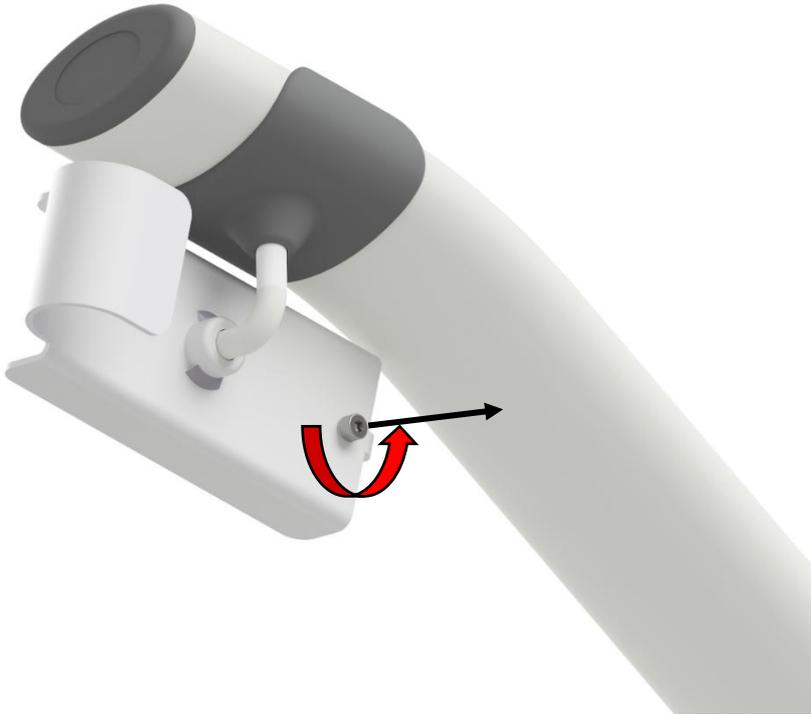

- d) The remote control can now be mounted or dismounted freely.

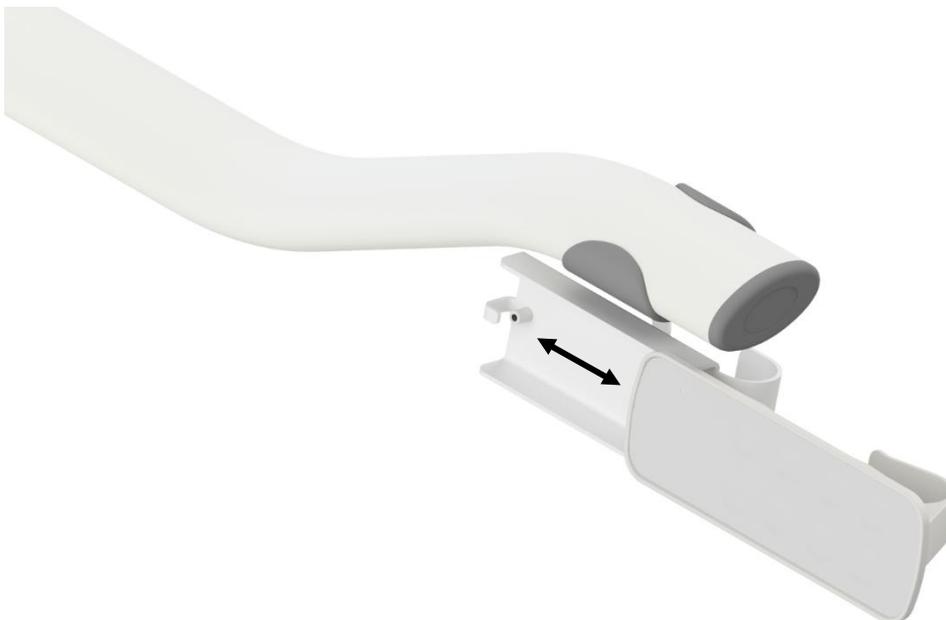

- e) If it's needed, the remote control can be locked to the Control Holder. This can be done by mounting the remote control on the Control Holder, then screwing in the locking screw in the threaded hole.

