



Fixed control for **Toilet Lifter**

User Guide


40-45070

Mounting instructions

a. Place the cover on the floor close to the lifting unit. Certain parts of this procedure can be easier, if the lifting unit is in its top position. If possible, do this before proceeding.




b. Install the cover for the control in the pre-drilled holes with the enclosed M3 nuts and bolts.

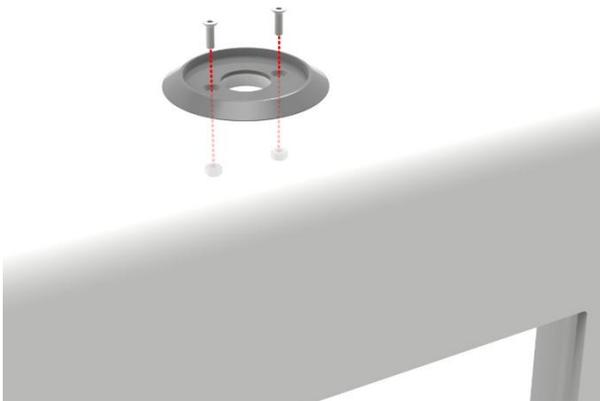



c. Remove the foil from the underside of the control and pull the chord through the center hole. Before pressing the control into the cover, make sure the "ROPOX"-logo is parallel with and facing the front of the cover.



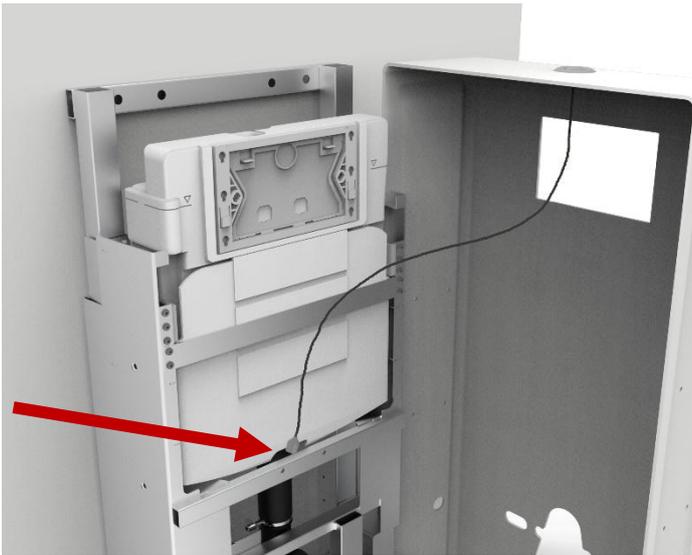

d. guide the cable along the front of the lifting unit and pull it behind the cross bar below the cistern, as indicated on the illustration

e. dismantle the control box from the lifting unit and insert the plug for the fixed control. Close the lid and re-attach the controlbox

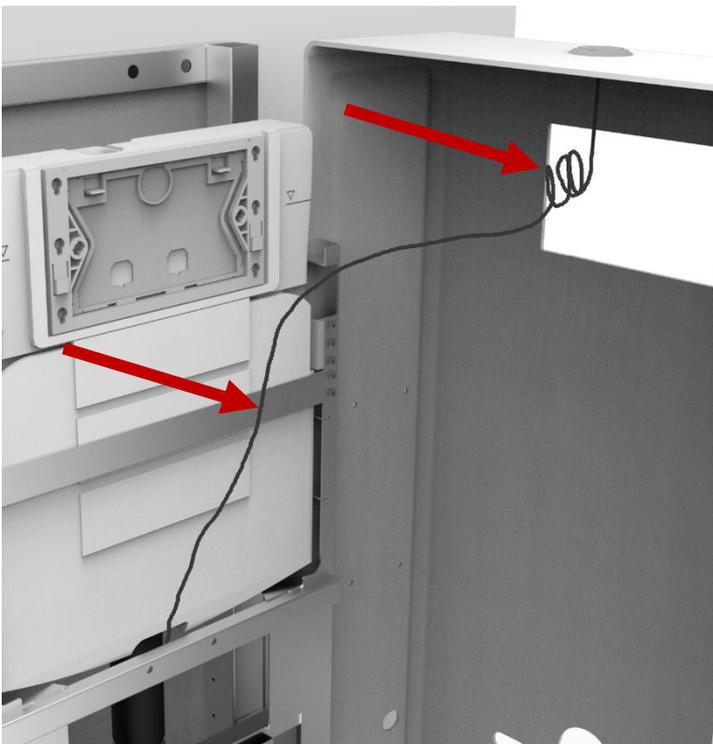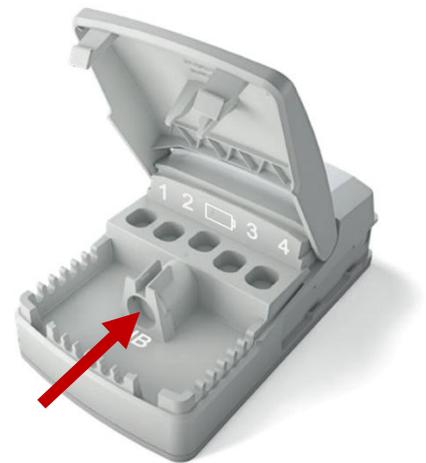

f. Use the attached strips to secure the wire to the belt as indicated.

Roll up the remaining wire and strip it together in a bundle, as close to the top of the cover as possible.

g. Check that the control works correctly. If it does head back to chapter 4.1.5 and continue at section "g" to finish assembling the toilet lifter.

If the control does not work correctly, double check all the connections and try again.