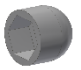

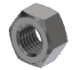


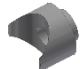

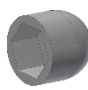
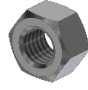


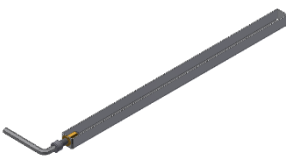


## Retrofitting of Bedrail for Vario and Mobilio

## Mounting of bedrail

### List of components

4x  M8	2x  M8x45	2x  M8
4x  M8	2x 	2x 
2x 	2x  M12	2x  M12
1x 	1-2x (Vario) 	1x (Vario) 

## Montering af sengehest - Mounting of bedrail - Montage des Schutzgitters

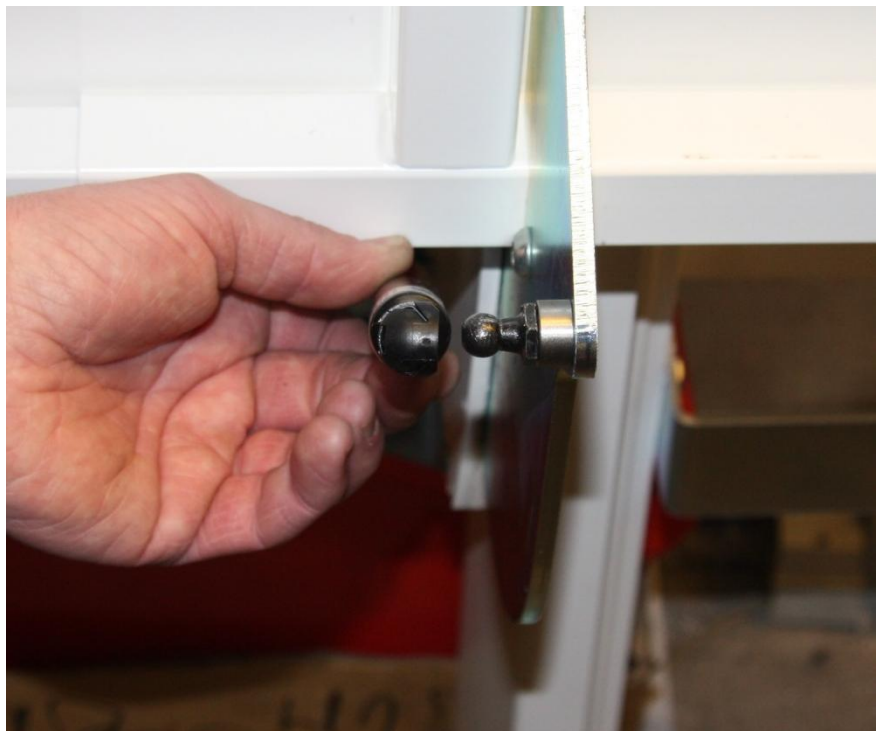
### Vario

Start by replacing the gas springs. Dismount gas spring by loosening the small spring plate on top of the gas spring. Do not remove it completely, but loosen it slightly so that the gas spring may be tipped out by means of a big screwdriver.



### Vario

Here the gas spring has been tipped out of the ball joint.



### Vario

At the opposite end of the gas spring, loosen the small spring plate slightly so that the gas spring may be pulled off.



### Vario

Press on the new gas spring at one end.



### **Vario**

Place the auxiliary tool on the gas spring and insert the bolt M10x40.



### **Vario**

Tighten the bolt so that the gas spring is compressed.  
Tighten it just so that it may be pressed on to the ball joint.



