

Ropox Side Cover, fixed

User Manual

Mounting Instructions

Keep this manual with the product at all times

Contents

Product information.....	3
Item number	3
Technical data.....	4
Mounting instructions	5
Maintenance	8
Cleaning	8
Complaints.....	9

General information

Product information



It is important to read this manual prior to mounting and use.



Product : Ropox Side Covers, fixed

Manufacturer: Ropox, DK-4700 Næstved, Tel. +45 55 75 05 00

Application – Ropox Side Covers are used together with Ropox Nursing Tables (Medi and Maxi). They provide safety for the person lying on the table and prevent objects and persons from getting jammed during height adjustment of the table.

Item number

Size of side cover	Item number
75.4 cm x 23.5 cm	50-50688

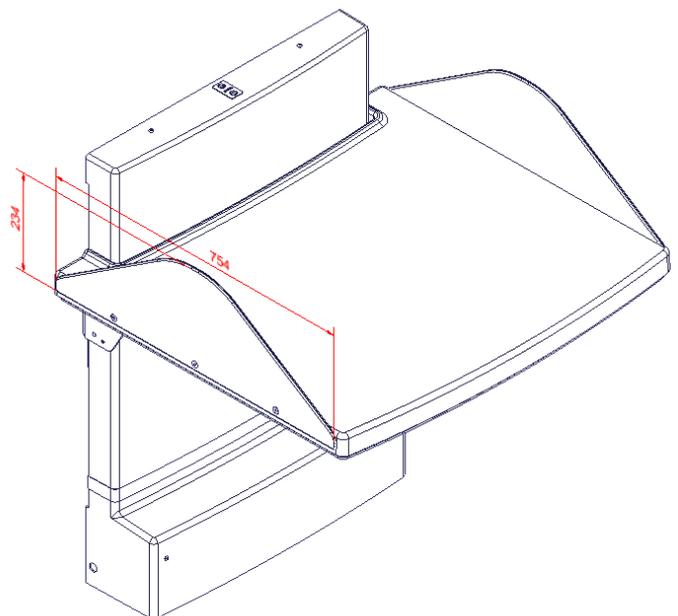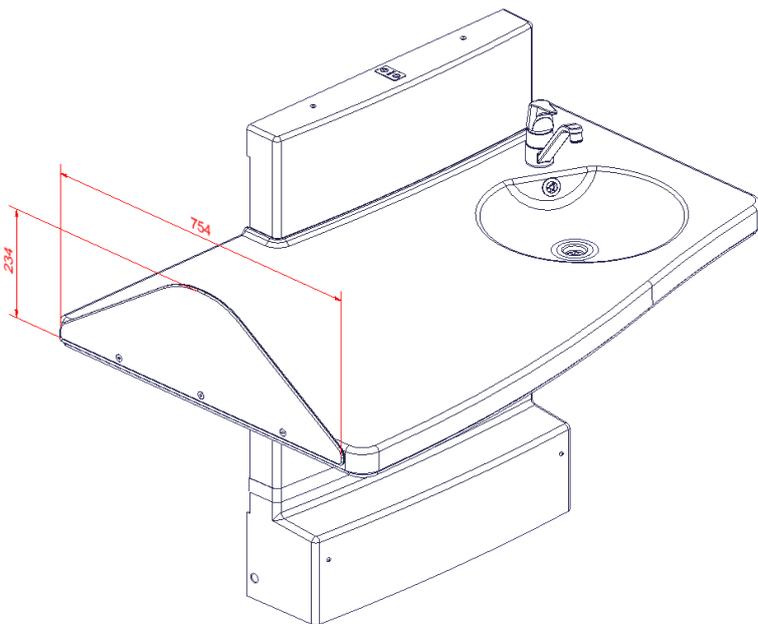


General information



Technical data

Material – fittings: Steel sheet St.37, powder-coated, white RAL 9010 mat


Material – cover plate: White 12 mm compact laminate

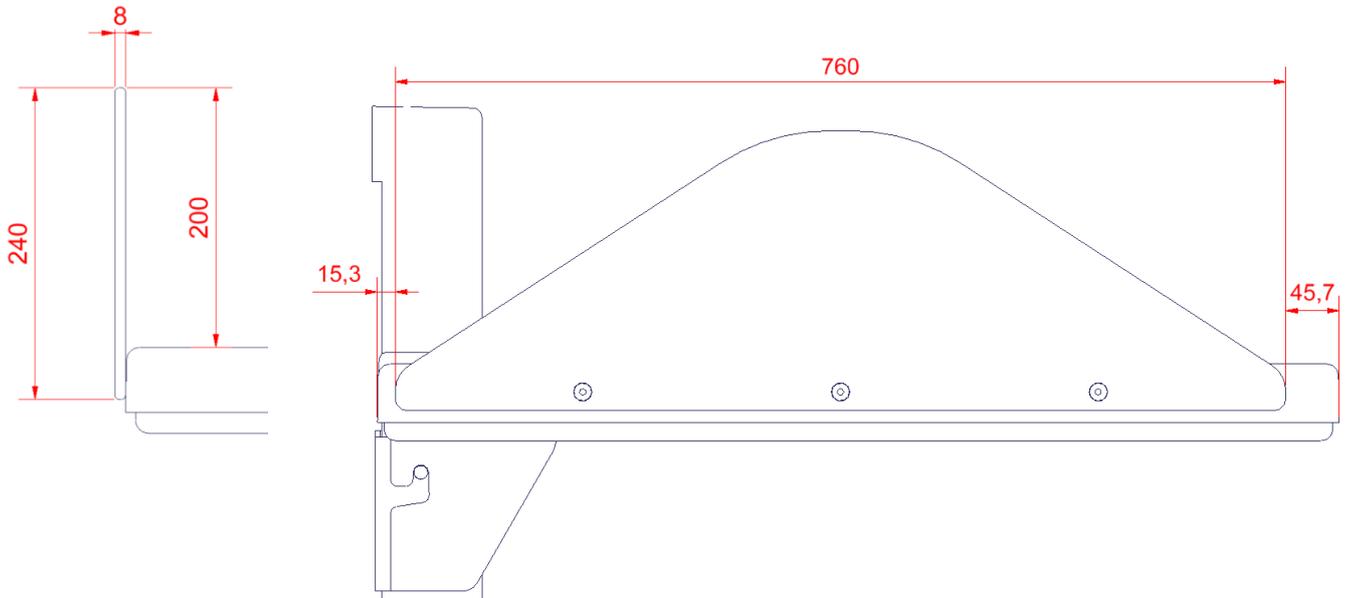


Mounting instructions

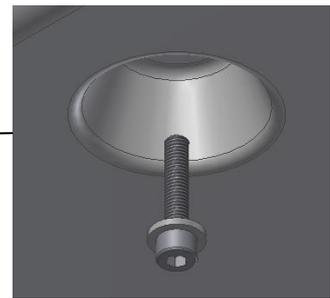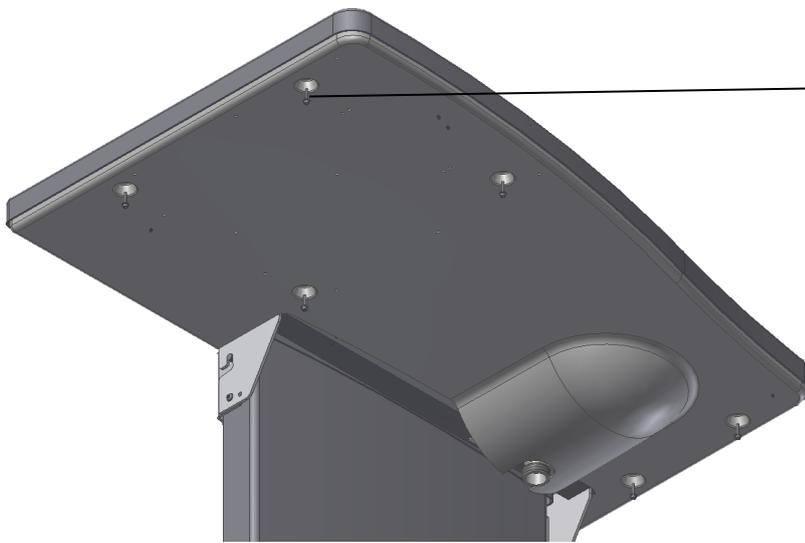

1. Remove the six bolts fastening the bottom plate to the tabletop.



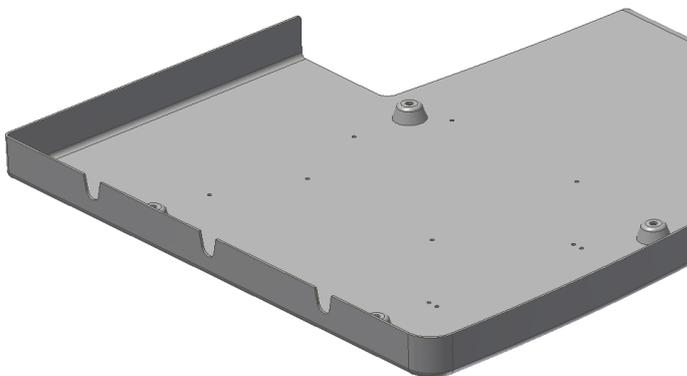


2. Cut three notches in the bottom plate according to the measures specified below

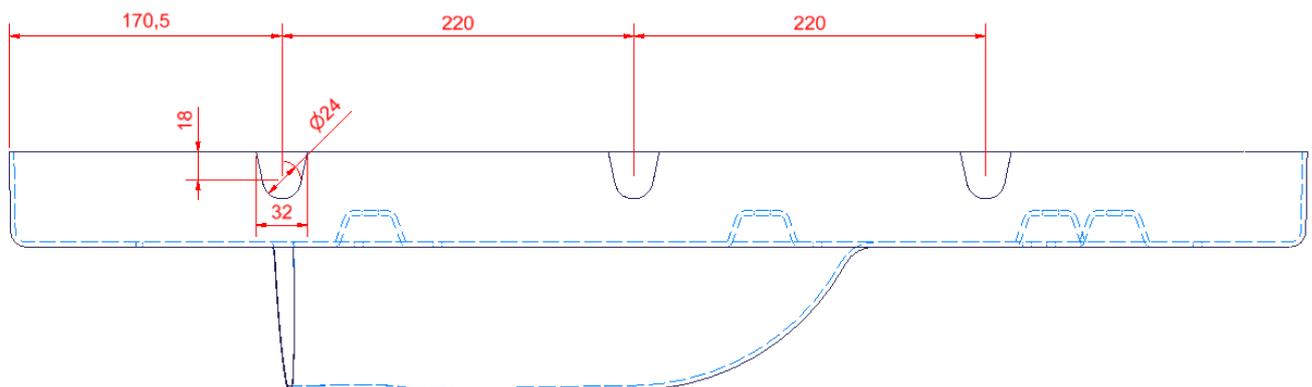


Mounting instructions


3. Drill three holes in the tabletop according to the measures specified below.

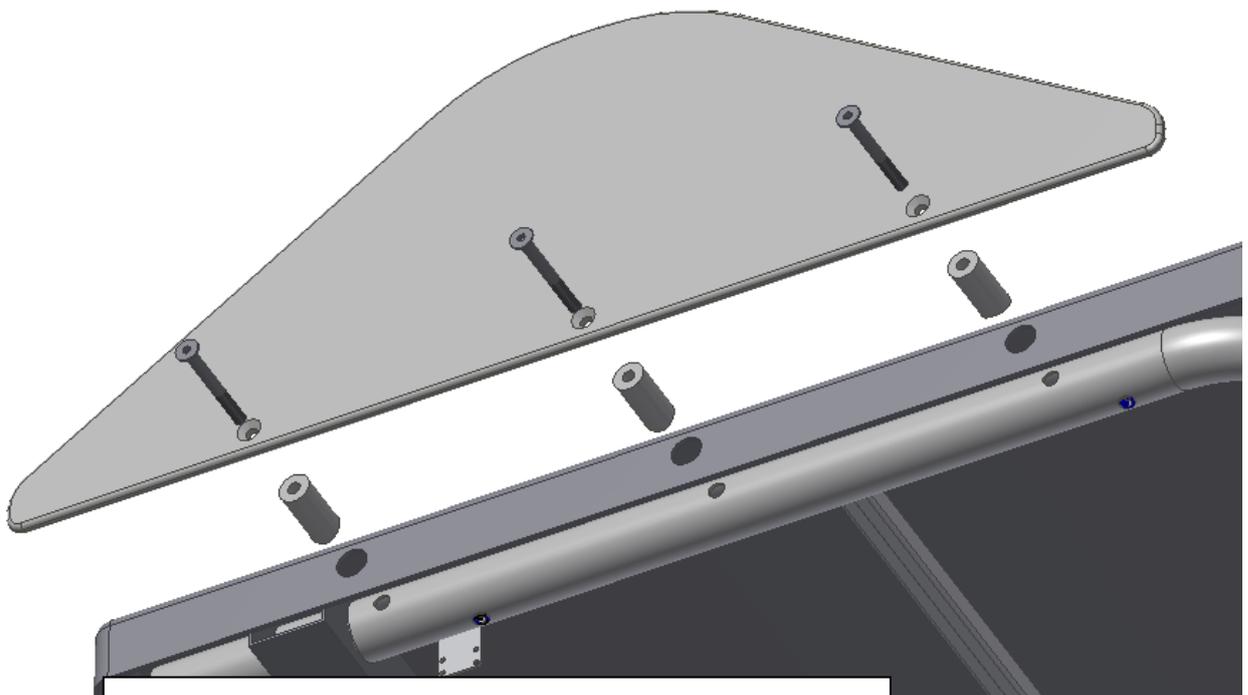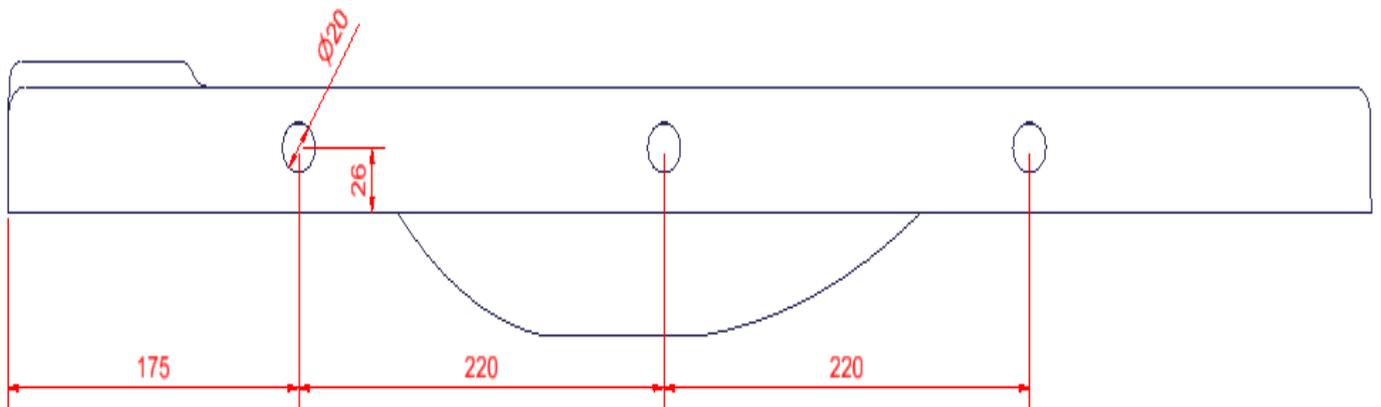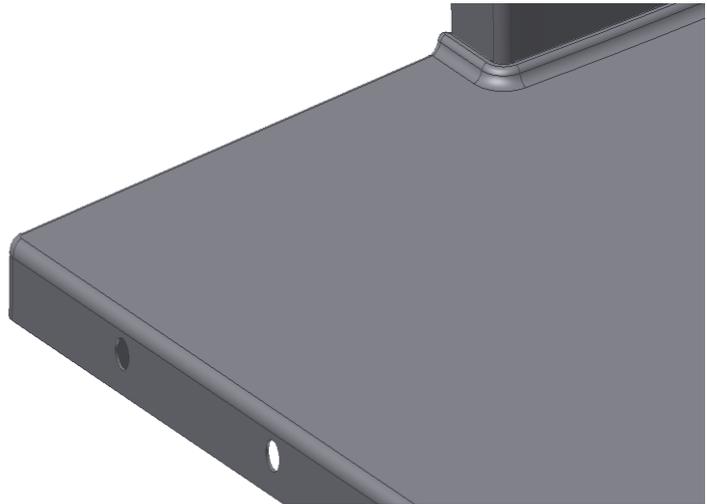

4. Fasten the side cover to the top frame through the three holes.

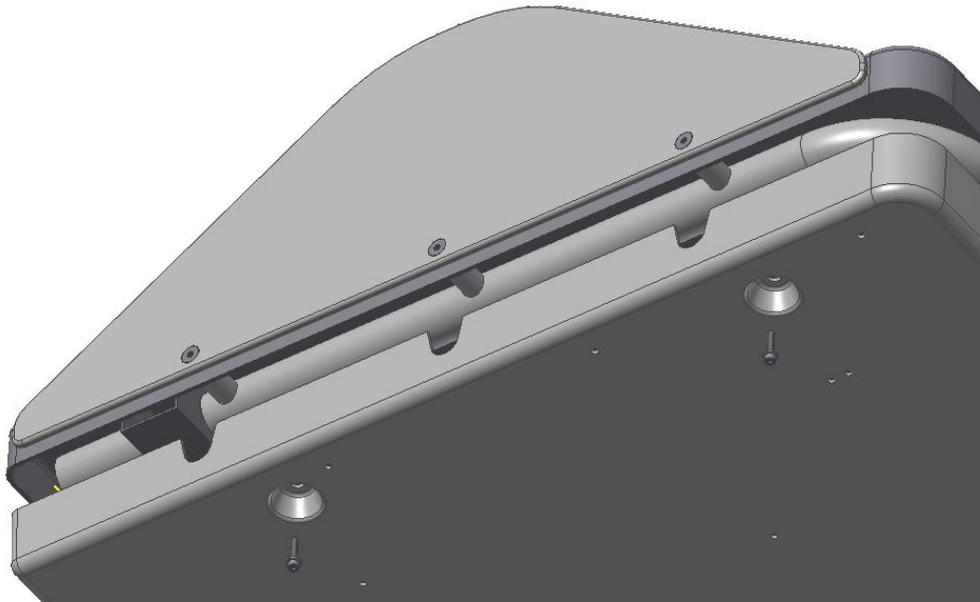

5. Mount the bottom plate using the six bolts.

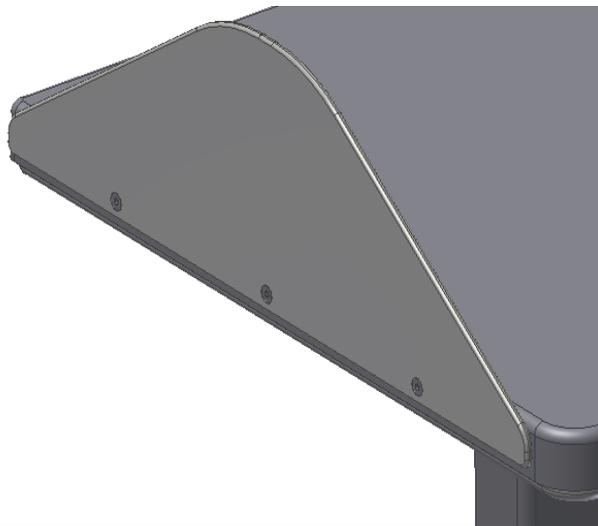

6. The side cover has been mounted and the nursing table is ready for use.

Maintenance

Ropox side covers are maintenance-free. However, for safety reasons we recommend regular inspection of all assembly bolts to ensure correct and secure fastening.

Cleaning

Simply clean the product with a soft cloth or sponge with soapy water of max. 60°C (pH 7) or cleaning alcohol.

Important:

Do not use scouring powder or other abrasives, as they will scratch the surface. Also, avoid the use of cleaning utensils with sharp edges. Never use strong mineral acids or alkaline products.

Complaints

Please see General Terms of Sale and Delivery on www.ropox.com



Ropox A/S

Ringstedgade 221
DK – 4700 Naestved
Tel.: +45 55 75 05 00
E-mail: info@ropox.dk
www.ropox.com