



# Svancare Bidet Elegant for Toilet lifter Installation manual

40-44276

TF 200.02.0025 / Date 21-11-2025

# Table of Contents

1. Componentlist.....	3
2. Installation of product.....	4
2.1 Make a cut-out in the outer cover, as shown.....	4
2.2 Mount the power socket on the frame.....	4
2.3 Connect nipples to t-joint and t-joint to water.....	5
2.4 Pull the bidet pipe through the frame as shown.....	5
2.5 Adjust the threaded rods and mount the outer cover .....	6
2.6 Connect the power cable for bidet to the power socket .....	6
2.7 Pipes and wires are pulled through the bidet, as shown.....	7
2.8 Shorten the ferrule and flush pipe .....	7
2.9 Mount the seal ring by the flush pipe and mount the ferrule and flush pipe in the bidet...	8
2.10 Mount the bidet on the threaded rods.....	8
2.11 Connect the power cable and water pipe to the bidet seat.....	9
2.12 Open the water in the wall and the filter and mount the bidet seat on the bowl.....	9
2.13 Mount the press pad on the cistern and test that everything works.....	10

# 1. Componentlist

	<b>1x Svancare Bidet Elegant</b> 40-44276
 	<b>1x T-joint 1/2"</b>
 	<b>2x Hex Nipple 1/2"</b>
 	<b>1x 1m water pipe 1/2"</b>
 	<b>1x Power socket with 2+ sockets</b>
 	<b>1x Cable mount</b>
 	<b>1x Cable tie</b>



————— *Not supplied by Ropox*

# 2. Installation of product

## 2.1 Make a cut-out in the outer cover, as shown




## 2.2 Mount the power socket on the frame By tying it to the cable mount or drilling two holes for the cable tie.




### 2.3 Connect nipples to t-joint and t-joint to water

Connect cistern pipe and bidet pipe to t-joint

*Make sure the water is turned off!*

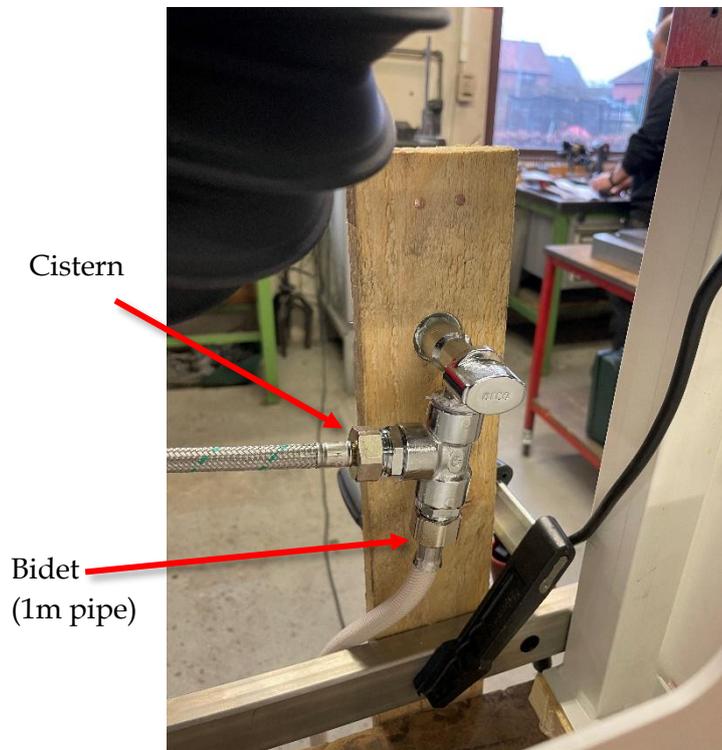


### 2.4 Pull the bidet pipe through the frame as shown

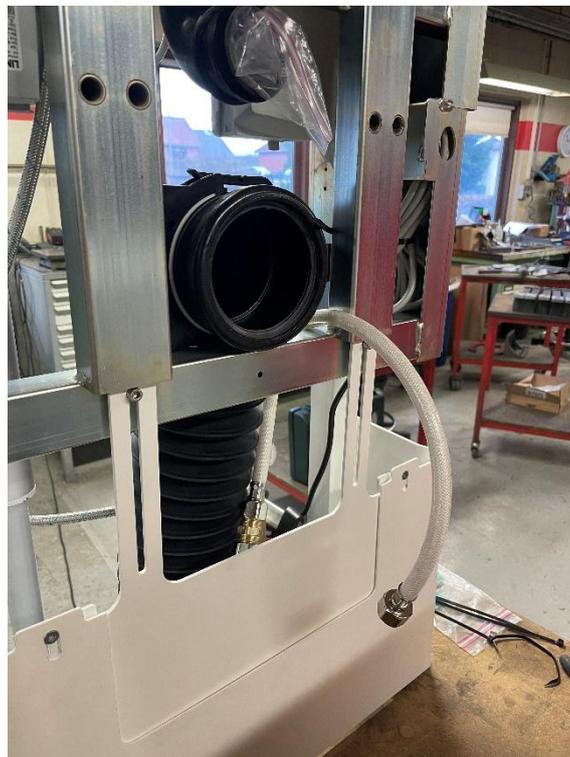

## 2.5 Adjust the threaded rods and mount the outer cover

Adjust threaded rods to 70mm and mount them in the middle most holes

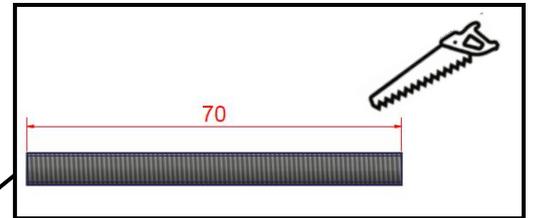

## 2.6 Connect the power cable for bidet to the power socket


Bidet water pipe is connected to the included filter. Close the water on the filter, then turn it on, on the wall.



## 2.7 Pipes and wires are pulled through the bidet, as shown



## 2.8 Shorten the ferrule and flush pipe Ferrule to 70mm and flush pipe to 48mm

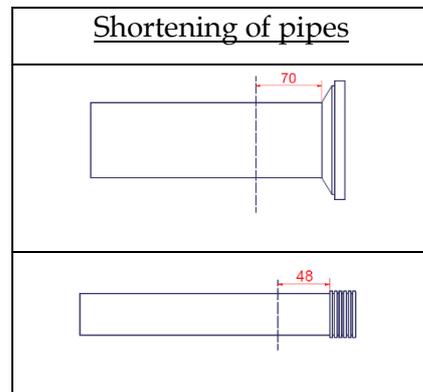

## 2.9 Mount the seal ring by the flush pipe and mount the ferrule and flush pipe in the bidet

Slide the bidet on the toilet lifter



## 2.10 Mount the bidet on the threaded rods

With the included shims and nuts



## 2.11 Connect the power cable and water pipe to the bidet seat



*Turn on the filter  
for water!*



## 2.12 Open the water in the wall and the filter and mount the bidet seat on the bowl

With the included screws and screw caps



## 2.13 Mount the press pad on the cistern and test that everything works

Electric and water

