

### Disengaging the guard rail




Pulling these two levers upwards unlocks the guard rail, allowing it to be dropped down and pushed down beneath the Shower/Changing bed.  
*It's locked into place by friction, meaning you don't have to push till you hear it click.*

### Engaging the guard rail



To engage the guard rail, simply grab it by the white rail and pull it towards yourself and upwards until it snaps into place.

### Storing it away

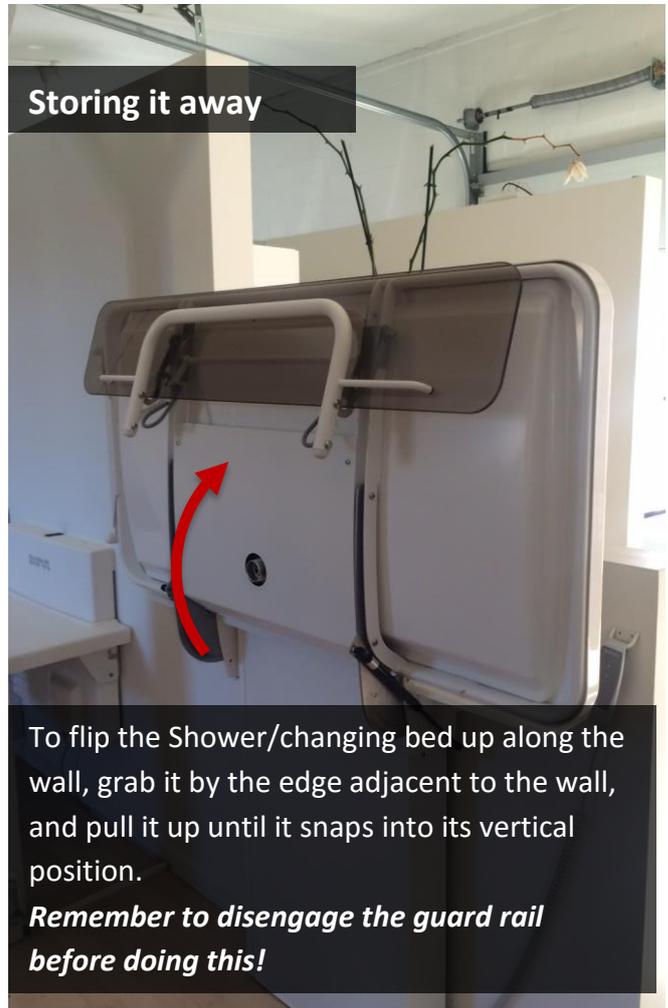

To flip the Shower/changing bed up along the wall, grab it by the edge adjacent to the wall, and pull it up until it snaps into its vertical position.

***Remember to disengage the guard rail before doing this!***