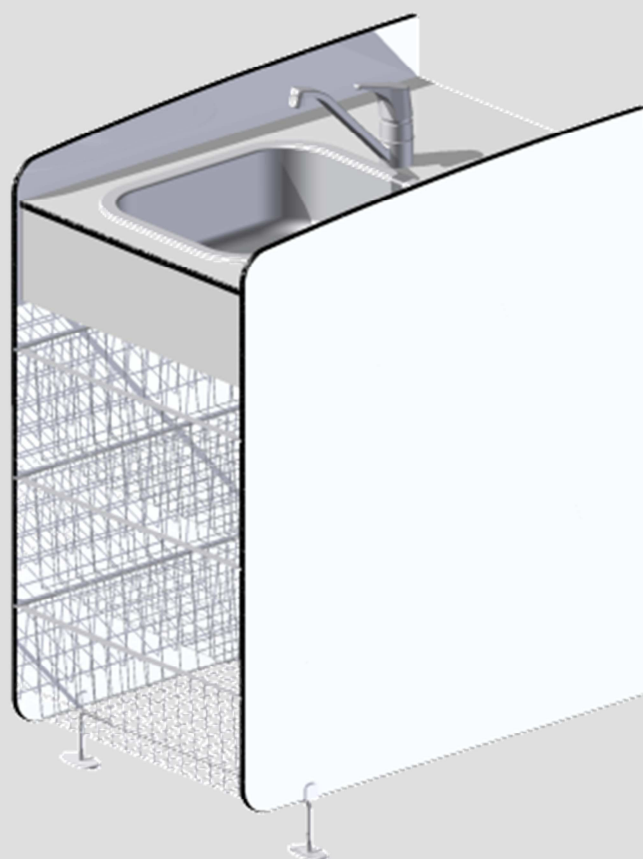
Ropox Kurve unit

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Brugermanual

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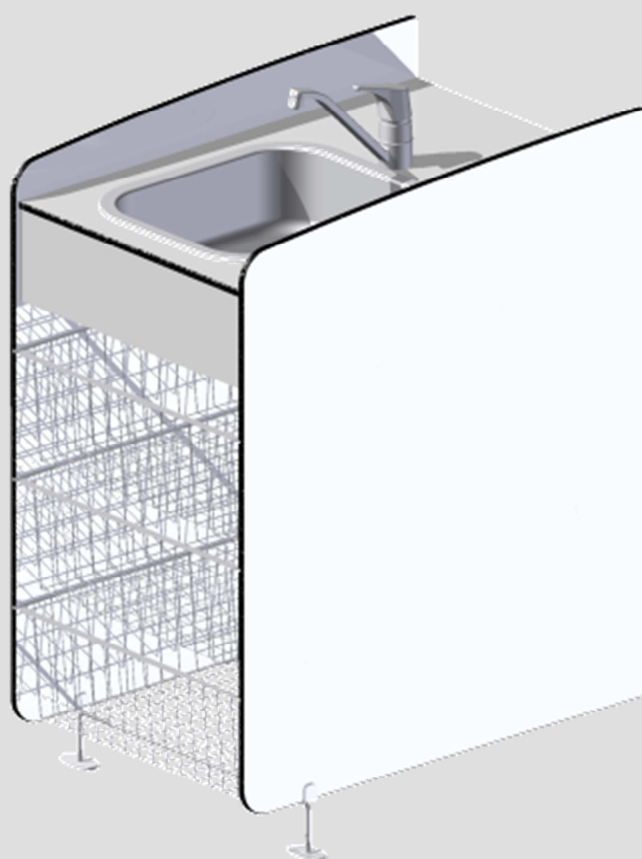
Mounting Instructions

User Manual

Keep this manual with the product at all times

Contents

1. Technical data.....	15
2. Models	16
3. Dimensions	16
4. List of components	17
5. Mounting instructions	19
6. Cleaning/ maintenance.....	222
7. CE declaration.....	233
8. Complaint	244

1. Technical data

Product name:	Ropox Basket unit
Item numbers:	50-50696 – Ropox Basket unit with wash basin 50-50697 – Ropox Basket unit without wash basin
Working height:	95 cm
Material:	Steel 37
Table top and side covers:	Compact Laminate
Surface treatment:	Powder coating
Max. load on frame (static):	30 kg evenly distributed over the table top
Complaint:	See Complaint
Manufacturer	Ropox A/S, DK-4700 Næstved, Tel.: +45 55 75 05 00 e-mail: info@ropox.dk , www.ropox.dk

2. Models

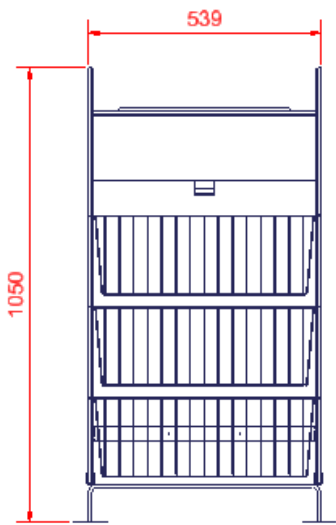
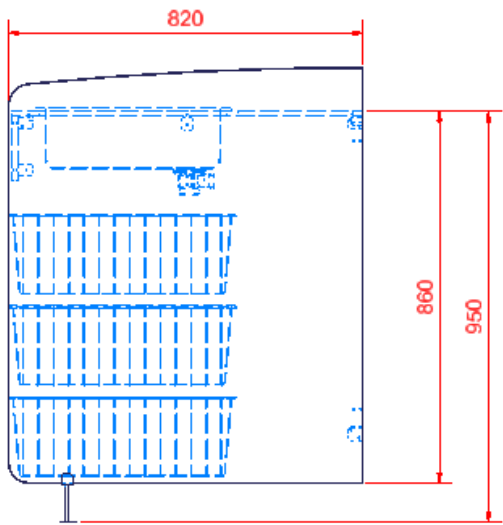
With wash basin

Without wash basin

3. Dimensions

4. List of components

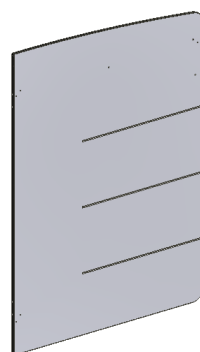
Side cover left 1 pc.

Item number: 50*50670-211

Side cover right 1 pc.

Item number: 50*50670-212

Tabletop 1 pc.

Provided with brackets

Item number:

Tabletop with cutout for wash basin: 50*50670-213

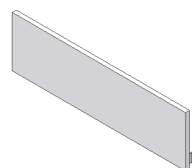
Tabletop without cutout for wash basin: 50*50670-225

Front fascia 50*50670-214 1 pc.

Provided with brackets

Wire baskets 3 pcs.

Bracket 50*50670-215: 3 pcs.
2 pcs. is mounted on the tabletop

Supporting foot 50*50670-218: 1 pc.

Bracket 50*50670-217 4 pcs.
2 pcs. is mounted on the tabletop
2 pcs. is mounted on the front fascia

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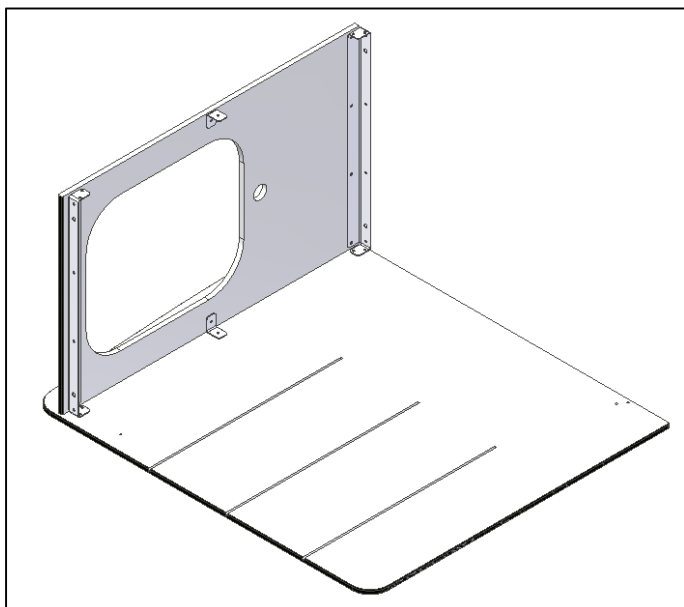
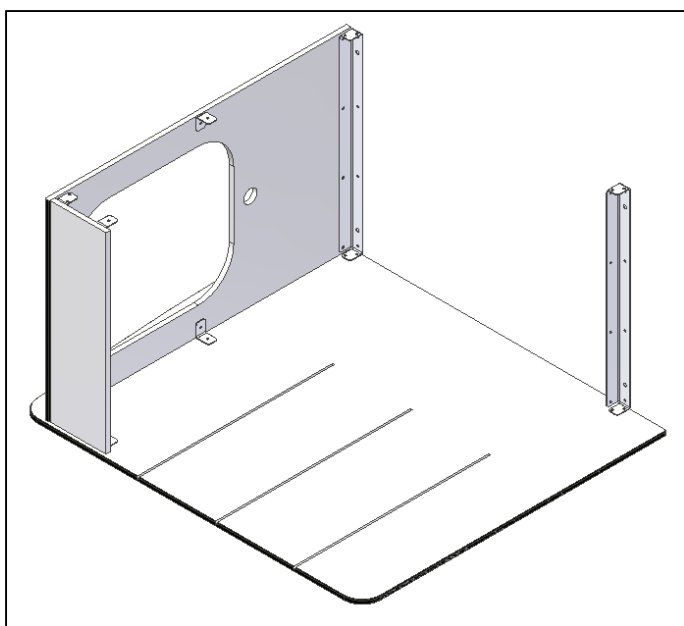
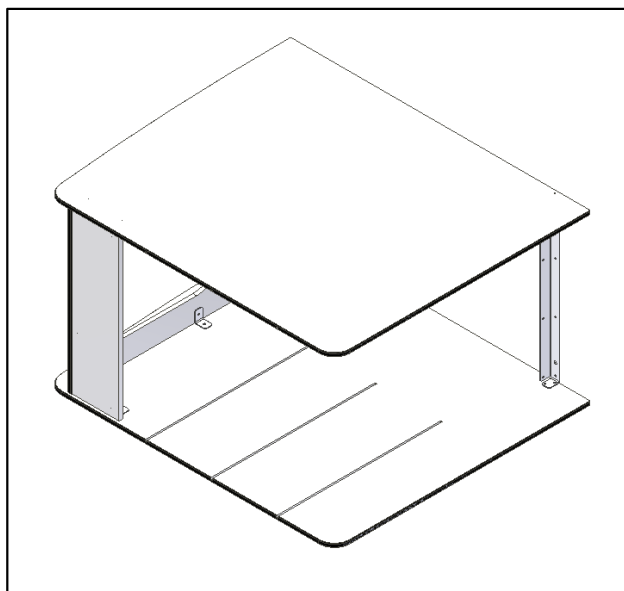
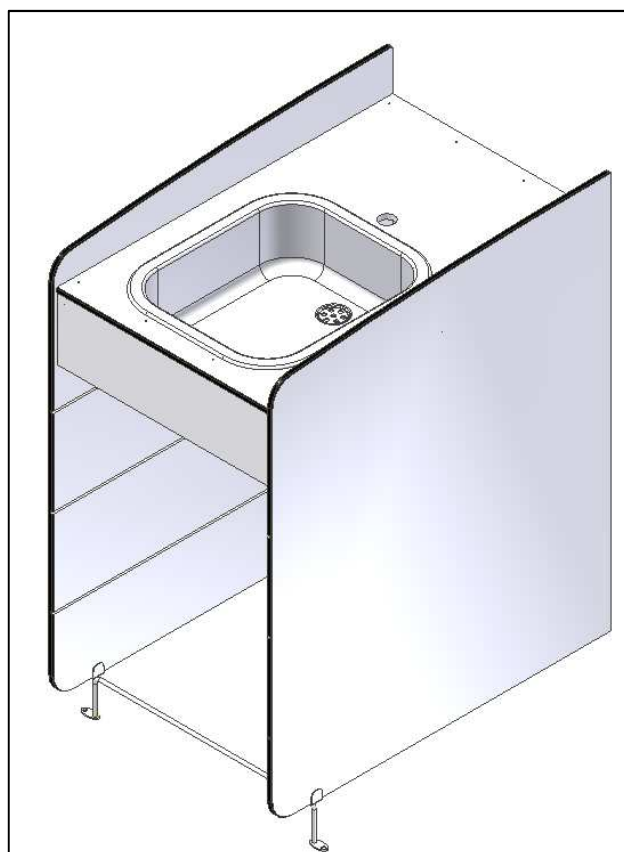
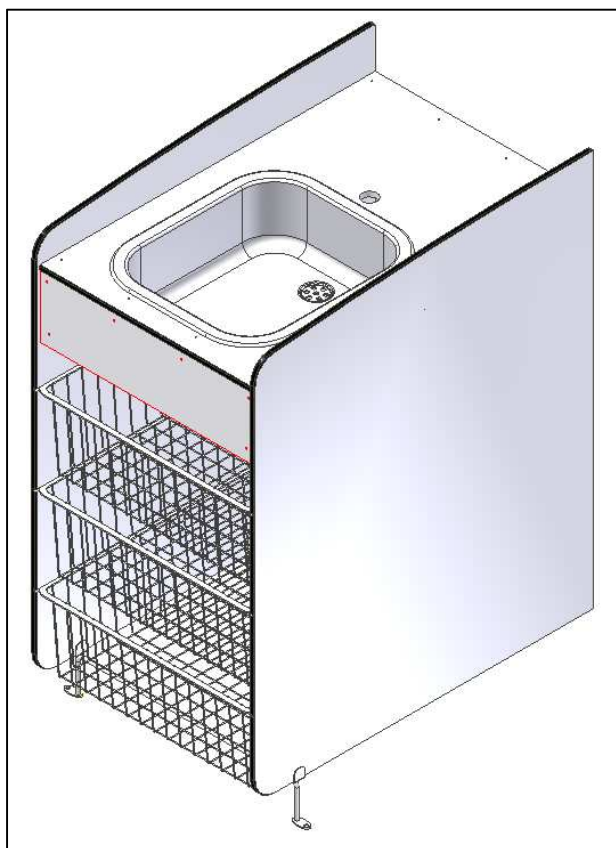
6. Cleaning/ maintenance

Ropox Basket unit are maintenance-free. However, for safety reasons we recommend regular inspection of all assembly bolts to ensure correct and secure fastening.

Cleaning:

Do not use a high-pressure washer to clean this product.

Simply use a soft cloth or sponge with soapy water of max. 60°C (pH 7) or cleaning alcohol.

Important:

Do not use scouring powder or other abrasives, as they will scratch the surfaces.

Also, avoid the use of cleaning utensils with sharp edges. Never use strong mineral acids or alkaline products.

7. CE declaration

Ropox Basket unit with wash basin: 50-50696

Ropox Bsket unit without wash basin: 50-50697

Are in accordance with the following directive and standards:

DIRECTIVES

Council directive 93/42/EEC of 14 June 1993 concerning medical devices

Machinery Directive No. 2006/42/EC, amended by Directive No. 98/37/EC.

STANDARDS

DS/EN ISO 12182: 2012

DS/EN ISO 14971: 2001

Dato: 01.05.15

Tommy Jensen
Adm. direktør

8. Complaint

See General Terms of Sale and Delivery on www.ropox.dk

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